


Seat Plate and Front Actuator Support Bracket For Elevating Seat

NOTE: For 3rd Generation Storm Series Chairs Only. (ESS-PTO) Elevate w/ PTO was discontinued 10/16/06. After 10/15/06, refer to the Formula CG Powered Seating System Parts Catalog Form #06-106 for Elevate components on a 3G Storm Series.

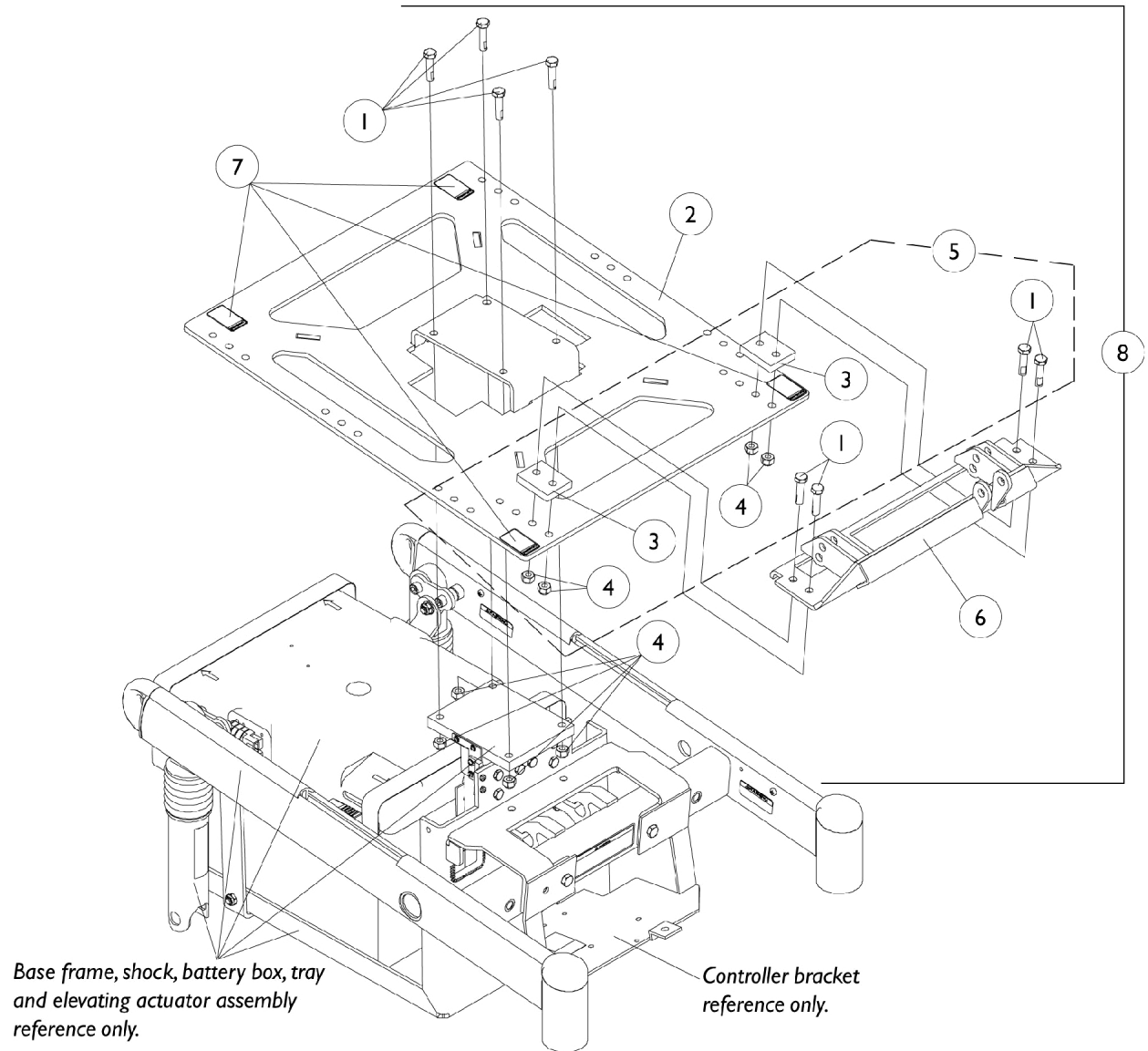

Seat Plate and Front Actuator Support Bracket For Elevating Seat

NOTE: For 3rd Generation Storm Series Chairs Only. (ESS-PTO) Elevate w/ PTO was discontinued 10/16/06. After 10/15/06, refer to the Formula CG Powered Seating System Parts Catalog Form #06-106 for Elevate components on a 3G Storm Series.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930337-NLA	NLA - Package, Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	25	8
2		1109615	Plate, Seat P.T.O. with Warning Decals	1	1
3		1108225-NLA	NLA - Spacer, Seat Plate, Black	1	2
4		1026158	Locknut (5/16-18)	1	8
5	A	1109081-NLA	NLA - Hardware, Front Actuator Support Bracket Mounting	1	1
6		1108230-NLA	NLA - Bracket, Actuator Support, Front, Black	1	1
7		1079203	Decal, Warning Pinch Point	1	4
8	B	1108427-NLA	NLA - Bracket Assembly, Seat Plate and Front Actuator Support	1	1

NOTE: A - Includes items 1, 3 and 4
B - Includes items 1-4, 6 and 7