


# Seat Frame Support & Mounting Hardware

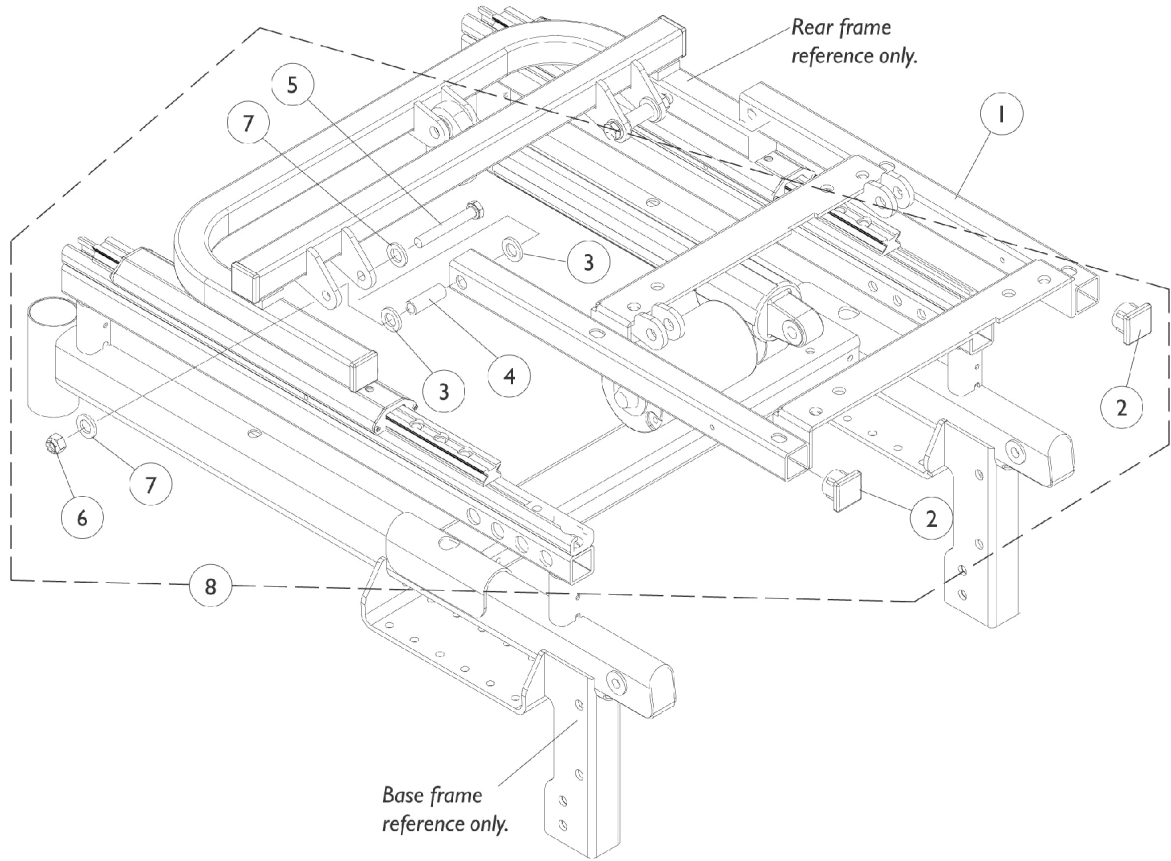

## Seat Frame Support & Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1087626-NLA	NLA - Support, Seat Frame	1	1
1	C	1121053-NLA	NLA - Support, Seat Frame and Spacer Plate with Hardware	1	1
2		1071237	Plug Button, Square, Black (1")	1	2
3		0055800-NLA	NLA - Washer, Nylon (7/16 x 3/4 x 3/32")	1	4
4		0060300-NLA	NLA - Bushing, Bronze (5/16 x 7/16 x 1-1/4")	1	2
5		1070412-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	2
6		1026158	Locknut (5/16-18)	1	2
7		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
8	A	1106540-NLA	NLA - Hardware, Seat Frame Support Mounting	1	1

NOTE: A - Includes items 2-7  
 B - Seat support frame revised 2/17/03 to accommodate a spacer plate for 14"-15" wide seat frames. For 14"-15" wide PTO models made before 2/17/03 order Item 1 part number 1121053. The revised support frame is still used for all 16"-22" wide PTO models.  
 C - Used for 14"-15" wide PTO models made before 2/17/03. Spacer plate and hardware included. Spacer plate can be found on the A.S.B.A. Seat Mounting Hardware page.