
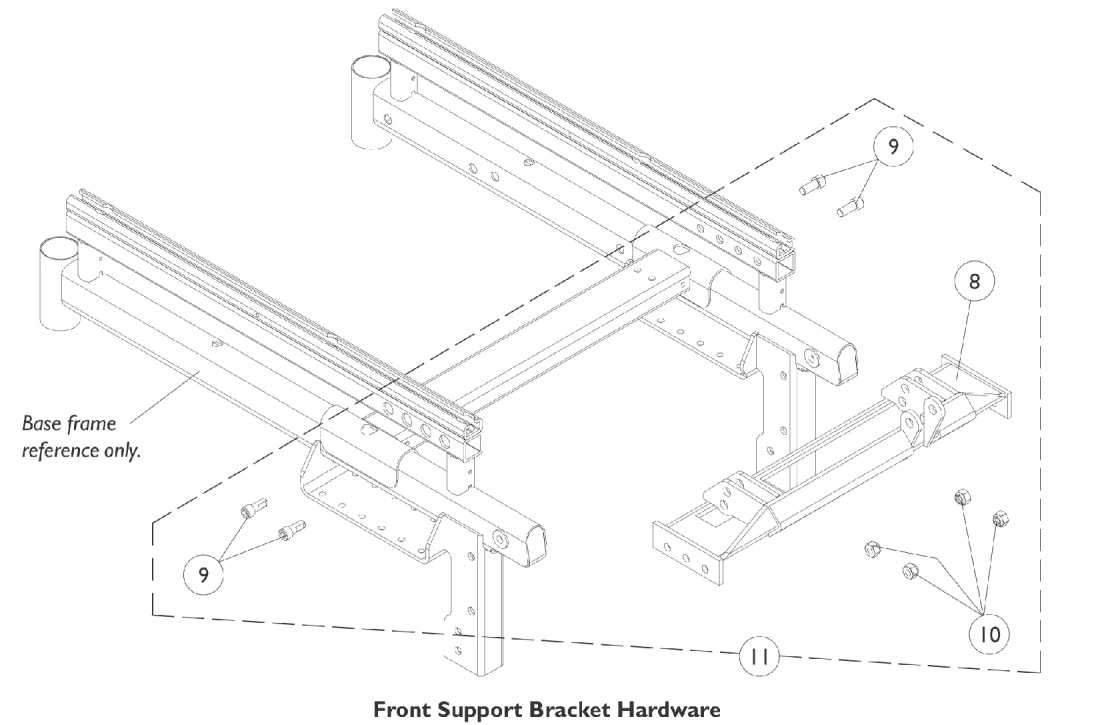


Tilt Link & Front Support Bracket Hardware

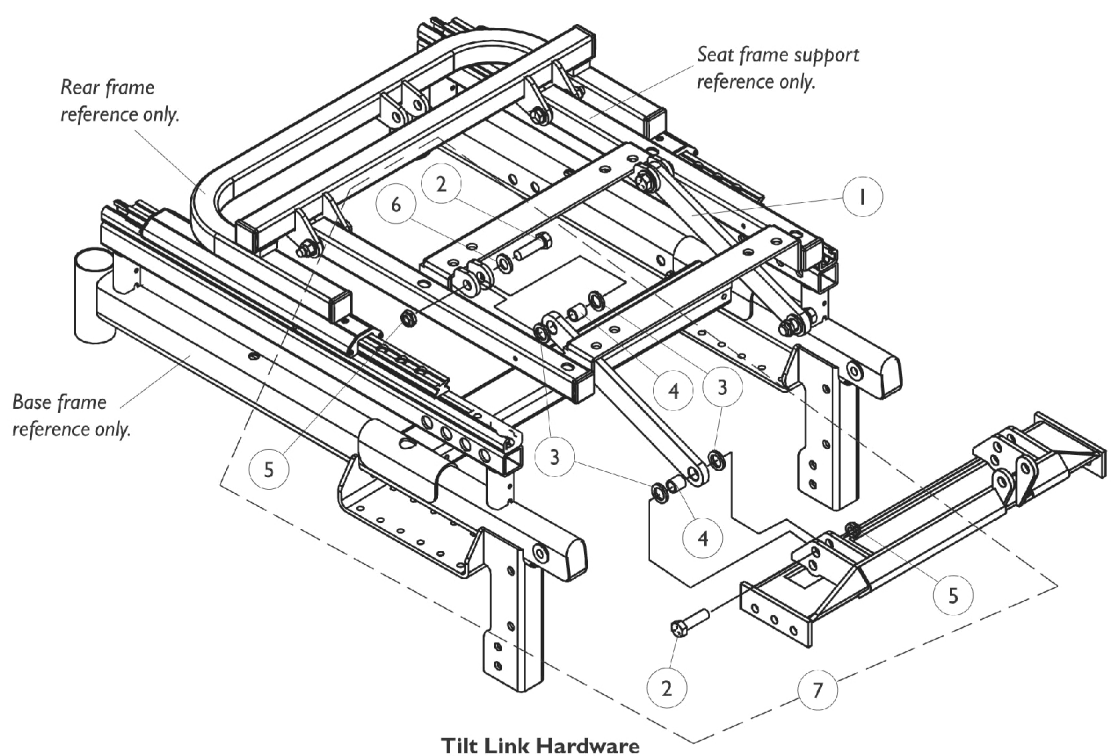

Tilt Link & Front Support Bracket Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1087634-NLA	NLA - Link, Tilt, Black	1	2
2		1107708-NLA	NLA - Screw, Shoulder Hex Socket Head (3/8 x 3/4", 5/16-18)	1	4
3		0139900	Washer, Nylon (1/2 x 3/4 x 3/32")	1	8
4		1087636-NLA	NLA - Bushing, Bronze (3/8 x 1/2 x 1/2")	1	4
5		1028769	Locknut (5/16-18)	1	4
6		20105X000-NLA	NLA - Washer, Flat (1/4 x 5/8 x 1/16")	1	2
7		1106541-NLA	NLA - Hardware, Tilt Link Mounting	1	1
8		1106393-NLA	NLA - Bracket, Support, Front, Black	1	1
9		1086944-NLA	NLA - Screw, Socket Head (5/16-18 x 3/4")	1	4
10		1026158	Locknut (5/16-18)	1	4
11	B	1106551-NLA	NLA - Kit, Front Support Bracket Hardware	1	1
NOTE: A - Includes items 2-6 B - Includes items 8-10					