


# ASBA Seat Frame and Hardware (14"-15" Wide)

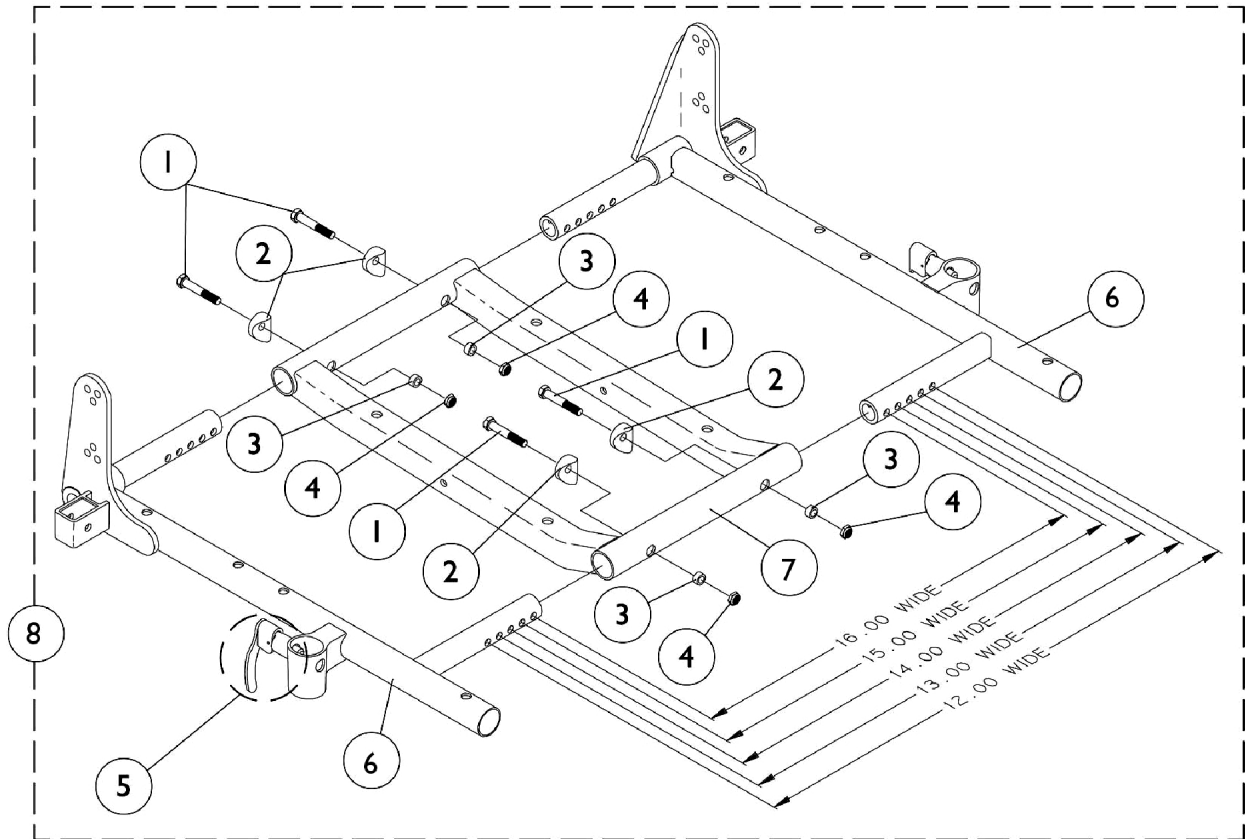

## ASBA Seat Frame and Hardware (14"-15" Wide)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	4
2		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
3		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		1025736	Locknut (1/4-20)	1	4
5		47471X074	Spring, Arm Lock	1	1
5		1028594	Pin, Arm Lock, Black	1	1
5		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
6-8		See Chart	#s 6-8 ASBA Seat Frame and Hardware (14"-15" Wide)	1	1

### #s 6-8 Seat Frames

14" - 15" Wide

Item Number	Description	14"-15" Deep	16"-17" Deep	18"-19" Deep	20"-21" Deep	22" Deep
6	Seat Frame, Rt. See Note(s) A	1070473-NLA	1070473-NLA	1070526-NLA	1070528-NLA	1070530-NLA
6	Seat Frame, Lt. See Note(s) A	1070474-NLA	1070474-NLA	1070527-NLA	1070529-NLA	1070531-NLA
7	Seat Frame Center	1070476-NLA	1070476-NLA	1070476-NLA	1070533-NLA	1070533-NLA
8	Seat Frame Assembly See Note(s) B	1070469-NLA	1070469-NLA	1070534-NLA	1070535-NLA	1070536-NLA
NOTE: A - Seat frame includes item 5 arm lock assembly. B - Includes items 1-7						