
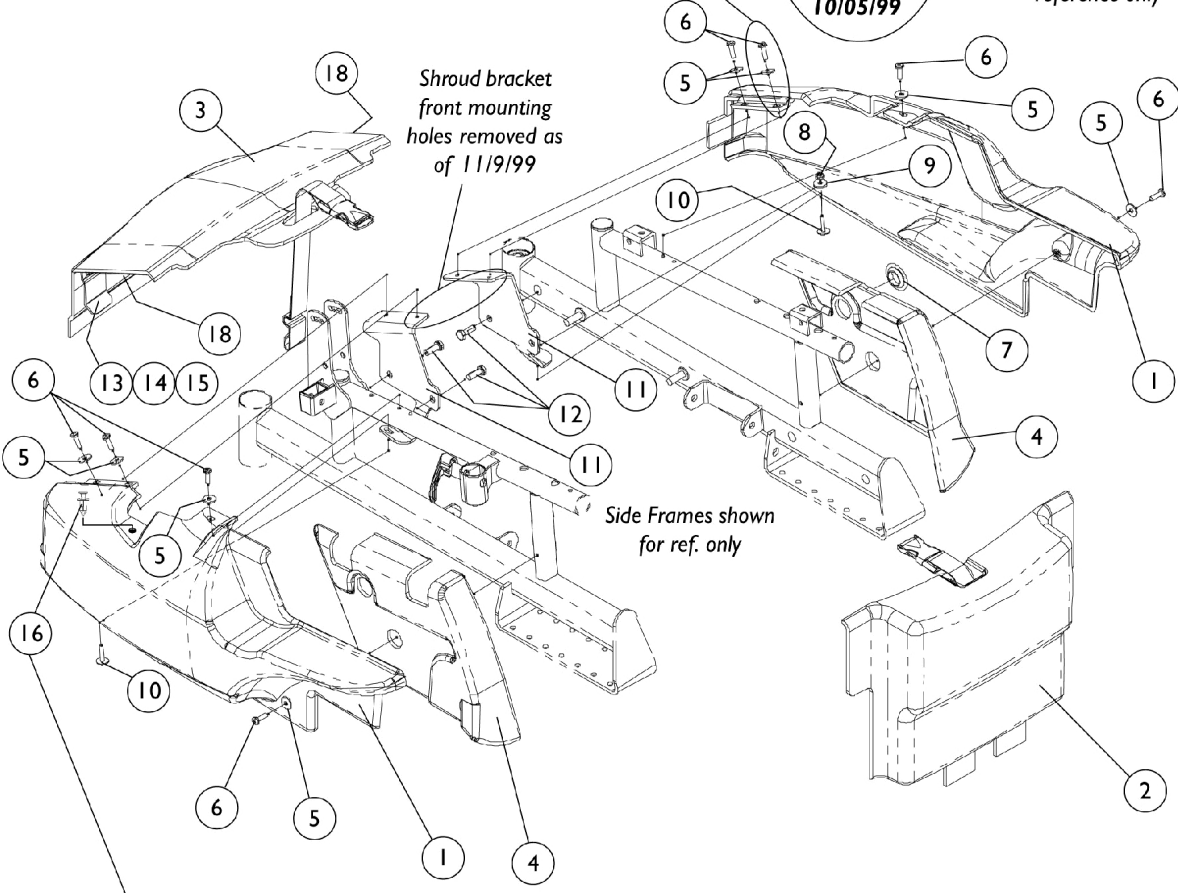


# Shroud Covers (Before 3/1/00)

**NOTE:** After 10/05/99 item 5 and 6 shown were replaced with a dual lock fastener for both sides and adding dual lock to the trunk lid as well. See detail "A" for side panel shroud application.

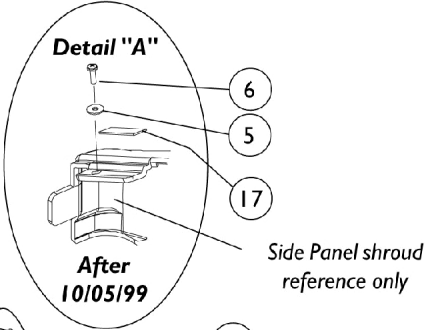

**NOTE:** Panel fastener is not used on high back captain seat shroud covers

# Shroud Covers (Before 3/1/00)

## Item #'s 1-4 Shroud Covers

Item Number	Description	Bright Plum Metallic	Bright Blue Metallic	Blue Green Chameleon	Candy Apple Red	Bright Silver Metallic
1	Side Panel with Rubber Bushing (Captain & Sling Seat) - Right	1087147-NLA See Note(s) B	1087148-NLA See Note(s) B	1087149-NLA See Note(s) B	1087150-NLA See Note(s) B	1087151-NLA See Note(s) B
1	Side Panel with Rubber Bushing (Captain & Sling Seat) - Left	1087152 See Note(s) B	1087153-NLA See Note(s) B	1087154-NLA See Note(s) B	1087155-NLA See Note(s) B	1087156-NLA See Note(s) B
2	Battery Cover Ranger II Only - 16" W	1085576-NLA See Note(s) C, A	1085577-NLA See Note(s) A, C	1085578-NLA See Note(s) A, C	1085579-NLA See Note(s) A, C	1086624-NLA See Note(s) A, C
2	Battery Cover Ranger II Only - 18" W	1085580-NLA See Note(s) A, C	1085581-NLA See Note(s) A, C	1085582-NLA See Note(s) C	1085583 See Note(s) C	1086625-NLA See Note(s) C, A
2	Battery Cover Ranger II Only - 20" W	1085584	1085585	1085586-NLA See Note(s) A	1085587	1086626-NLA See Note(s) A
3	Trunk Lid - 16" W	1087157-NLA See Note(s) C, A	1087158-NLA See Note(s) A, C	1087159-NLA See Note(s) A, C	1087160-NLA See Note(s) A, C	1087161-NLA See Note(s) A, C
3	Trunk Lid - 18" W	1087162-NLA See Note(s) A, C	1087163-NLA See Note(s) C	1087164-NLA See Note(s) C	1087165-NLA See Note(s) C	1087166-NLA See Note(s) A, C
3	Trunk Lid - 20" W	1087167 See Note(s) C	1087168-NLA See Note(s) A	1087169-NLA See Note(s) C	1087170-NLA See Note(s) C	1087171-NLA See Note(s) A, C
4	Filler w/o Plug Button (Captain) - Black Only - Right	1088309-NLA	1088309-NLA	1088309-NLA	1088309-NLA	1088309-NLA
4	Filler w/o Plug Button (Captain) - Black Only - Left	1088310-NLA	1088310-NLA	1088310-NLA	1088310-NLA	1088310-NLA
4	Filler w/o Plug Button (Sling Seat) - Black Only - Right	1088311-NLA	1088311-NLA	1088311-NLA	1088311-NLA	1088311-NLA
4	Filler w/o Plug Button (Sling Seat) - Black Only - Left	1088312-NLA	1088312-NLA	1088312-NLA	1088312-NLA	1088312-NLA

## Shroud Covers (Before 3/1/00)

NOTE: A - Shroud is No Longer Available. See Shroud pages dated "After 2/29/00" for color conversion options.  
B - Decals not included. See page titled, "Packaging, Literature and Decals."  
C - Captain seat shrouds in 18" wide only.

## Shroud Covers (Before 3/1/00)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-4		See Chart	Item #'s 1-4 Shroud Covers	1	1
5	A	1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	8
6	A	1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	8
7		1089724-NLA	NLA - Plug Button, Black	1	2
8		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
9		1070594-NLA	NLA - Bushing, Rubber	1	1
10		1070575-NLA	NLA - Screw, SEMS w/ Washer (#10-32 x 7/8")	1	2
11		1088261-NLA	NLA - Bracket, Shroud, Black - Right	1	1
11		1088262	Bracket, Shroud, Black - Left	1	1
12		1050161-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1")	1	3
13		1070577-NLA	NLA - Plate, Trunk Clip	1	2
14		1070588-NLA	NLA - Screw, Phillips Truss Head (#10-32 x 1/2")	1	4
15		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
16	B	1088497-NLA	NLA - Fastener, Panel (5/8" W x 3/4" L)	1	1
17		1070555-NLA	NLA - Fastener, Dual Lock (5/8" W x 2" L)	1	2
18		1070556-NLA	NLA - Fastener, Dual Lock (1" W x 2" L)	1	2
	C,D	1087172	Hardware, Side Panel Attaching	1	1

NOTE: A - After 10/05/99 quantity changed from 8 to 6 when dual lock fastener was added to side panel shrouds and trunk lid. See detail "A" on previous page.  
 B - Sling Seat Only.  
 C - Includes Items 5, 6, 10, 11, 12, 15 and 17.  
 D - Not Shown.