


Seat Frame Mounting Hardware (Adult 16-24" Wide)

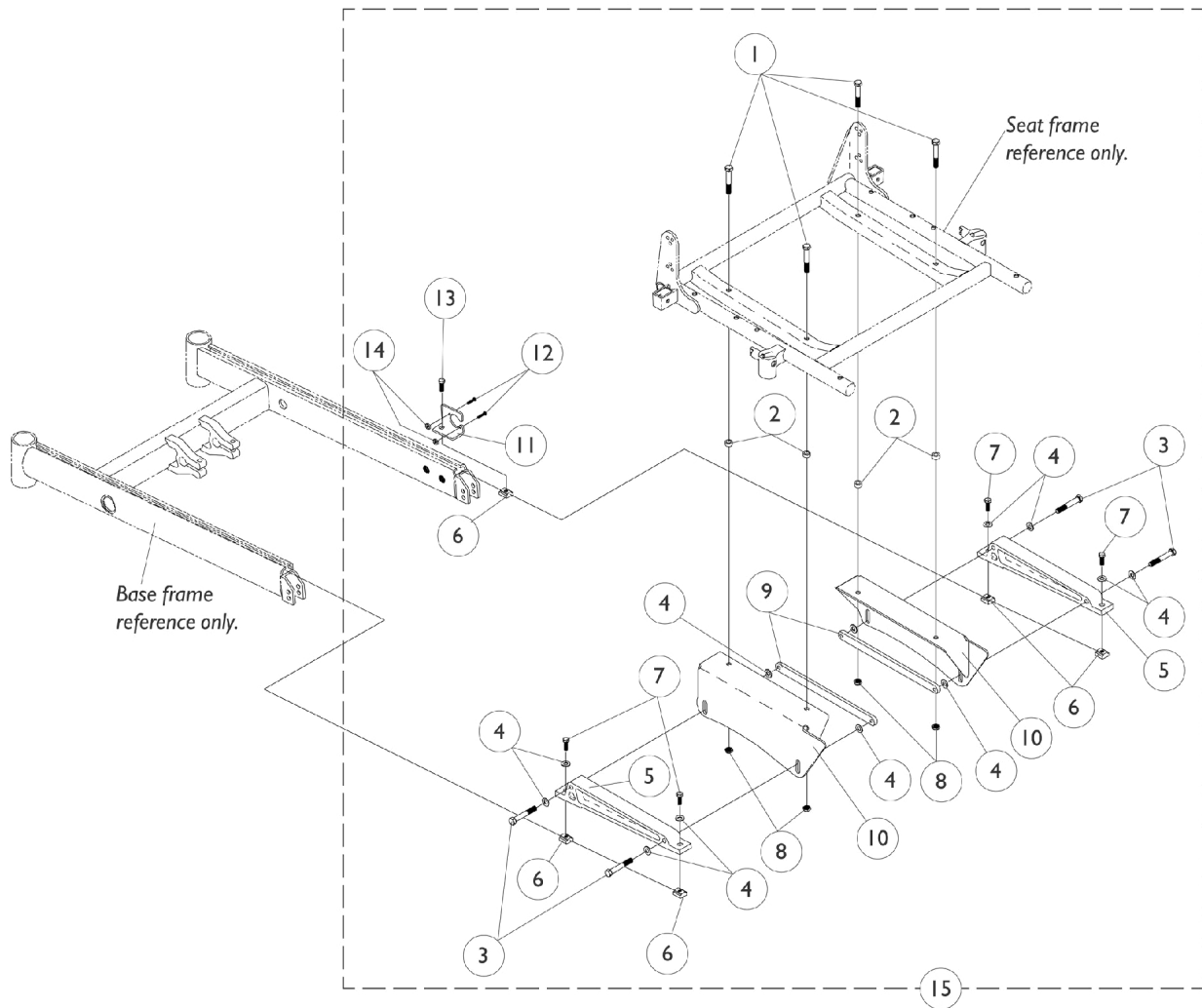

Seat Frame Mounting Hardware (Adult 16-24" Wide)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
1	B	1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	4
2	A	1070413-NLA	NLA - Spacer (21/64 x 1/2 x 17/64")	1	4
3		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
4		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
5		1070388	Support, Seat, Black	1	12
6		1069724	T-Nut (5/16-18)	1	2
7		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	5
8		1026158	Locknut (5/16-18)	1	4
9		1070396-NLA	NLA - Strap, Bolt, Black	1	4
10		1070343-NLA	NLA - Plate, Seat Mounting Low, Right	1	2
10		1070344-NLA	NLA - Plate, Seat Mounting Low, Left	1	2
10		1070345-NLA	NLA - Plate, Seat Mounting Medium, Right	1	2
10		1070346-NLA	NLA - Plate, Seat Mounting Medium, Left	1	2
10		1070347-NLA	NLA - Plate, Seat Mounting Tall, Right	1	2
10		1070348-NLA	NLA - Plate, Seat Mounting Tall, Left	1	2
11		1070430-NLA	NLA - Bracket, Charger Connector, Black	1	1
12		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	2
13		1061922-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	1
14		P0870X000	Nut, Hex Keps (#4-40)	1	1
15	C	1070495-NLA	NLA - Hardware, Seat Frame Mounting (16" W)	1	1
15	C	1070349-NLA	NLA - Hardware, Seat Frame Mounting (17" W)	1	1

NOTE: A - Used on the 16" wide chairs only.
B - Used on 17" and wider chairs only.
C - Item number 10 not included in package.