


# Seat Frame Mounting Hardware

(Junior 12-16" Wide)  
(Adult 14-15" Wide)

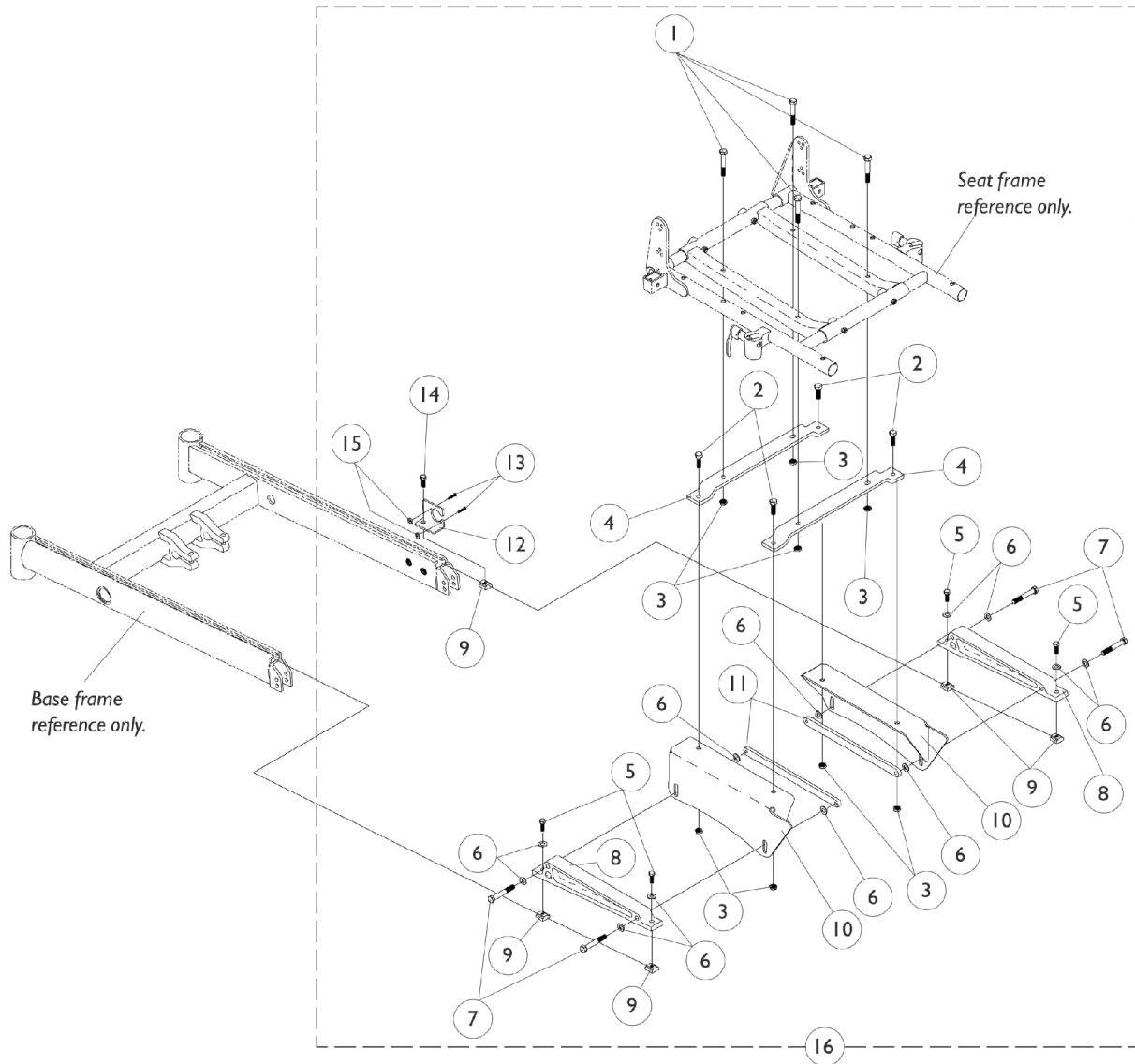

## Seat Frame Mounting Hardware (Junior 12-16" Wide) (Adult 14-15" Wide)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
2		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	4
3		1026158	Locknut (5/16-18)	1	8
4		1070465-NLA	NLA - Bracket, Mounting, Black - Junior	1	2
5		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	4
6		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	12
7		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
8		1070388	Support, Seat, Black	1	2
9		1069724	T-Nut (5/16-18)	1	5
10		1070343-NLA	NLA - Plate, Seat Mounting Low, Right	1	2
10		1070344-NLA	NLA - Plate, Seat Mounting Low, Left	1	2
10		1070345-NLA	NLA - Plate, Seat Mounting Medium, Right	1	2
10		1070346-NLA	NLA - Plate, Seat Mounting Medium, Left	1	2
10		1070347-NLA	NLA - Plate, Seat Mounting Tall, Right	1	2
10		1070348-NLA	NLA - Plate, Seat Mounting Tall, Left	1	2
11		1070396-NLA	NLA - Strap, Bolt, Black	1	2
12		1070430-NLA	NLA - Bracket, Charger Connector, Black	1	1
13		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	2
14		1061922-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	1
15		P0870X000	Nut, Hex Keps (#4-40)	1	1
16	A	1070470-NLA	NLA - Hardware, Seat Frame (Junior Seat Mounting)	1	1

NOTE: A - Item number 10 not included in package.