


Adjustable Flip-Up Removable Footboard

90FB2 (Sling Seat) and 90FB (Captain Seat)

Before 3/1/00

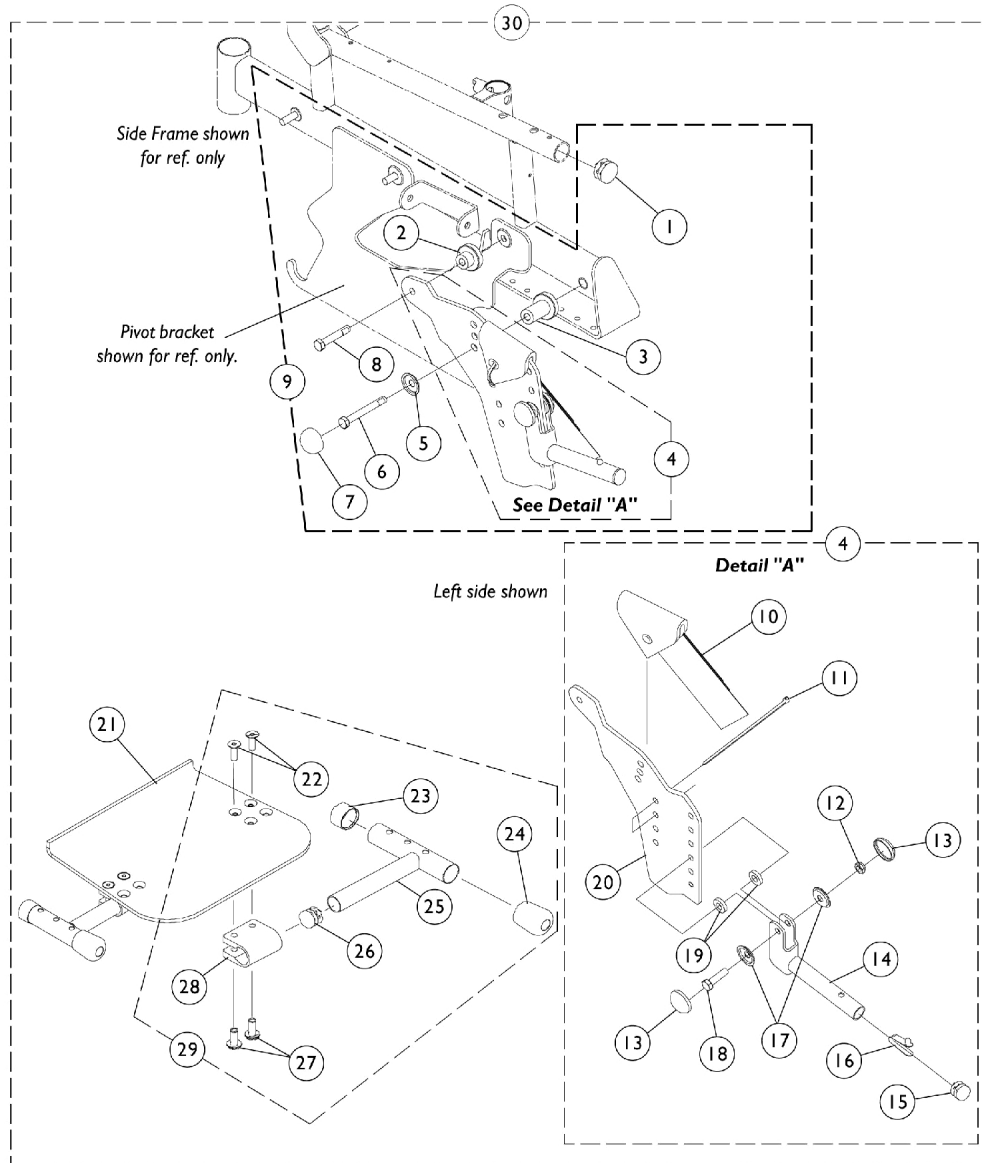

Adjustable Flip-Up Removable Footboard 90FB2 (Sling Seat) and 90FB (Captain Seat) Before 3/1/00

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	2
2		1098936-NLA	NLA - Bushing, Black (21/64 x 1-1/4 x 23/32")	1	2
3		1098937-NLA	NLA - Bushing, Black (21/64 x 1-1/4 x 1-19/32")	1	2
4	A	1080730-NLA	NLA - Footboard Pivot Assembly - Right	1	1
4	A	1080731-NLA	NLA - Footboard Pivot Assembly - Left	1	1
5		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
6		1043628-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 2-1/2")	1	2
7		1016291	Cap, Hub, Black	1	2
8		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	2
9	B	1086111-NLA	NLA - Footboard Mounting Hardware (Pair)	2	1
10		1081224-NLA	NLA - Guard, Footboard Pivot Bracket	1	1
11		98007X005	Tie Wrap (11-1/2" L)	1	2
12		1025730	Locknut (5/16-24)	1	1
13		1016291	Cap, Hub, Black	1	2
14		1074130-NLA	NLA - Tube, Pivot	1	1
15		1027097	Plug Button, Black (3/4")	1	1
16		40188X009	Button, Single Snap	1	1
17		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
18		1070552-NLA	NLA - Screw, Hex Head (5/16-24 x 1-1/8)	1	1
19		1074210-NLA	NLA - Washer, Nylon (23/64 x 3/4 x 5/32")	1	2
20		1081217-NLA	NLA - Bracket, Footrest Mounting, Black - Right	1	1
20		1081218-NLA	NLA - Bracket, Footrest Mounting, Black - Left	1	1
21	F	1074129-NLA	NLA - Footboard	1	1
22	D	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	4
23		15680X012	Collar, Anti-Rattle (7/8")	1	2
24		1016346	Cap, End, Tube, Black (7/8")	1	2
25		1074137-NLA	NLA - Tube, Footplate Support, Black	1	2
26		1027097	Plug Button, Black (3/4")	1	2
27	D	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	4
28	D	1058517	Clamp, Footplate	1	2
29	E	1080740-NLA	NLA - Footboard Assembly Hardware (Pair)	2	1
30	C,J	1086110-NLA	NLA - Footboard Assembly w/ Instructions	1	1
	I	1078193-NLA	NLA - Instruction Sheet, Footboard Assembly	1	1
	I	1095105-NLA	NLA - Kit, Footboard Conversion (16" W)	1	1
	H,I	1095106-NLA	NLA - Kit, Footboard Conversion (18" W)	1	1
	I	1095107-NLA	NLA - Kit, Footboard Conversion (20" W)	1	1

Adjustable Flip-Up Removable Footboard

90FB2 (Sling Seat) and 90FB (Captain Seat)

Before 3/1/00

NOTE: A - Includes Items 10-20.
B - Includes Items 1-8.
C - Footboard Assembly Kit before 3/1/00 is NLA. Refer to the Footboard Assembly Conversion Kits provided that will convert to a Footboard Assembly made after 2/29/00. Order according to the chair width. Captain Seats will use the 18" wide conversion kit
D - Items 22, 27, and 28 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change.
E - Includes Items 22-28.
F - Footboard before 3/1/200 is NLA. See page titled "Front Rigging - Adjustable Flip-Up Removable Footboard (After 2/29/00)" and select a footboard according to the chair width
H - Captain Seats will use the 18" wide conversion kit
I - Not Shown. For individual components for the conversion kits, See page titled "Front Rigging - Adjustable Flip-Up Removable Footboard (After 2/29/00)"
J - Includes Items 9, 21, and 29.