


## Active 3" Anti-Tip Wheel Assembly (RWD)

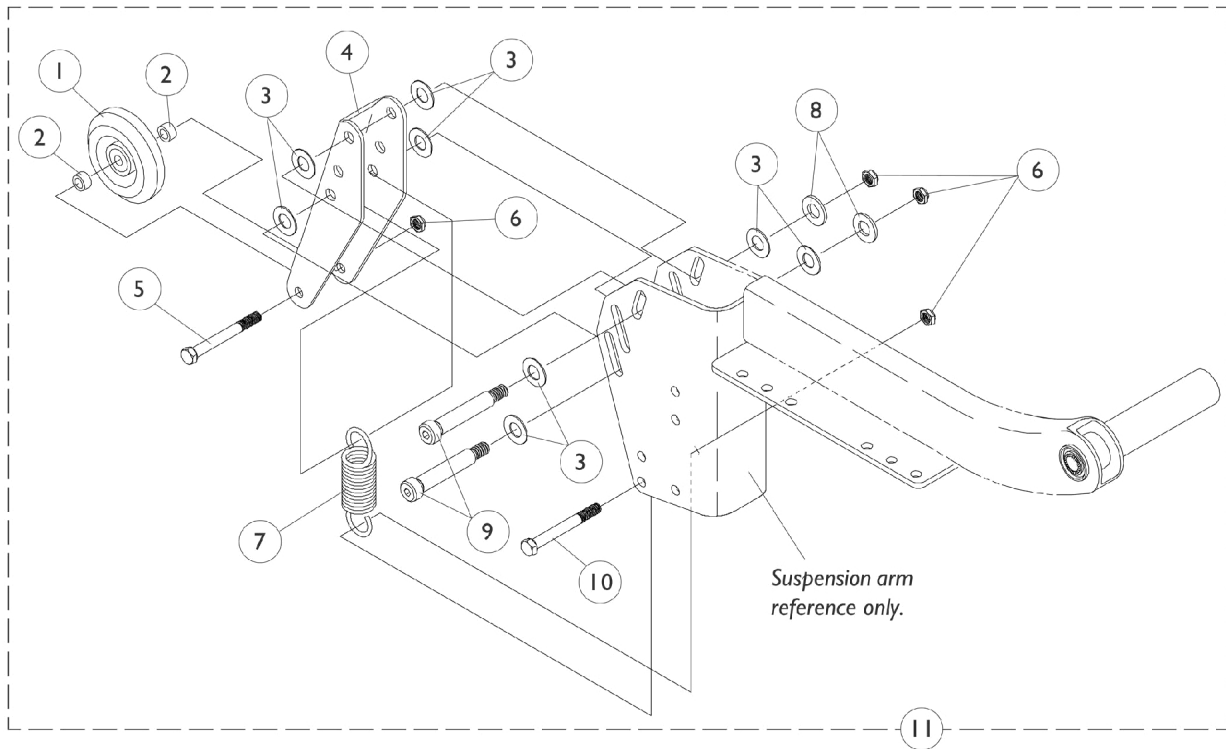

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1032190-NLA	NLA - Wheel Assembly (3" x 1")	1	2
2		1070413-NLA	NLA - Spacer (21/64 x 1/2 x 17/64")	1	4
3		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	16
4		1070410-NLA	NLA - Bracket, Anti-Tipper, Black	1	2
5		1070412-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	2
6		1026158	Locknut (5/16-18)	1	8
7		1070382-NLA	NLA - Spring, Extension (1 x 3-1/2" L)	1	2
8		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
9		1070411-NLA	NLA - Bolt, Shoulder (3/8 x 2-1/2")	1	4
10		1042486	Screw, Hex Head (5/16-18 x 2-3/4")	1	2
11		1070409-NLA	NLA - Anti-Tipper Assembly, Active (3") (Pair)	2	1