




6", 8", and 9" Caster Assemblies

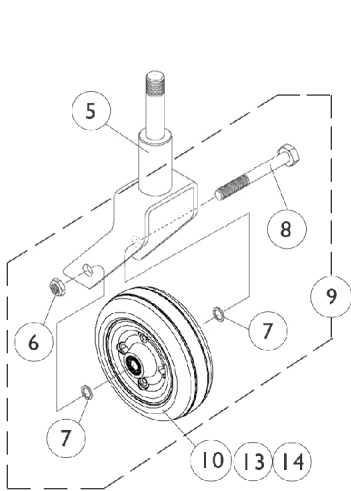


6" Caster Hardware

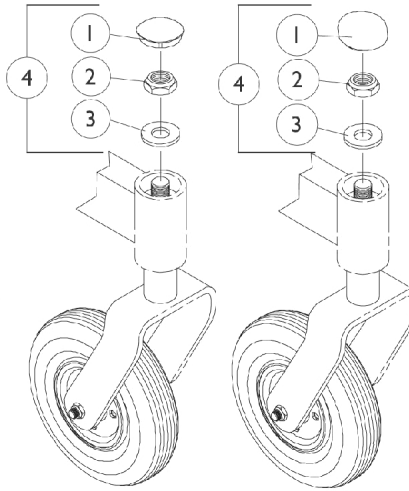


Before 3/1/01 After 2/28/01
Fork Mounting Hardware (All)

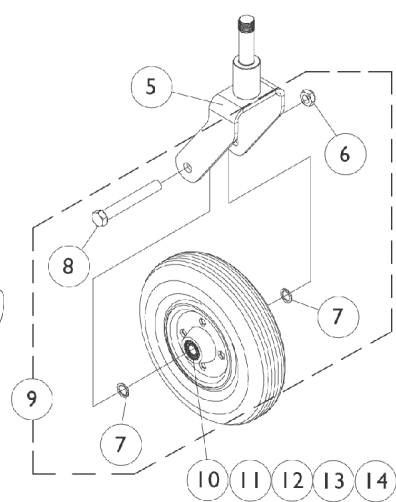


8" Caster Hardware

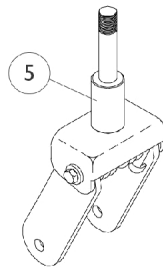

6" and 8" Caster Shock Fork

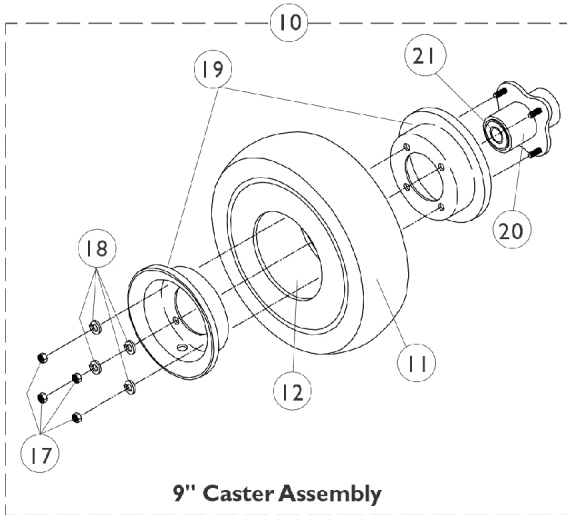

9" Caster Assembly

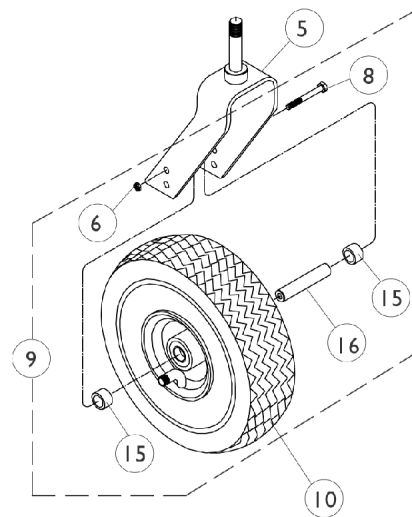

9" Caster Hardware

6", 8", and 9" Caster Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1027600-NLA	NLA - Cap, Black	1	1
1	D	1101737	Cap, End, Tube, Black	1	1
2		1028550	Locknut (5/8-18)	1	1
3		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
4	C,E	1050173-NLA	NLA - Hardware, Caster Fork Mounting	1	1
4	D,E	1104451	Hardware, Caster Fork Mounting	1	1
5	A,I	8881041458	Fork and Stem Assembly (6" Caster)	1	1
5	A,I	8881056379	Fork, Shock, Color (6")	1	1
5	A,I	8881039740-NL A	NLA - Fork and Stem Assembly (8" Caster)	1	1
5	A,J	8881056370-NL A	NLA - Fork, Shock, Color (8")	1	1
5	A,J	8881039754-NL A	NLA - Fork and Stem Assembly (9" and 10" Caster)	1	1
6	K	7930331-NLA	NLA - Locknut (7/16-20")	25	1
6	L	7930330-NLA	NLA - Locknut (5/16-24)	25	1
7		7930329	Washer, Steel (15/32 x 5/8 x 3/64")	25	2
8	K	1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
8	K,M	1058744-NLA	NLA - Screw, Hex Head (7/16-20 x 3-1/2")	1	1
8	L	1027730-NLA	NLA - Screw, Hex Head (5/16-24 x 5")	1	1
9	F,K	1042489	Hardware, Caster Axle	1	2
9	F,K	1058890-NLA	NLA - Hardware, Caster, Shock Fork (Pair)	2	1
9	G,L	1039757	Hardware, 9" Caster, Standard Fork (Per Side)	1	1
10	N	1072014-NLA	NLA - Wheel Assembly, Semi-Pneumatic, Urethane (6" x 2")	1	1
10	O	1115179	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2")	1	1
10	O	1122066	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2") (Pair)	2	1
10		1002002	Wheel Assembly w/ Bearings, Pneumatic (8" x 2")	1	1
10		1070811	Wheel Assembly, Semi-Pneumatic, Urethane, Light Grey (8" x 1-3/4")	1	1
10		1041498-NLA	NLA - Wheel Assembly, Pneumatic (9" x 2-3/4")	1	1
10	B	1029871-NLA	NLA - Wheel Assembly, Foam Filled (9 x 2-3/4")	1	1
11		40176X034	Tire, Pneumatic (8" x 2")	1	1
11		1027418-NLA	NLA - Tire, Pneumatic, Grey (9 x 2-3/4")	1	1
11	B	1028896-NLA	NLA - Tire w/ Foam Fill Insert, Grey (9" x 2-3/4")	1	1
12		1034942	Tube, Pneumatic Inner (8 x 2")	1	1
12		1027417	Tube, Pneumatic Inner (9 x 2-3/4")	1	1
13		1006049	Bearing (7/16")	1	2
14		1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1
17		1024426-NLA	NLA - Nut, Hex (5/16-24)	1	4
18		1024442-NLA	NLA - Washer, Lock (5/16 x 9/16 x 1/16")	1	4
19	H	1013460-NLA	NLA - Kit, Rim, Outer/Inner Half w/ Hardware (10")	1	1
19	H	1025399-NLA	NLA - Kit, Rim, Outer/Inner Half w/ Hardware (9")	1	1
20		1013466	Hub with Bearings Assembly	1	1
21		1015610	Bearing with Grip Ring (5/8")	1	2

6", 8", and 9" Caster Assemblies

NOTE: A - Specify frame color when ordering.
B - Tire comes with insert molded inside.
C - Before 3/1/01
D - After 2/28/01
E - Includes items 1-3
F - Includes items 6-8 to accommodate 1 side only.
G - Includes items 6, 8, 15, and 16. Hardware sold complete.
H - Includes items 17-18
I - 300 LB Weight Limit and under
J - 400 LB Weight Limit and under
K - 6 and 8" Casters
L - 9" Casters
M - Shock Fork
N - Service part No Longer Available 8/15/03. Due to product improvement, order (pair) part number 1122066 dated after 8/14/03 so each side matches. The deep profile caster has 2 tread grooves w/ round corners instead of 4 tread grooves w/ square corners.
O - Service part after 8/14/03