


# Free Wheel Hub Mounts

## (Available On-Chair as of 4/23/99)

**NOTE: To change a non-free wheel hub to a free wheeling hub, see section titled "Wheels, Drive Wheel Hub Mount, Free-Wheeling Hub KITS (as of 4/23/99)".**

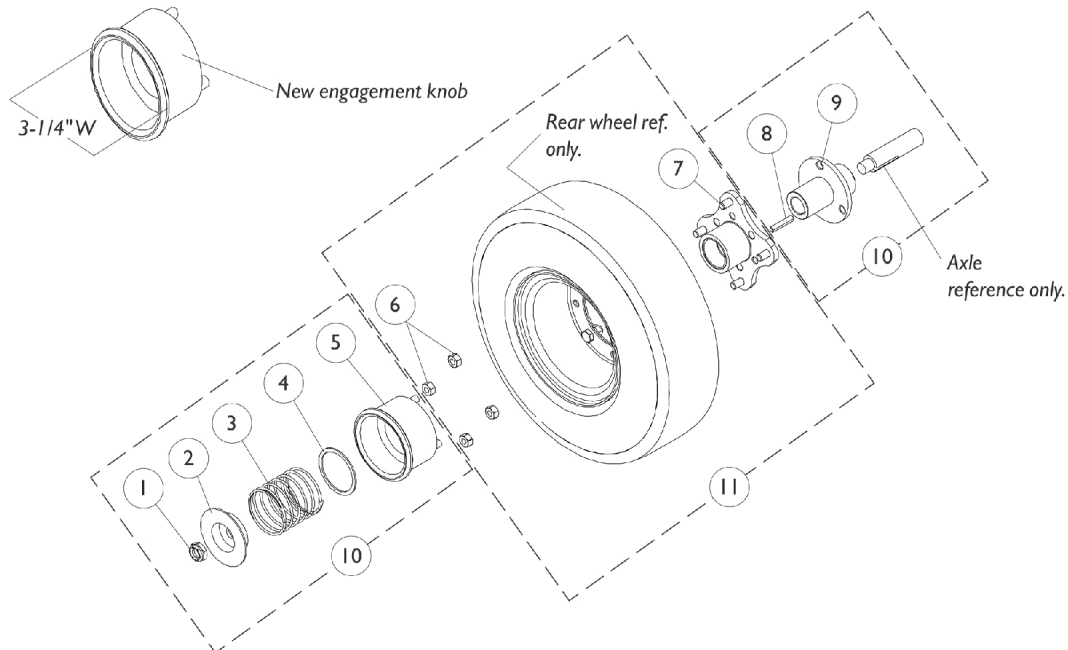

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930302	Locknut (1/2-20)	25	1
2		1078090	Cup, Washer	1	1
3		1079002-NLA	NLA - Spring, Compression (2-3/32 x 2-1/4 x 3/32")	1	1
4		1079001-NLA	NLA - Washer, Flat (1-13/16 x 2-1/8 x 1/16")	1	1
5		1077022	Knob, Engagement, Black	1	1
6		1024426-NLA	NLA - Nut, Hex (5/16-24)	1	4
7		1077023	Hub Assembly	1	1
8		1000180	Key (3/16 x 1-1/4")	1	1
9		1077021	Sleeve, Axle	1	1
10		1079004	Free Wheel Hub Assembly	1	1
11		1081358	Drive Wheel Assembly, Pneumatic w/ Hub, Grey Tire/Black Rim (14 x 3")	1	1
11		1081359	Drive Wheel Assembly, Flat Free w/ Hub, Grey Tire/Black Rim (14 x 3")	1	1
11		1081362-NLA	NLA - Drive Wheel Assembly, Pneumatic w/ Hub, Grey Tire/Black Rim (14 x 4")	1	1
11		1081363	Drive Wheel Assembly, Flat Free w/ Hub, Grey Tire/Black Rim (14 x 4")	1	1