


# Wheel Lock

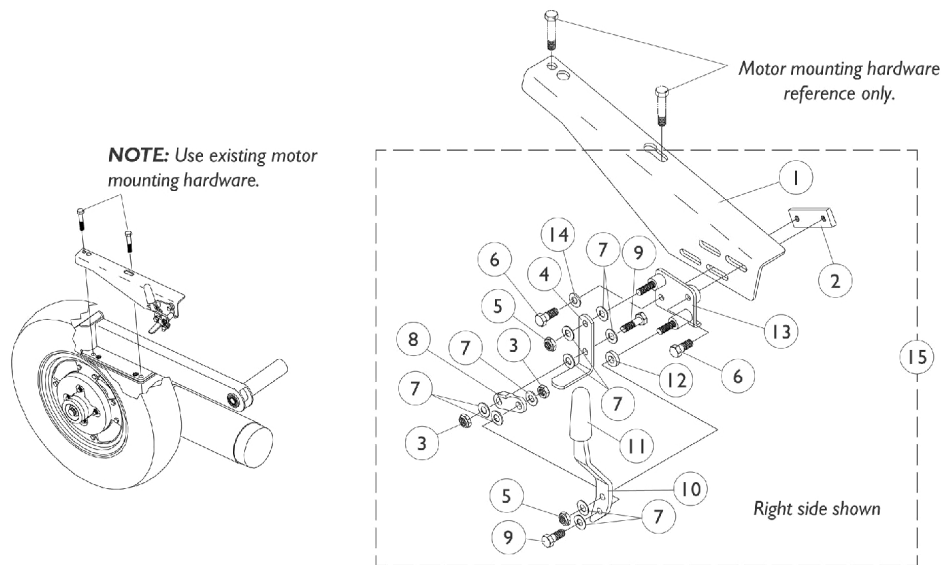

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1054510-NLA	NLA - Bracket, Wheel Lock Mount, Black - Right	1	1
1		1054512-NLA	NLA - Bracket, Wheel Lock Mount, Black - Left	1	1
2		1054508-NLA	NLA - Link, Inside, Black	1	1
3		7930334-NLA	NLA - Locknut (1/4-28)	25	2
4		1031833-NLA	NLA - Shoe, Wheel Lock	1	1
5		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
6		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	2
7		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	9
8		1018733-NLA	NLA - Link, Stop, Black	1	1
9		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	2
10		1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
10		1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
11		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
12		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
13		1054513-NLA	NLA - Plate, Wheel Lock Base, Black - Right	1	1
13		1054514-NLA	NLA - Plate, Wheel Lock Base, Black - Left	1	1
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
15	A	1054515-NLA	NLA - Wheel Lock Assembly, Right	1	1
15	A	1054516-NLA	NLA - Wheel Lock Assembly, Left	1	1
	B	1063339-NLA	NLA - Hub, Wheel (14" x 4")	1	1

**NOTE:** A - When ordering wheel locks for chairs made between 10/10/97 through 1/28/98 that have 14 x 4" drive wheels, two wheel lock hubs part number 1063339 need to be ordered as well.  
 B - Not Shown