
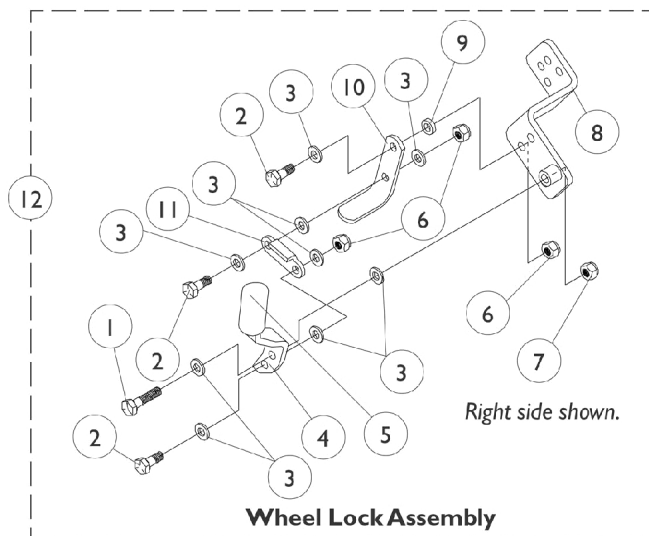


# Wheel Lock Assembly

## Gearless/Brushless

**NOTE: Gearless/ Brushless motors were made available on-chair as an option for Storm Arrow, Ranger X and Torque 11/2/98. On 7/1/99 Gearless/ Brushless motors became standard for Storm Arrow only.**

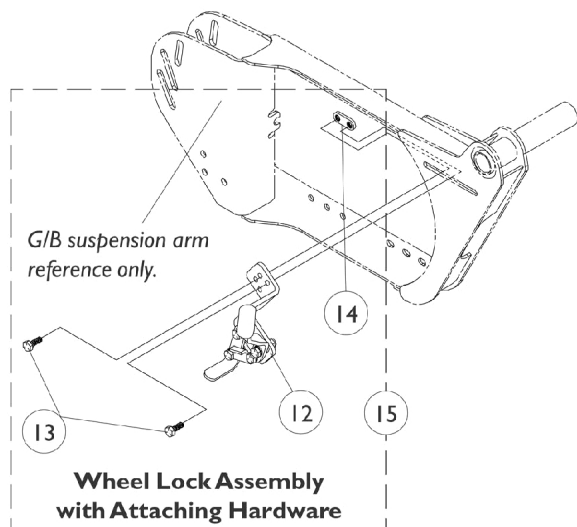

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
2		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	3
3		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	9
4		1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
4		1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
5		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
6		7930334-NLA	NLA - Locknut (1/4-28)	25	3
7		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
8		1078222-NLA	NLA - Bracket, Wheel Lock, Black - Right	1	1
8		1078223	Bracket, Wheel Lock, Black - Left	1	1
9		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
10		1031833-NLA	NLA - Shoe, Wheel Lock	1	1
11		1018733-NLA	NLA - Link, Stop, Black	1	1
12	B	1079266-NLA	NLA - Wheel Lock Assembly, GB - Right	1	1
12	B	1079267-NLA	NLA - Wheel Lock Assembly, GB - Left	1	1
13		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	2
14		1078199-NLA	NLA - Nut	1	1
15	A	1079252-NLA	NLA - Wheel Lock Assembly w/ Mounting Hardware - Right	1	1
15	A	1079253-NLA	NLA - Wheel Lock Assembly w/ Mounting Hardware - Left	1	1

NOTE: A - Includes items 12-14.  
B - Includes items 1-11.