


# Battery Box Support Frame for Group 24 Batteries

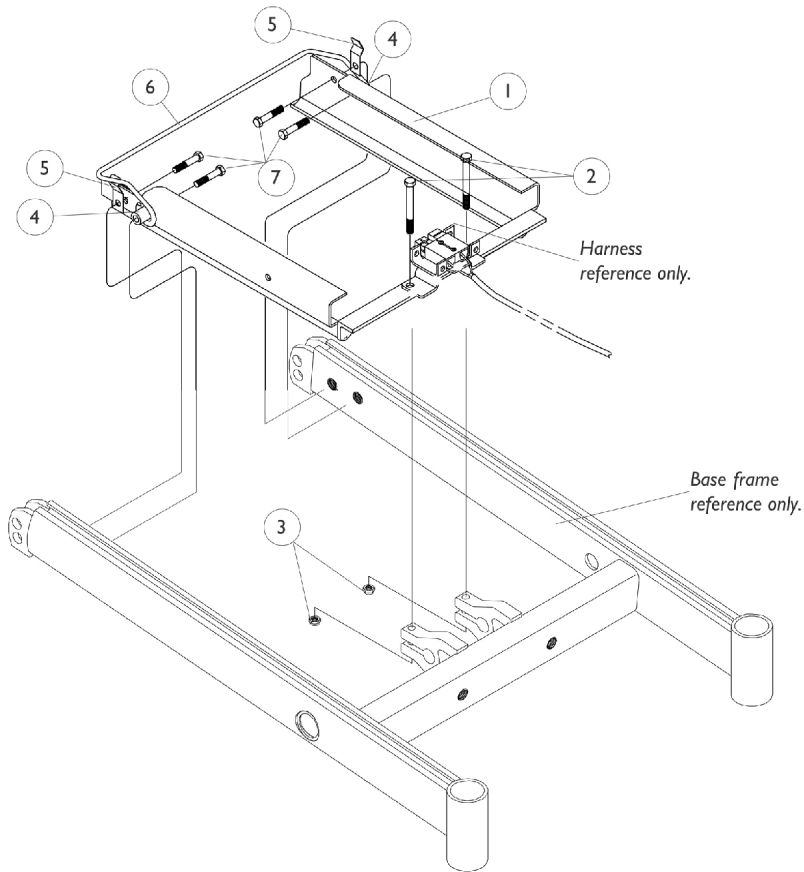

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1039715-NLA	NLA - Frame, Battery Box Support, Black	1	1
1	B	1086974	Frame, Battery Box Support, Black	1	1
2		1042484-NLA	NLA - Screw, Hex Head (5/16-18 x 2")	1	2
3		1026158	Locknut (5/16-18)	1	2
4		1039649-NLA	NLA - Spacer (3/8 x 5/8 x 25/32")	1	2
5		1049550-NLA	NLA - Clip, Retainer	1	2
6		1049549-NLA	NLA - Retainer Bar, Battery Box	1	1
7		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
NOTE: A - Before 9/24/99 B - After 9/23/99					