


# Adjustable Flip-Up Removable Footboard

## 90FB2 (Sling Seat) and 90FB (Captain Seat)

After 2/29/00

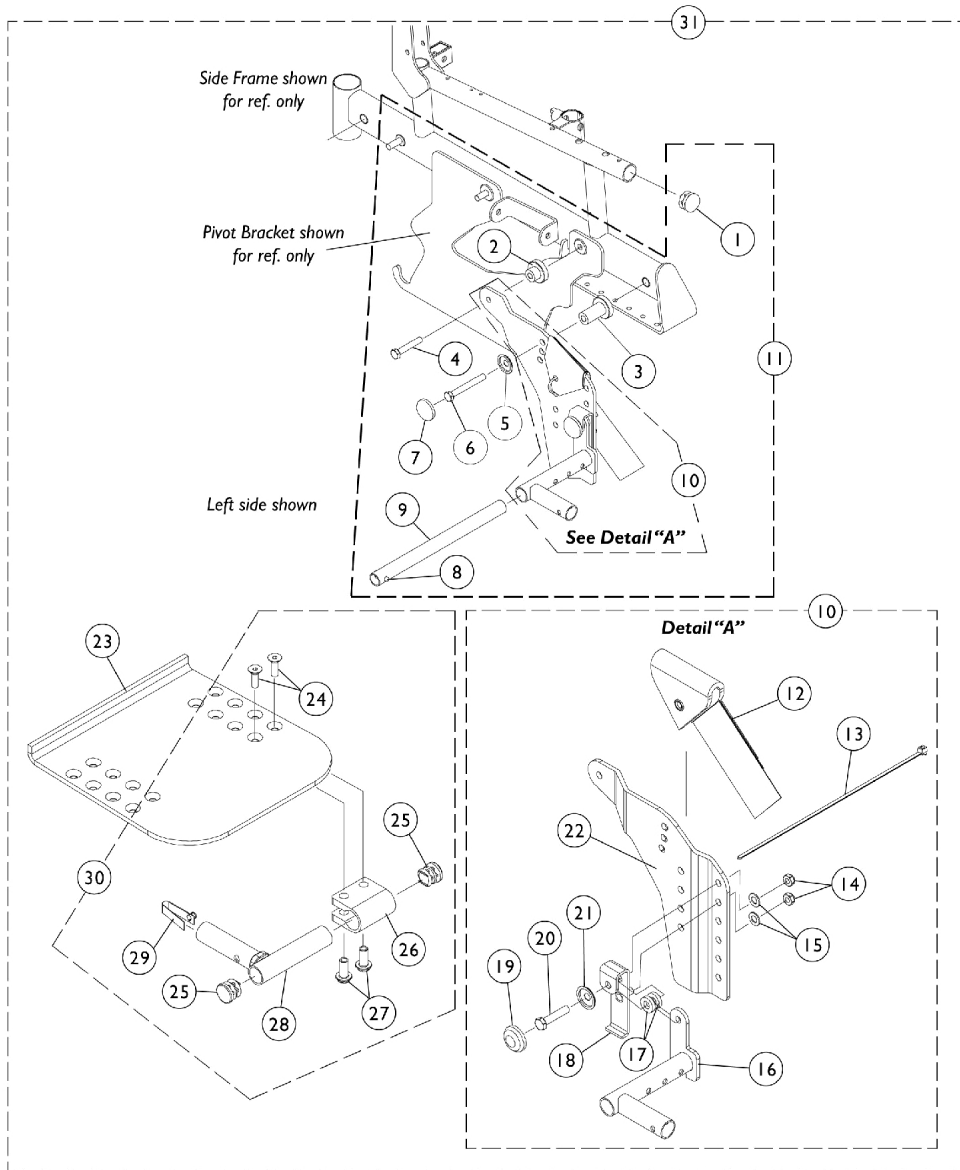

## Adjustable Flip-Up Removable Footboard 90FB2 (Sling Seat) and 90FB (Captain Seat) After 2/29/00

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	2
2		1098936-NLA	NLA - Bushing, Black (21/64 x 1-1/4 x 23/32")	1	2
3		1098937-NLA	NLA - Bushing, Black (21/64 x 1-1/4 x 1-19/32")	1	2
4		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	2
5		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	2
6		1043628-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 2-1/2")	1	2
7		1016291	Cap, Hub, Black	1	2
8		40188X007-NLA	NLA - Button, Spring Double	1	1
9		1095124-NLA	NLA - Support Tube w/ Spring Assembly	1	1
10	A	1095101-NLA	NLA - Footboard Pivot Assembly - Right	1	1
10	A	1095102-NLA	NLA - Footboard Pivot Assembly - Left	1	1
11	B	1095104-NLA	NLA - Footboard Mounting Hardware	2	1
12		1081224-NLA	NLA - Guard, Footboard Pivot Bracket	1	1
13		98007X005	Tie Wrap (11-1/2" L)	1	2
14		1028769	Locknut (5/16-18)	1	2
15		1074210-NLA	NLA - Washer, Nylon (23/64 x 3/4 x 5/32")	1	2
16		1094614-NLA	NLA - Bracket, Footboard Receiving, Black - Right	1	1
16		1094615-NLA	NLA - Bracket, Footboard Receiving, Black - Left	1	1
17		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
18		1094632-NLA	NLA - Bracket, Stop Pivot Support, Black	1	1
19		1016291	Cap, Hub, Black	1	1
20		1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	1
21		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	1
22		1081217-NLA	NLA - Bracket, Footrest Mounting, Black - Right	1	1
22		1081218-NLA	NLA - Bracket, Footrest Mounting, Black - Left	1	1
23		1094612-NLA	NLA - Footboard, Black (16" W)	1	1
23	C	1094613-NLA	NLA - Footboard, Black (18" W)	1	1
23		1094634-NLA	NLA - Footboard, Black (20" W)	1	1
24		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	4
25		1027097	Plug Button, Black (3/4")	1	4
26		1058517	Clamp, Footplate	1	2
27		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	4
28		1094611-NLA	NLA - Support Tube w/ Snap Tube, Black	1	2
29		40188X009	Button, Single Snap	1	2
30	D	1095103-NLA	NLA - Footboard Hardware	1	1
31	E	1095105-NLA	NLA - Kit, Footboard Conversion (16" W)	1	1
31	C,E	1095106-NLA	NLA - Kit, Footboard Conversion (18" W)	1	1
31	E	1095107-NLA	NLA - Kit, Footboard Conversion (20" W)	1	1
	F	1095125-NLA	NLA - Instruction Sheet, Footboard Assembly	1	1

NOTE: A - Includes Items 12-22.  
B - Includes Items 1-7, 9, and 10.  
C - Captain seats will use footboards for 18" wide chairs.  
D - Includes Items 24-29.  
E - Includes Items 11, 23, and 30.  
F - Not Shown.