


Footrest Support Assembly J932A-14

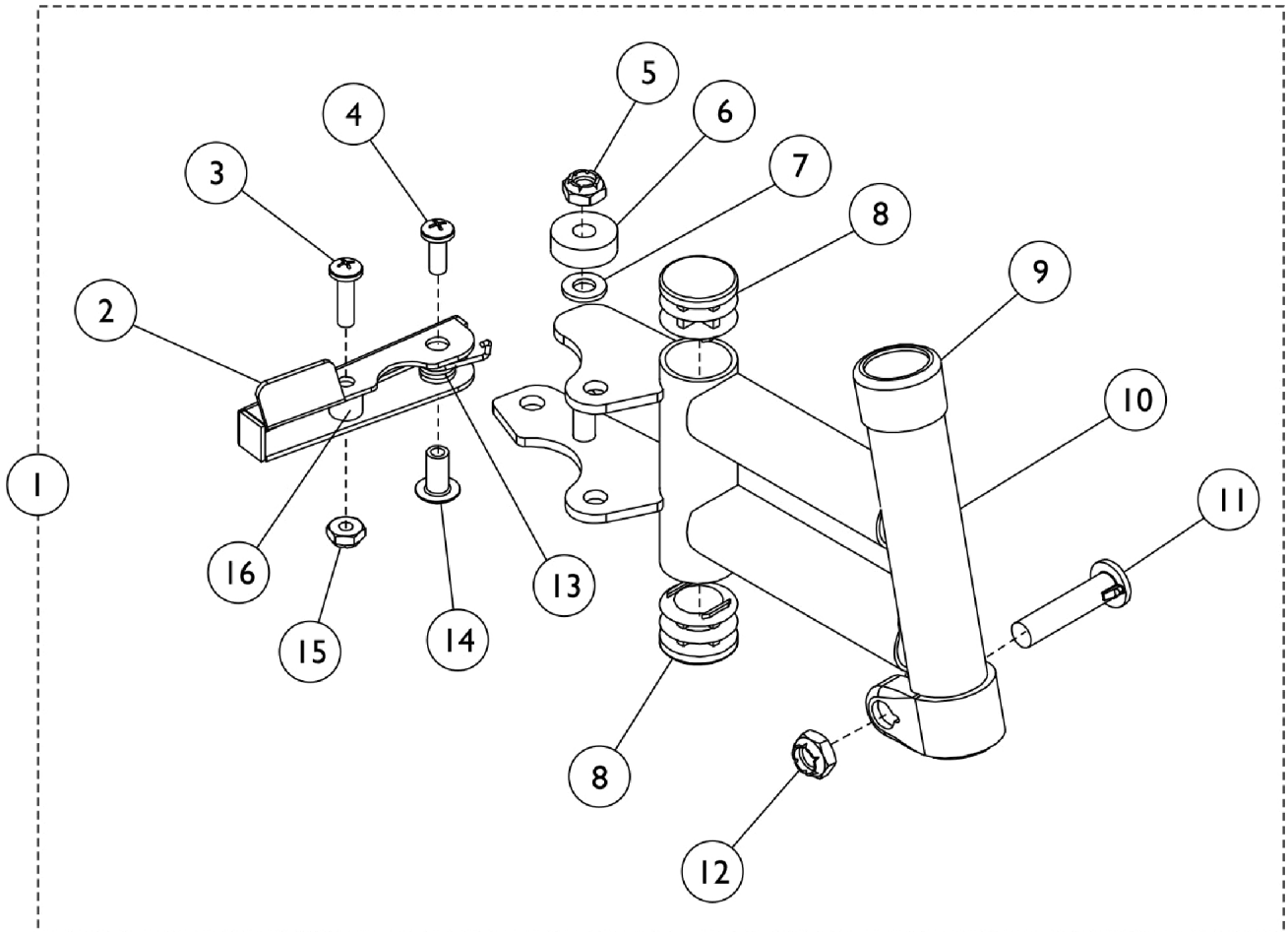

Footrest Support Assembly J932A-14

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881101006	Footrest Support Assembly, Color - Right	1	1
1	A,B	8881101007	Footrest Support Assembly, Color - Left	1	1
1	B	1101004-NLA	NLA - Footrest Support Assembly, Chrome - Right	1	1
1	B	1101005-NLA	NLA - Footrest Support Assembly, Chrome - Left	1	1
1	B	1133679	Footrest Support Assembly, Black - Right	1	1
1	B	1133680	Footrest Support Assembly, Black - Left	1	1
2		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
2		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
3		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1025195	Locknut (1/4-20)	1	1
6		1100705	Washer (1/4 x 3/4 x 1/4")	1	1
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
8		1027095	Plug Button, Black (7/8")	1	2
9		15680X012	Collar, Anti-Rattle (7/8")	1	1
10		1101000-NLA	NLA - Tube, Footrest Support, Chrome - Right	1	1
10		1101001-NLA	NLA - Tube, Footrest Support, Chrome - Left	1	1
10	A	8881101002	Tube, Footrest Support, Color - Right	1	1
10	A	8881101003	Tube, Footrest Support, Color - Left	1	1
10		1133677	Tube, Footrest Support, Black - Right	1	1
10		1133678	Tube, Footrest Support, Black - Left	1	1
11		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
12		1025730	Locknut (5/16-24)	1	1
12		1107321	Nut, Low Cap (5/16"-24)	1	1
13		1028707	Spring, Release - Right	1	1
13		1028708	Spring, Release - Left	1	1
14		1028846-NLA	NLA - Rivnut (#10-32)	1	1
15		1025735	Locknut (#10-32)	1	1
16		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1

NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."
B - Includes items 2-16.