


Footrest Support Assembly - J933A

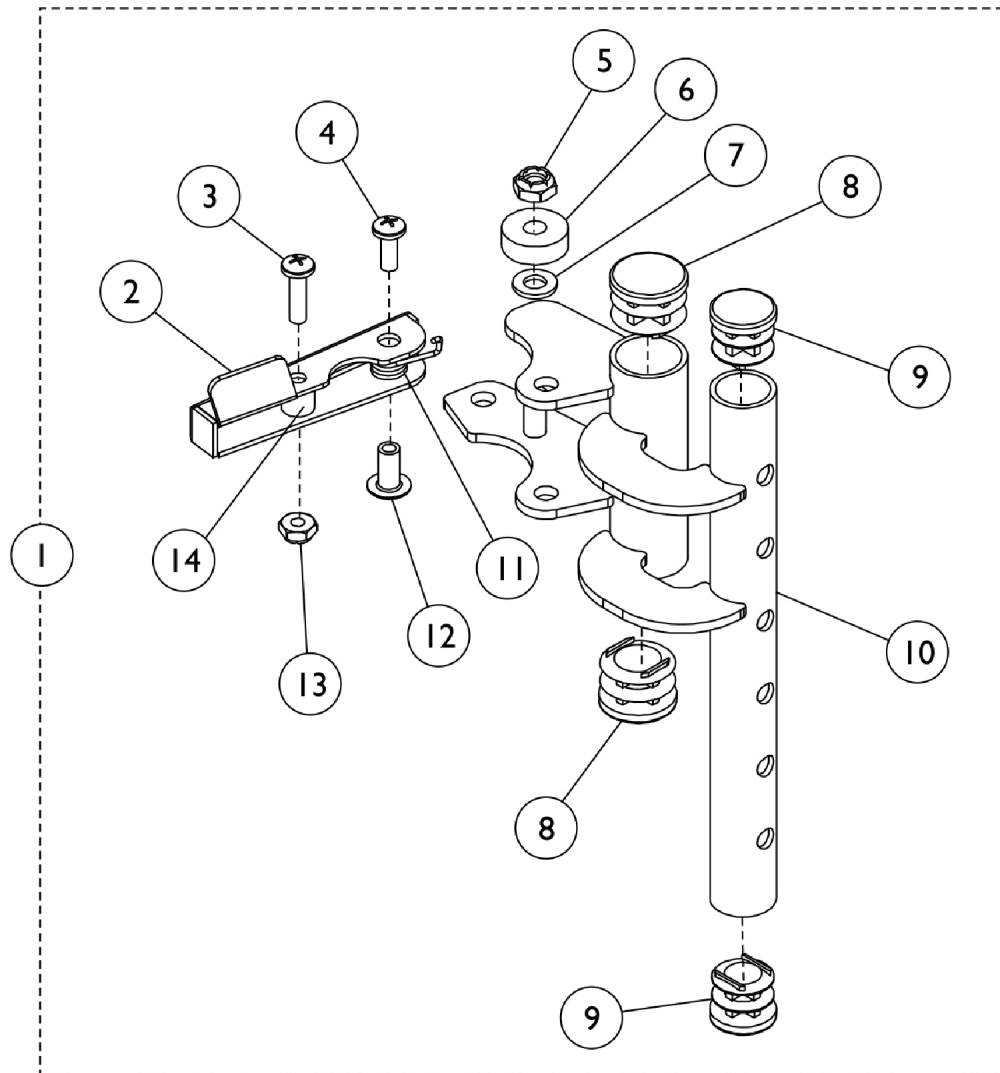

Footrest Support Assembly - J933A

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | A,B | 8881101012 | Footrest Support Assembly, Color - Right | 1 | 1 |
| 1 | A,B | 8881101013 | Footrest Support Assembly, Color - Left | 1 | 1 |
| 1 | B | 1101014 | Footrest Support Assembly, Chrome - Right | 1 | 1 |
| 1 | B | 1101015 | Footrest Support Assembly, Chrome - Left | 1 | 1 |
| 2 | | 1031469-NLA | NLA - Handle, Release, Black - Right | 1 | 1 |
| 2 | | 1031470-NLA | NLA - Handle, Release, Black - Left | 1 | 1 |
| 3 | | 1026114 | Screw, Phillips Pan Head (#10-32 x 1/2") | 1 | 1 |
| 4 | | 1025716 | Screw, Phillips Pan Head (#10-32 x 11/16") | 1 | 1 |
| 5 | | 1025195 | Locknut (1/4-20) | 1 | 1 |
| 6 | | 1100705 | Washer (1/4 x 3/4 x 1/4") | 1 | 1 |
| 7 | | 1026907 | Washer (1/4 x 1/2 x 1/16") | 1 | 1 |
| 8 | | 1027095 | Plug Button, Black (7/8") | 1 | 2 |
| 9 | | 1027097 | Plug Button, Black (3/4") | 1 | 2 |
| 10 | | 1100631 | Tube, Footrest Support, 90 Degree, Chrome - Right | 1 | 1 |
| 10 | | 1100632 | Tube, Footrest Support, 90 Degree, Chrome - Left | 1 | 1 |
| 10 | A | 8881100631 | Tube, Footrest Support, 90 Degree, Color - Right | 1 | 1 |
| 10 | A | 8881100632 | Tube, Footrest Support, 90 Degree, Color - Right | 1 | 1 |
| 11 | | 1028707 | Spring, Release - Right | 1 | 1 |
| 11 | | 1028708 | Spring, Release - Left | 1 | 1 |
| 12 | | 1028846-NLA | NLA - Rivnut (#10-32) | 1 | 1 |
| 13 | | 1025735 | Locknut (#10-32) | 1 | 1 |
| 14 | | 47565X057 | Bushing (7/32 x 3/8 x 11/32") | 1 | 1 |

NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."
B - Includes items 2-14.