


Pivot Slide Tube with Footplate Assembly - J932A-14

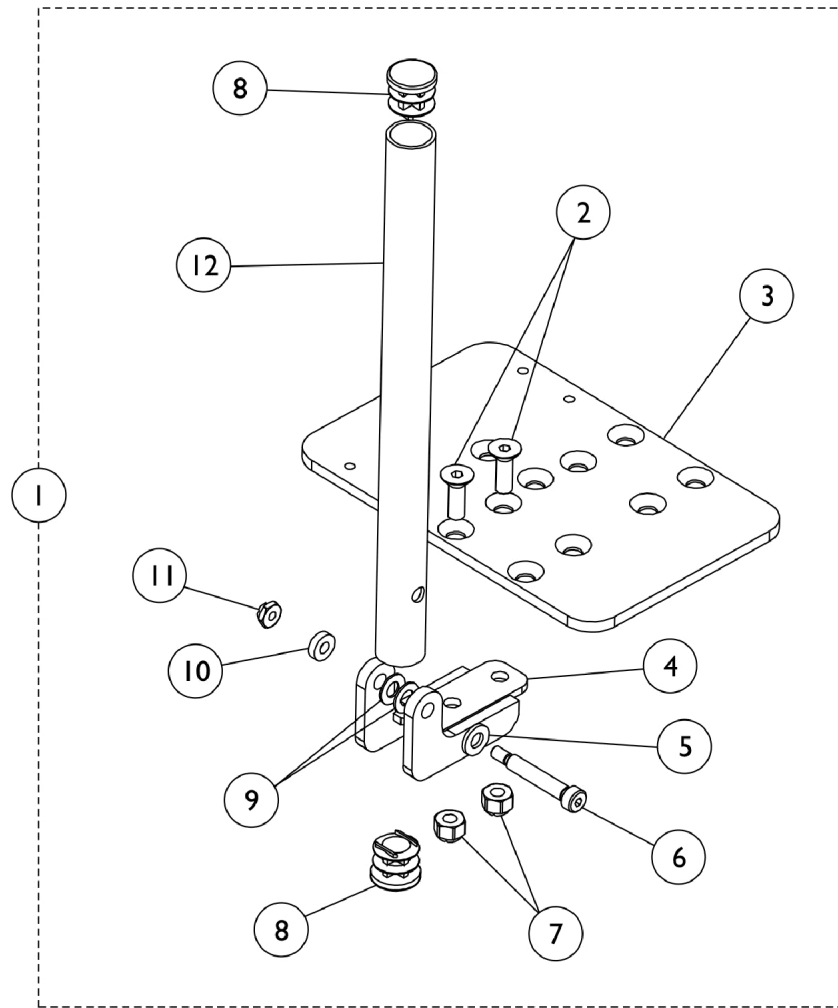

Pivot Slide Tube with Footplate Assembly - J932A-14

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,C	1102491	Pivot/Slide Tube w/ Footplate Assembly, Black - Right (12" Footplate)	1	1
1	A,B,C	1102492	Pivot/Slide Tube w/ Footplate Assembly, Black - Left (12" Footplate)	1	1
1	B,D	1102489-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly, Chrome - Right (12" Footplate)	1	1
1	B,D	1102490-NLA	NLA - Pivot/Slide Tube w/ Footplate Assembly, Chrome - Left (12" Footplate)	1	1
2		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1100988	Footplate, Black (3-1/4" W x 6" L) (10" W)	1	1
3		1100989	Footplate, Black (4-1/4" W x 6-1/2" L) (12" W)	1	1
4		1101020	Block, Hinge	1	1
5		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	1
6		1000212	Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)	1	1
7		1025736	Locknut (1/4-20)	1	2
8		1027097	Plug Button, Black (3/4")	1	2
9		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	2
10		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
11		1026035-NLA	NLA - Locknut (#10-24)	1	1
12	D	1102483-NLA	NLA - Tube, Slide, Chrome	1	1
12	A	1102484	Tube, Slide, Black	1	1

NOTE: A - Specify frame color when ordering. See category titled, "Upholstery Colors, Frame Finishes and Packaging."
 B - Includes items 2-12.
 C - Black
 D - Chrome