


Head Support

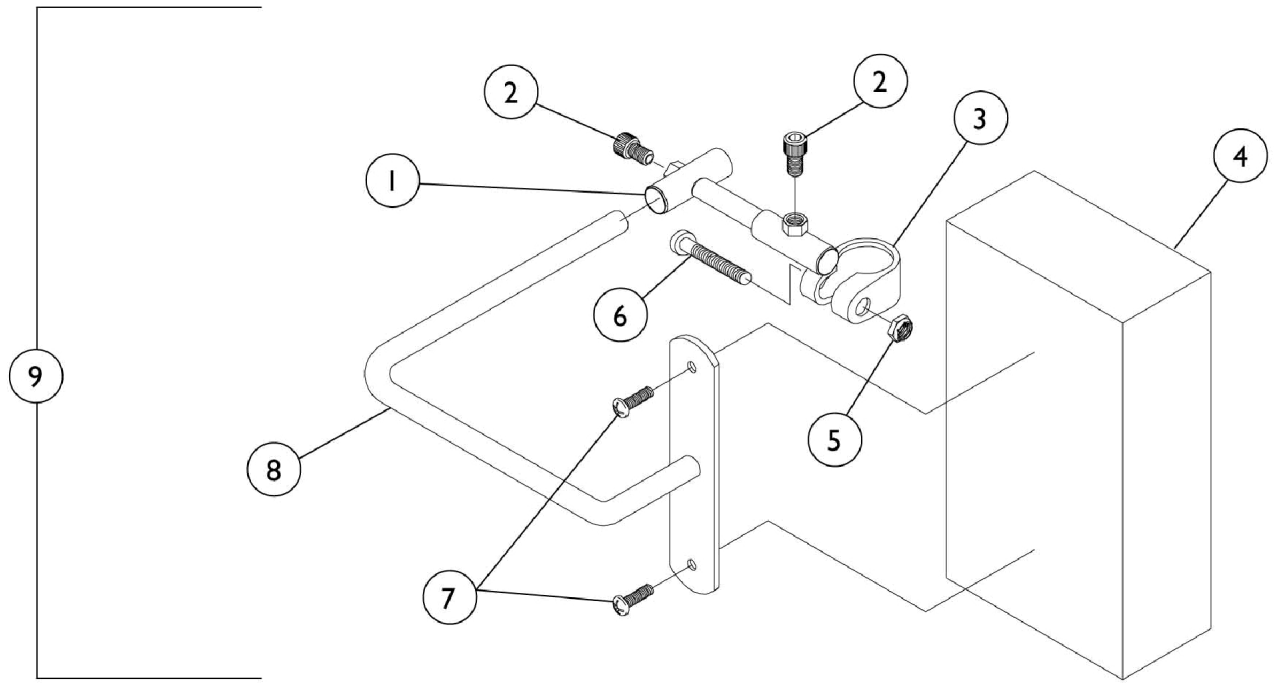

Head Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		03018X042	Wing Bracket Assembly, Chrome	1	1
2		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	2
3		03018X043	Bracket, Clamp, Black- Right	1	1
3		03018X046	Bracket, Clamp, Black - Left	1	1
4	A,E	8881123986-NL A	NLA - Pad, Headrest	1	1
4	A	88804529X040- NLA	NLA - Pad, Headrest	1	1
5		60264X001	Locknut (5/16-24)	1	1
6		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
7		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
8	B	03018X045	Bracket, Pad Support, Chrome	1	1
8	C	03018X044-NLA	NLA - Bracket, Pad Support, Black	1	1
9	A,B	8881050191-NL A	NLA - Head Support Assembly (Pair)	1	1
9	A,C	8881050192-NL A	NLA - Head Support Assembly (Pair)	1	1
	D,F	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Specify upholstery color when ordering, see category titled, "Upholstery Color Options." B - Non-Recliner C - Recliner D - Not Shown E - No Longer Available. Use 88804529X040 as replacement. F - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					