



Adjustable Height Arms

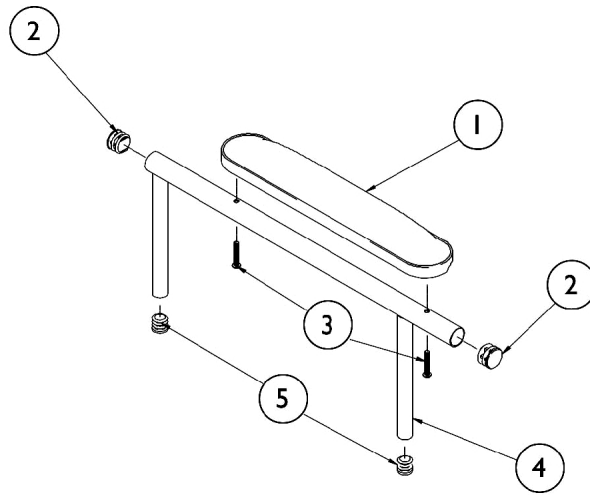

**Conventional Adjustable Height
"19/29" Style Arm**

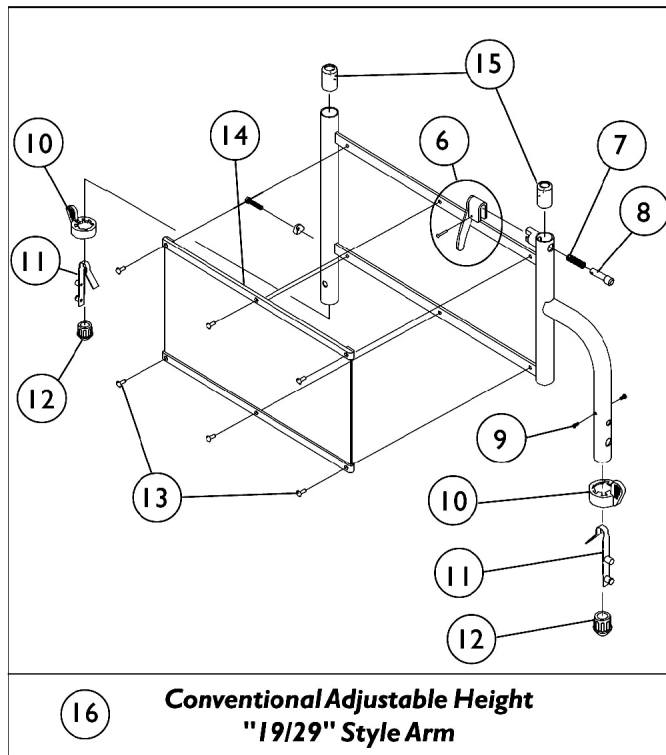

16

**Conventional Adjustable Height
"19/29" Style Arm**

Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,C	8881053578	Kit, Arm Pad w/ Screws, Desk Length (9" L x 2" W) (Pair)	2	1
1	B,C	8881053577	Kit, Arm Pad w/ Screws, Full Length (14" L x 2" W) (Pair)	2	1
1	D	1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
1	D	1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
1	B,C	8881011811-NL A	NLA - Pad, Arm, Vinyl, Desk Length (9" L x 2" W)	1	1
1	B,C	8881015932-NL A	NLA - Pad, Arm, Vinyl, Full Length - Right (14" L x 2" W)	1	1
1	B,C	8881015933-NL A	NLA - Pad, Arm, Vinyl, Full Length - Left (14" L x 2" W)	1	1
1	A,D	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1	A,D	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
3	E	1094270-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")	2	1
4		1038470	Arm Tube Assembly, Conventional w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
4		1038472	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
5		1035614	Plug Button, Black (5/8")	1	2
6		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
7		47471X074	Spring, Arm Lock	1	1
8		47471X068	Pin, Locking	1	1
9		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
10		1015450	Handle, Quad Release	1	2
11		1018225	Spring, Torsion with Button Lock	1	2
12		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
13		1028645	Rivet	1	6
14		1064129	Guard, Clothing	1	1
15		1036150	Anti-Rattle	1	2
16		1057992	Arm Base Assembly, Removable, Black - Right	1	1
16		1057993	Arm Base Assembly, Removable, Black - Left	1	1

NOTE: A - As of 1/3/02 upholstered waterfall armpads were replaced with black urethane waterfall armpads. Order in pairs with screws for chairs manufactured before 1/3/02.
 B - When ordering please specify color U67 Black.
 C - R32/R51 Models
 D - LX/LXP Models
 E - For Black Urethane waterfall Arm Pads Only