



Battery Boxes & Trays


UI BATTERY BOXES

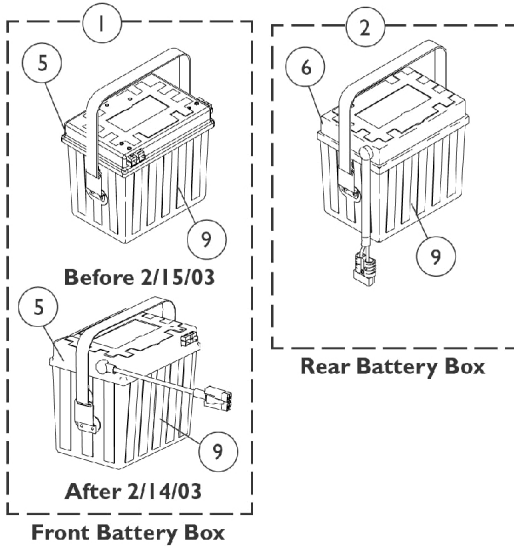

22NF BATTERY BOXES

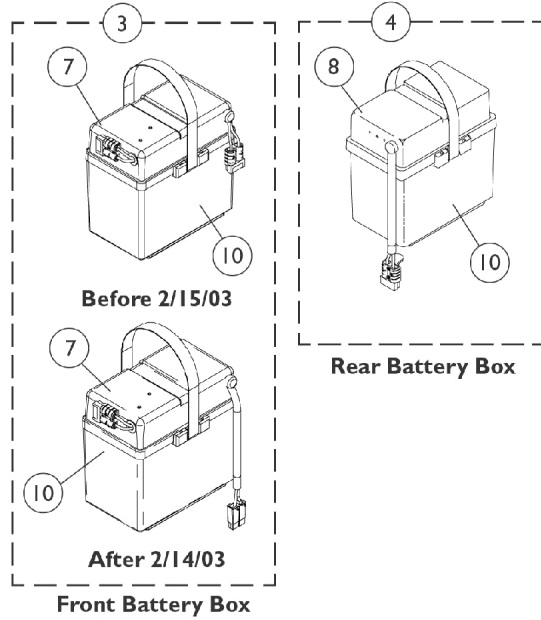

BATTERY TRAY ASSEMBLY

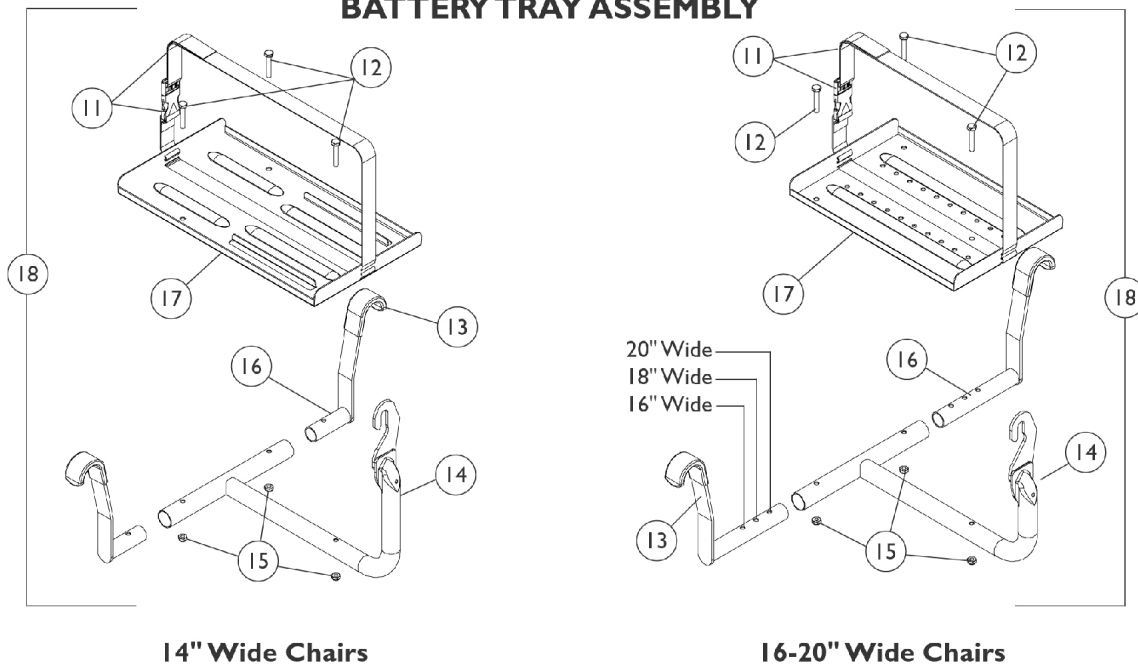

Battery Boxes & Trays

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	H	1107689-NLA	NLA - Battery Box Assembly, Front, UI	1	1
1	I	1115916-NLA	NLA - Battery Box Assembly, Front, U1	1	1
2		1107692-NLA	NLA - Battery Box Assembly, Rear, UI	1	1
3	H	1107690-NLA	NLA - Battery Box Assembly, Front, 22NF	1	1
3	I	1115917-NLA	NLA - Battery Box Assembly, Front, 22NF	1	1
4		1097007	Battery Box Assembly, Rear, 22NF	1	1
5	H	1107684-NLA	NLA - Top, Battery Box, Front, UI	1	1
5	I	1115466-NLA	NLA - Battery Box Top, Front, U1	1	1
6		1107687-NLA	NLA - Top, Battery Box, Rear, UI	1	1
7	H	1107680-NLA	NLA - Battery Box Lid Assembly, Front, 22NF	1	1
7	I	1115248-NLA	NLA - Top, Battery Box, Front, 22NF	1	1
8	G	1097003	Battery Box Top, Rear, 22NF	1	1
	C	1089385	Fuse, MIDI (Non-G/B & G/B) (80 AMP)	1	1
9	D	1009268-NLA	NLA - Battery Box Bottom Assembly, U1	1	2
	C,J	1013319	Strap, Battery	1	1
	C,J	1003177-NLA	NLA - Rivet	1	4
	C,J	16348X005-NLA	NLA - Washer, Steel (5/32 x 3/8 x 1/32")	1	4
10	E	1057944	Battery Box Bottom with Strap, 22NF	1	2
	C,K	1057954-NLA	NLA - Strap, Battery	1	1
	C,K	1041433-NLA	NLA - Pin, Handle	1	2
	A,C,F	1060318-NLA	NLA - Hardware, Battery Terminal, UI Batteries (Pair)	2	1
	B,C,G	1058618-NLA	NLA - Hardware, Battery Terminal, 22NF Batteries (Pair)	2	1
11		1107671	Buckle with Strap	1	1
12		1074642-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	3
13		1107153-NLA	NLA - Cover, Hanger	1	2
14		1106305-NLA	NLA - Bracket, Hanger, Battery Tray, Hemi, Black	1	1
14		1106306-NLA	NLA - Bracket, Hanger, Battery Tray, Adult, Black	1	1
15		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	3
16	F,L	1108616-NLA	NLA - Bracket, Hanger, Battery Tray, Black	1	2
16	G	1106293	Bracket, Hanger, Battery Tray, Black	1	2
17	F	1106310-NLA	NLA - Tray, Battery (UI Batteries)	1	1
17	G	1106309-NLA	NLA - Tray, Battery (22NF Batteries)	1	1
18	F	1107673-NLA	NLA - Battery Tray Assembly, Hemi	1	1
18	F	1107674-NLA	NLA - Battery Tray Assembly, Adult	1	1
18	G	1107675	Battery Tray Assembly, Hemi	1	1
18	G	1107676	Battery Tray Assembly, Adult	1	1

Battery Boxes & Trays

NOTE: A - Hardware includes small battery terminal caps (gray and orange) 1/4-20 x 7/8" hex flange screws, 1/4-20 hex flange nuts, cable ties, locking tab washers and instruction sheet for two batteries.
B - Hardware includes battery clamps, battery clamp caps (red and black), small battery terminal caps (red and black), 1/4-20 x 7/8" hex flange screws, 1/4-20 hex flange nuts, cable ties and locking tab washers two batteries.
C - Not Shown.
D - Includes: Battery Strap (1013319); Pop Rivet (5/32) (1003177); Washer (5/32 x 3/8 x 1/32) (16348X005)
E - Includes: Battery Strap (1057954); Handle Pin (1041433)
F - 14" Wide
G - 16-20" Wide
H - Before 2/15/03 for MKIV controller only
I - After 2/14/03 for MK5 controller only
J - Part included with item #9
K - Part included with item #10
L - No longer available as a service part, use Item 18 (1107673 or 1107674) for the replacement.