
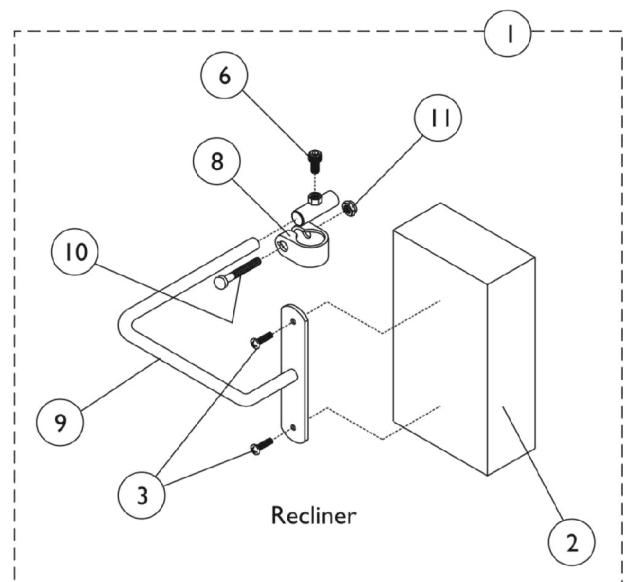


Torso Support

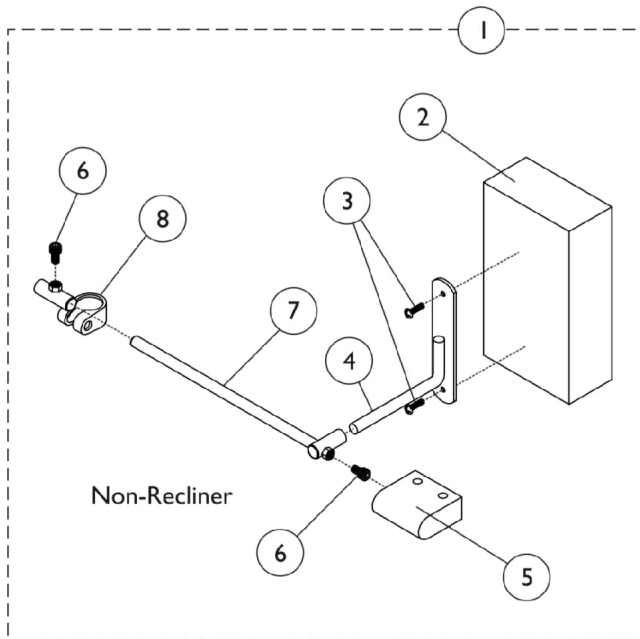

Torso Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881050189-NL A	NLA - Torso Support Assembly (Pair)	2	1
1	A,C	8881050190-NL A	NLA - Torso Support Assembly (Pair)	2	1
2	D	8881123986-NL A	NLA - Pad, Headrest	1	1
2	D,F	88804529X040- NLA	NLA - Pad, Headrest	1	1
3		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
4		03018X016-NLA	NLA - Bracket, Support, Chrome	1	1
5	D	88804529X043- NLA	NLA - Cover Torso Bracket	1	1
6		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
7		03018X012-NLA	NLA - Rod, Support w/ Guide Tube	1	1
8		03018X043	Bracket, Clamp, Black- Right	1	1
8		03018X046	Bracket, Clamp, Black - Left	1	1
9		03018X041-NLA	NLA - Bracket, Pad Support, Chrome	1	1
10		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
11		60264X001	Locknut (5/16-24)	1	1
	E,G	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Includes items 2-11 and an instruction sheet. B - For use on Non-Recliner style chairs. C - For use on Recliner style chairs. D - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors and Packaging." E - Not Shown. F - No Longer Available. Use 88804529X040 as replacement. G - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					