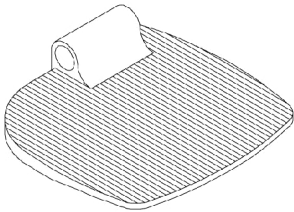
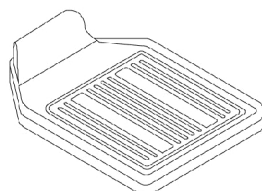
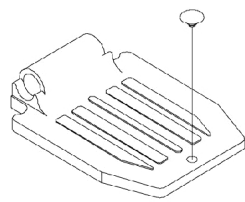
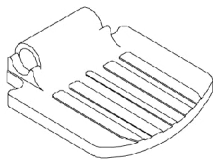



# Pivot & Slide Tube w/ Footplate

For the Following Legrests:  
P904, PAL4A, and PAS4A

Note: Heel Loops are not included. See footrest and legrest accessories.

Note: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

			
1350 Extra large	1450 Deluxe	16 to 20" Wide 1650 Standard Alum.	14" Wide 1650 Standard Alum.

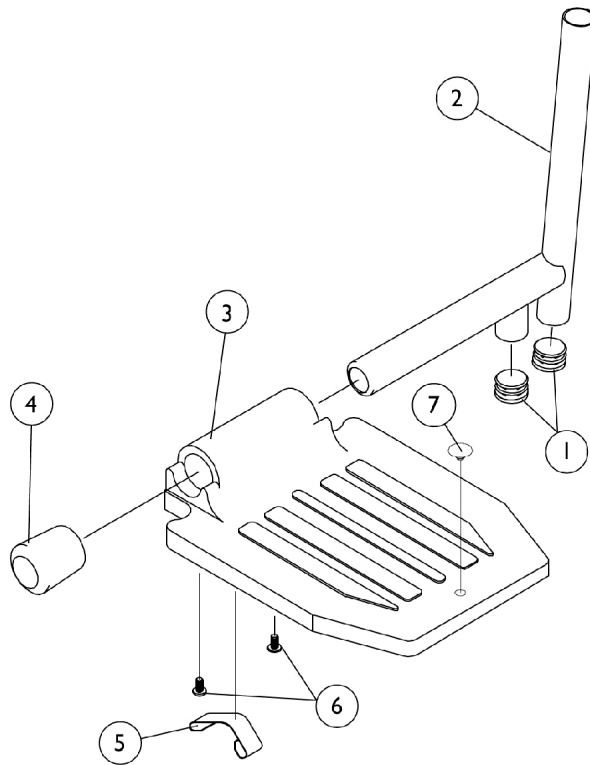

# Pivot & Slide Tube w/ Footplate

For the Following Legrests:  
P904, PAL4A, and PAS4A

Note: Heel Loops are not included. See footrest and legrest accessories.

Note: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	A	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	B	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	C	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
2	D	1070762-NLA	NLA - Tube, Pivot/Slide w/ Plug Button, Deluxe Aluminum, Black	1	1
3	E	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3	F	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	F	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3	F	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	E	1002343	Footplate, Aluminum, Extra Large, Black - Right (7" W x 8-1/2" L)	1	1
3	E	1002344	Footplate, Aluminum, Extra Large, Black - Left (7" W x 8-1/2" L)	1	1
3	F	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3	F	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	G	1016346	Cap, End, Tube, Black (7/8")	1	1
4	H	1027095	Plug Button, Black (7/8")	1	1
5	G	1006981	Spring, Tension	1	2
5		1036120-NLA	NLA - Spring	1	2
5	H,I	1036123	Spring, Leaf	1	1
6		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Standard Aluminum (1650) and Extra Large Aluminum (1350) footplates for P93M  
 B - Standard Aluminum (1650) and Extra Large Aluminum (1350) footplates for PW93 and PW93E  
 C - Deluxe Aluminum (1450) footplate for P93M  
 D - Deluxe Aluminum (1450) footplate for PW93 and PW93E  
 E - 14" Wide  
 F - 16-20" Wide  
 G - Standard Aluminum Footplate  
 H - Extra Large Aluminum Footplate  
 I - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.