


Side Frame and Hardware

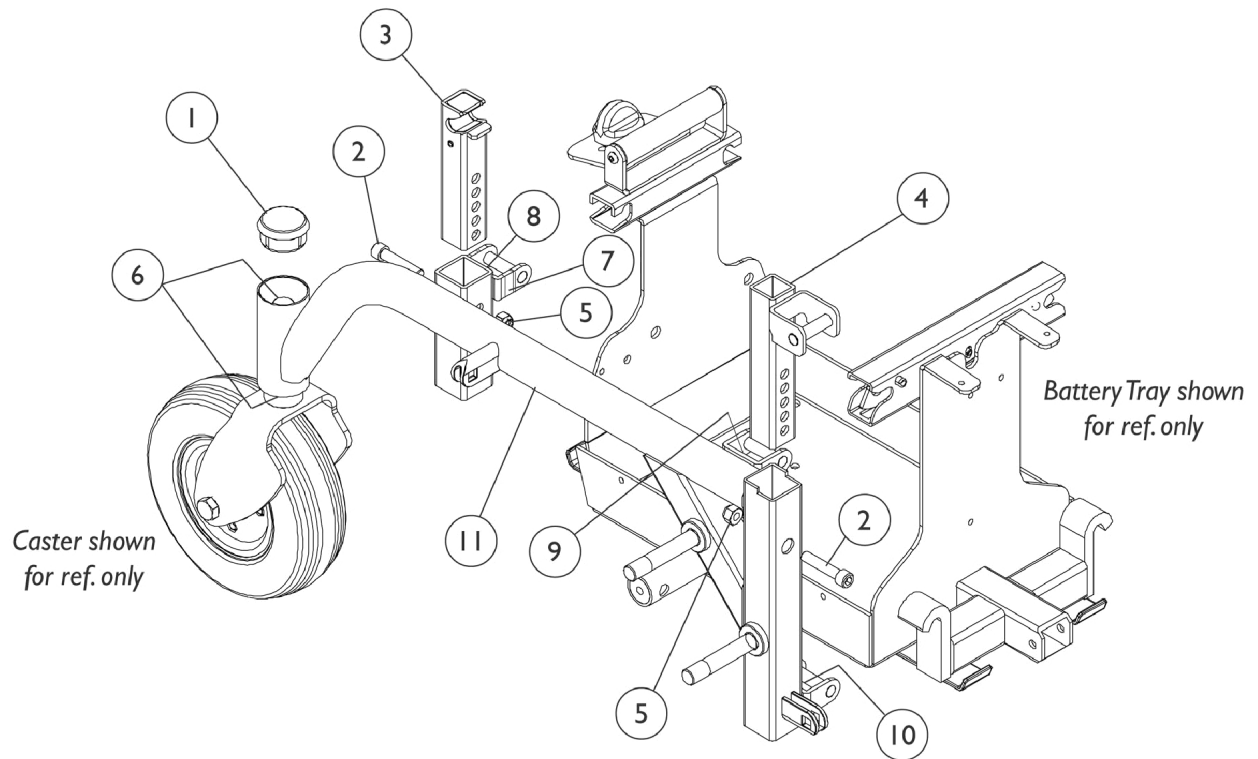

Side Frame and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	1105026	Plug Button, Head Tube, Black	1	1
2		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	2
3		1109019-NLA	NLA - Slide Tube, Seat Height Adjustment - Rear	1	2
3		1152529-NLA	NLA - Slide Tube Assembly, Seat Height Adjustment, Rear	1	2
4		1109215-NLA	NLA - Slide Tube Assembly, Seat Height Adjustment, Front	1	1
5		1026158	Locknut (5/16-18)	1	2
6		40117X006	Bearing (5/8")	1	2
7		1109198-NLA	NLA - Label, Yellow Stripe	1	2
8		1109200-NLA	NLA - Label Marker (2)	1	1
9		1109199-NLA	NLA - Label Marker (1)	1	1
10		1109201-NLA	NLA - Label Marker (3)	1	1
11	A	1109320-NLA	NLA - Side Frame Assembly - Right	1	1
11	A	1109321-NLA	NLA - Side Frame Assembly - Left	1	1
	B,C	1109023-NLA	NLA - Hardware, Frame, Side	1	1
NOTE: A - Includes items 6-10 B - Includes items 1-5 C - Not Shown D - When replacing a "Slide Tube, Seat Height Adjustment - Rear" part # 1109019 with the part # 1152529 both sides must be replaced simultaneously.					