


Crossbraces and Hardware

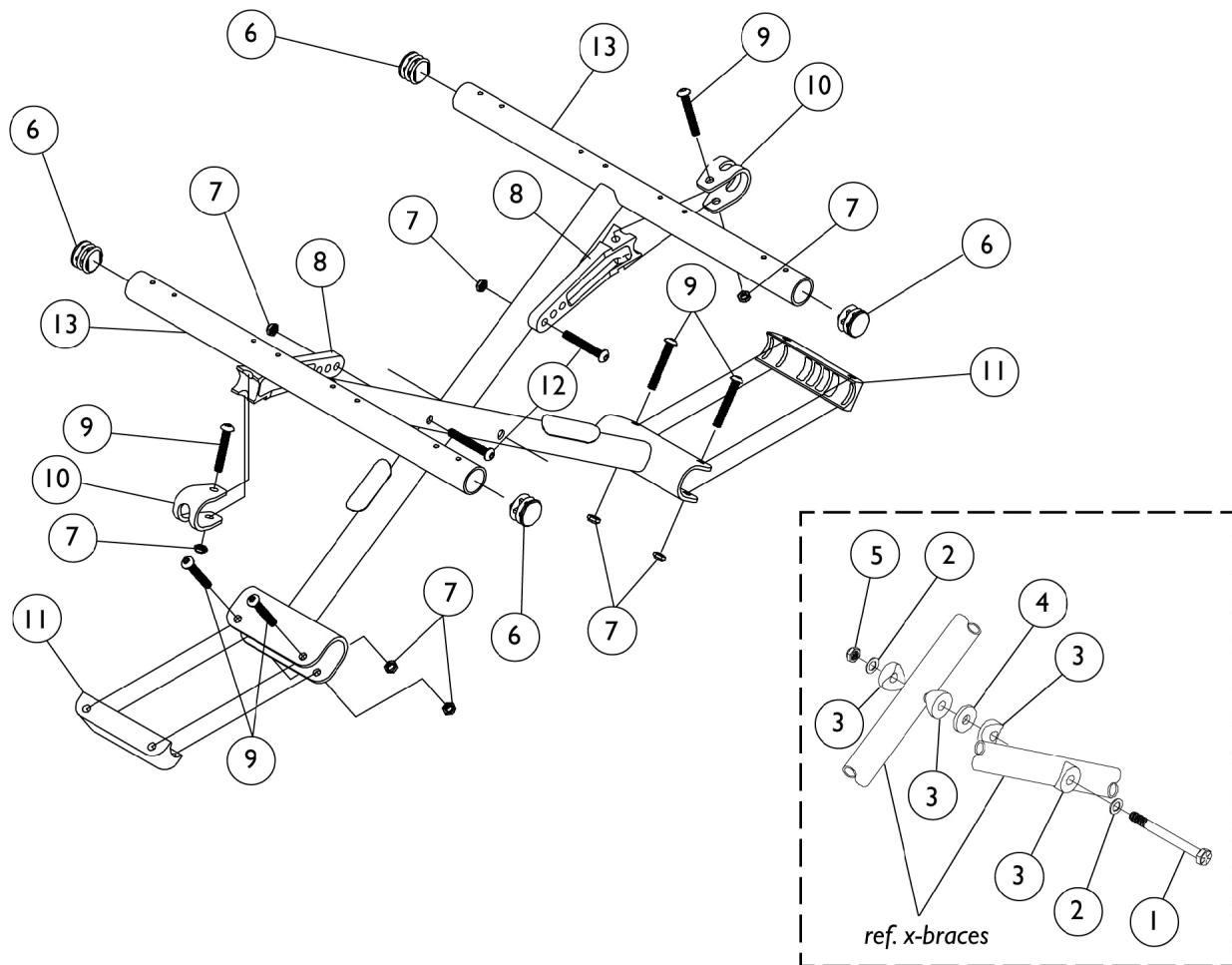

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016261	Screw, Hex Head (5/16-24 x 3-1/4")	1	2
2		1016652	Washer (5/16 x 9/16 x 1/32")	1	1
3		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	2
4		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	4
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		1029501	Plug Button, Black (1")	1	1
7		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	8
8		1019843-NLA	NLA - Link, Pivot, Black	1	4
9		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
10		1012180	Bracket, Pivot, Black	1	6
11		1012172-NLA	NLA - Clamp, Saddle	1	2
12		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	2
13		1019848-NLA	NLA - Crossbrace (10" W)	1	1
13		1019849-NLA	NLA - Crossbrace (12" W)	1	1
13		1019850-NLA	NLA - Crossbrace (14" W)	1	1
	A	1095497	Decal, Invacare Logo (1.6)	1	1
		7930364-NLA	NLA - Package, Locknut (1/4-20)	--	--

NOTE: A - Not Shown