
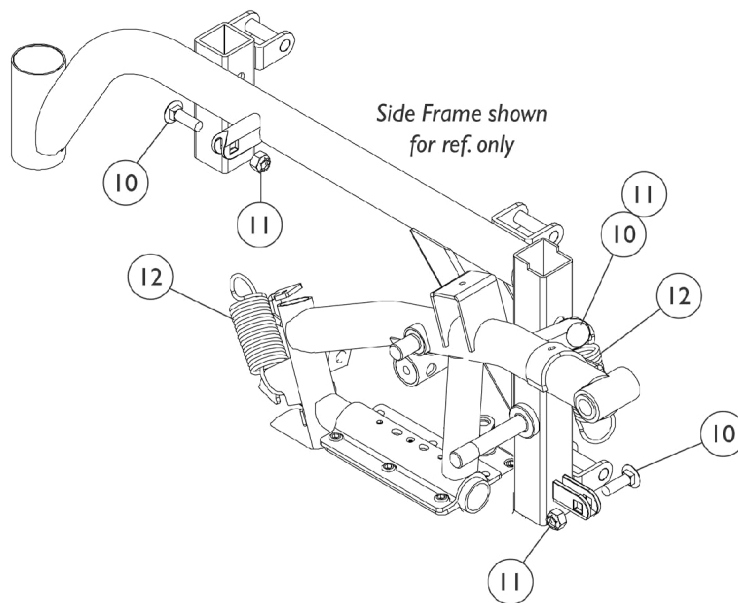


SureStep Walking Beams and Hardware

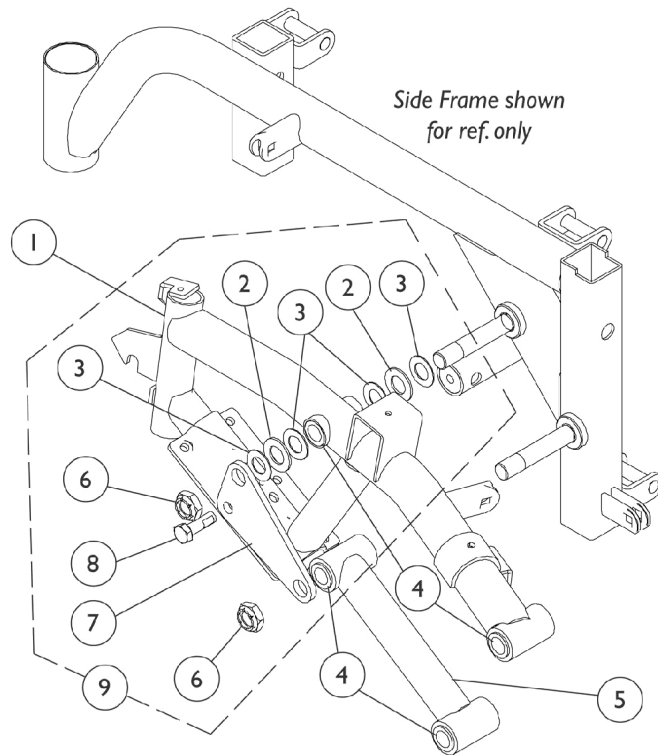

SureStep Walking Beams and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1109025-NLA	NLA - Walking Beam, SureStep - Right	1	1
1	A	1109026-NLA	NLA - Walking Beam, SureStep - Left	1	1
2		40157X015-NLA	NLA - Washer, Nylon (9/16 x 1 x 5/8")	1	2
3		1106492-NLA	NLA - Washer (1/2 x 29/32 x 1/32")	1	4
4		1106485	Bushing (1/2 x 5/8")	1	8
5	A	1109018	Tube, Pivot, Black	1	1
6		1025737	Locknut (1/2-20)	1	2
7		1106078-NLA	NLA - Plate Reinforcement	1	1
8		1050161-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1")	1	1
9	B	1108066-NLA	NLA - Kit, SureStep Walking Beam Hardware	1	1
10		1105030	Bolt, Round Head Square Neck (5/16-18 x 1")	1	3
11		1026158	Locknut (5/16-18)	1	3
12		1105028-NLA	NLA - Spring, Extension (1-1/4 x 3-1/2" L)	1	2
NOTE: A - Includes 4 each Bushings, item 4 B - Includes items 2, 3, 6-8					