



Back Canes & Back Upholstery

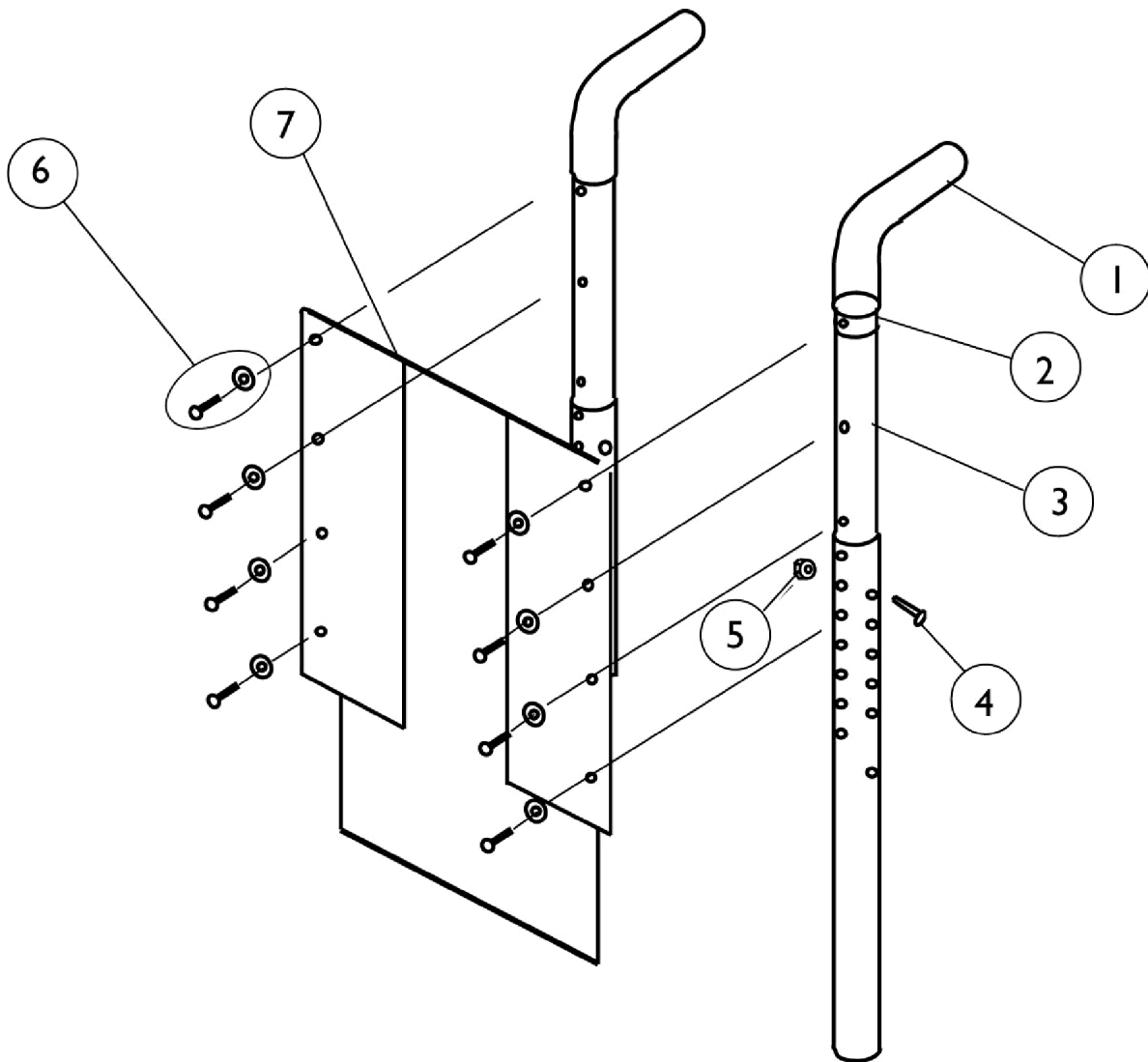

Back Canes & Back Upholstery

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006229	Grip, Foam (8")	1	2
2		1027097	Plug Button, Black (3/4")	1	2
3		1036516-NLA	NLA - Cane, Back with Plug Button, Black	1	2
4		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
5		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
6		1076283	Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	8
6		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
7		See Chart	Back Upholstery without Hardware	1	1

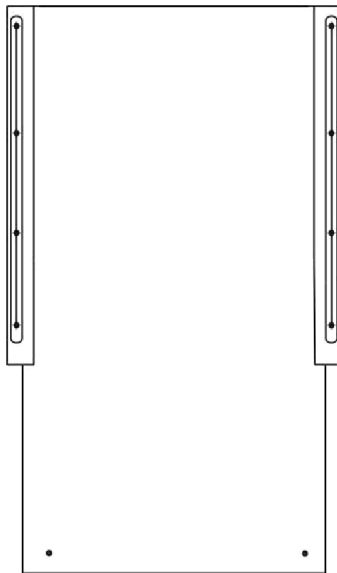

Back Upholstery
without Hardware
- Nylon

Back Upholstery without Hardware

Upholstery	10" Wide	12" Wide	14" Wide
Black Nylon	1019861-NLA	1019862-NLA	1019863-NLA
Vinyl	8881037222-NLA See Note(s) A	8881037223-NLA See Note(s) A	8881037224-NLA See Note(s) A

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors"