


Non-Removable Half Arm

Desk Length

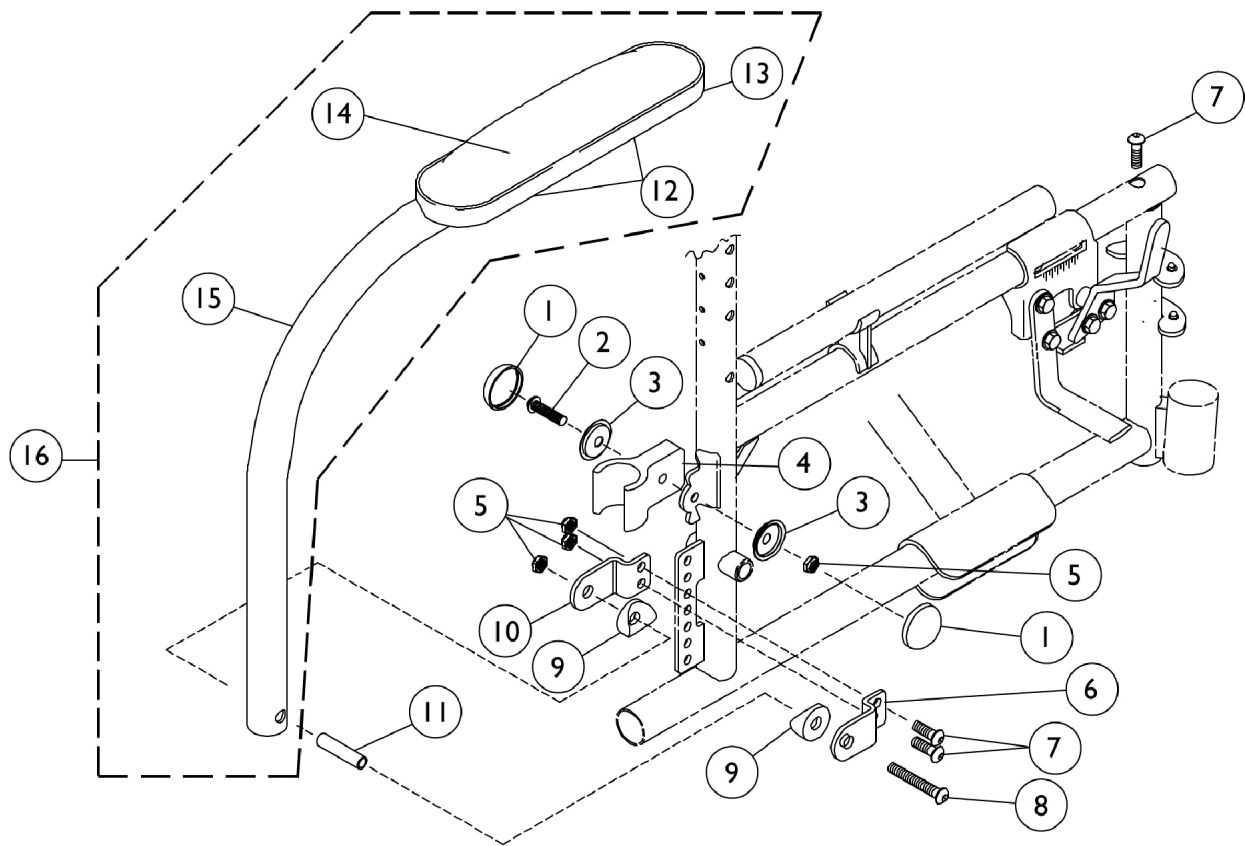

Non-Removable Half Arm Desk Length

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016291	Cap, Hub, Black	1	2
2		1009837	Screw, Socket Button Head (1/4-20 x 1")	1	1
3		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	2
4		1019841	Lock, Arm, Black	1	1
5		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	4
6		1019957-NLA	NLA - Bracket, Outside, Black	1	1
7		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	3
8		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	1
9		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	2
10		1019958-NLA	NLA - Bracket, Inside, Black	1	1
11		1019936	Spacer (1/4 x 11/32 x 1-7/16")	1	1
12		1036501-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-3/8")	2	2
13		1029501	Plug Button, Black (1")	1	1
14	B	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
15		1019933-NLA	NLA - Half Arm Tube - Right	1	1
15		1019934-NLA	NLA - Half Arm Tube - Left	1	1
16	B	1020033-NLA	NLA - Half Arm Tube with Arm Pad - Right	1	1
16	B	1020034-NLA	NLA - Half Arm Tube with Arm Pad - Left	1	1
16	B	1036499-NLA	NLA - Half Arm Tube with Plug Button - Right	1	1
16	B	1036500-NLA	NLA - Half Arm Tube with Plug Button - Left	1	1
	A,C	1036502-NLA	NLA - Half Arm Attaching Hardware (Pair)	2	1
NOTE: A - Not Shown B - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" C - Includes items 1-11					