
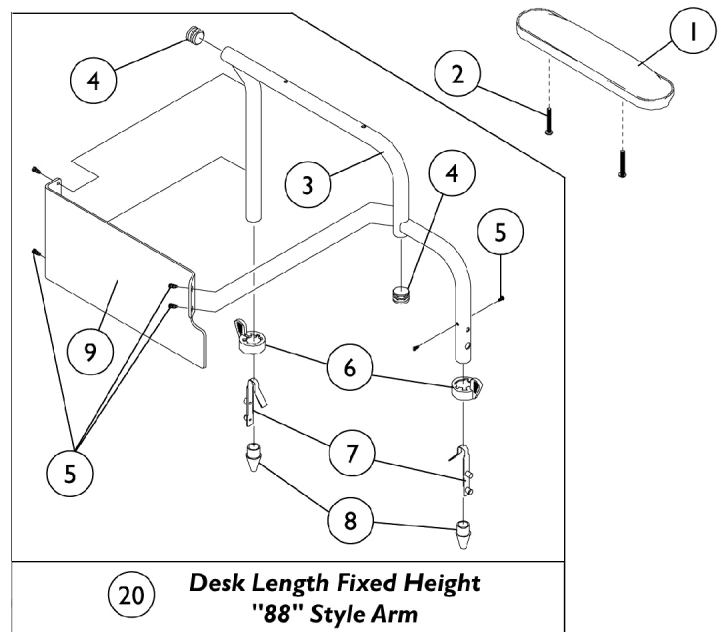


# Space Saver Fixed Height Arm

## Desk Length - "88"

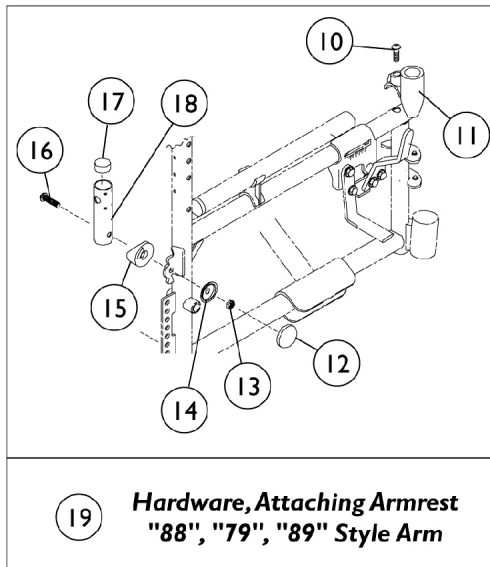

## Space Saver Fixed Height Arm

### Desk Length - "88"

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	2
3		1019978-NLA	NLA - Tube, Armrest, Black - Right	1	1
3		1019979-NLA	NLA - Tube, Armrest, Black - Left	1	1
4		1027097	Plug Button, Black (3/4")	1	2
5		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
5		1116900	Screw, Self Tapping (#6-20 x 1/2")	1	6
6		1015450	Handle, Quad Release	1	2
7		1018225	Spring, Torsion with Button Lock	1	2
8		1020060	Plug, Arm	1	2
9	D,E	1123305-NLA	NLA - Kit, Clothing Guard, Desk Length w/ Screws (Pair)	1	1
10		1003400	NLA - Screw, Socket Head (1/4-20 x 7/8")	1	1
11		1019842-NLA	NLA - Socket, Armrest, Front	1	1
12		1016291	Cap, Hub, Black	1	1
13		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
14		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	1
15		1015426	Spacer, Coved Cam - Right	1	1
15		1015427	Spacer, Coved Cam - Left	1	1
16		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
17		1023975-NLA	NLA - Bushing (3/8 x 3/4 x 3/8")	1	1
18		1015856	Socket, Armrest	1	1
19	B	1036503-NLA	NLA - Hardware, Attaching, Armrest	1	1
20	C,E	1036509-NLA	NLA - Arm Assembly, Desk Length, Black - Right	1	1
20	C,E	1036510-NLA	NLA - Arm Assembly, Desk Length, Black - Left	1	1
<b>NOTE:</b> A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" B - Includes items 10-18 C - Includes items 3-9 D - Clothing Guards are not sold individually. Kit includes clothing guards, screws and decals. E - Due to Clothing Guard change, it is recommended (for chairs built Before 11/25/03) when only one Arm is requested the Clothing Guard Kit (p/n 1123305) be ordered also so that the arms will match.					