
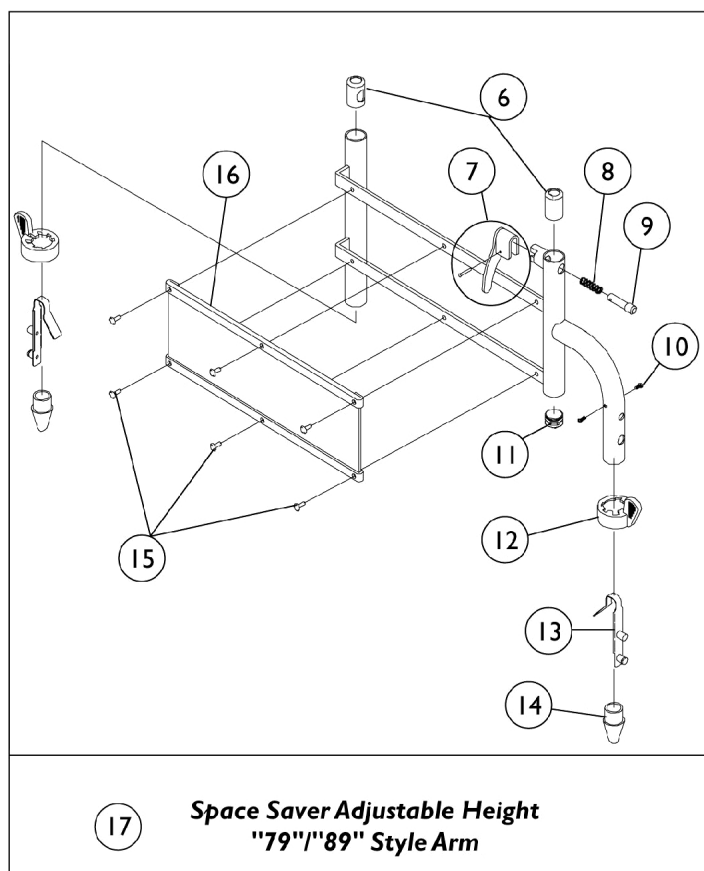


Space Saver Adjustable Height Arm

Desk/Full Length - "79"/"89"

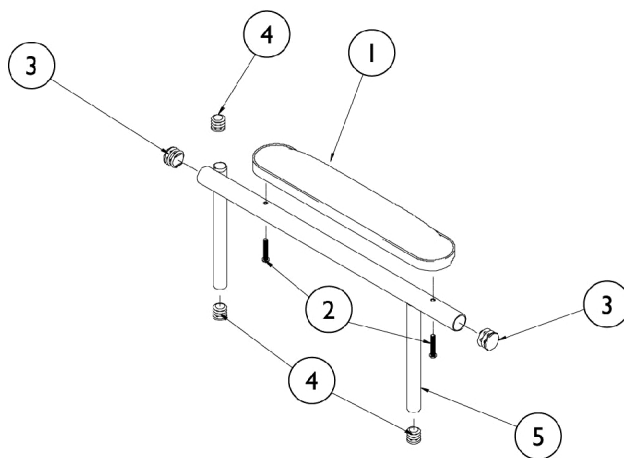

Space Saver Adjustable Height Arm Desk/Full Length - "79"/"89"

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	2
3		1027095	Plug Button, Black (7/8")	1	2
4		1035614	Plug Button, Black (5/8")	1	3
5		1043317-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Right	1	1
5		1043318-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Left	1	1
5		1043319-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right	1	1
5		1043320-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Left	1	1
5		1043313-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Right	1	1
5		1043314-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Left	1	1
5		1043315-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right	1	1
5		1043316-NLA	NLA - Arm Tube w/ Plug Buttons, Desk Length, Black - Left	1	1
6		1036150	Anti-Rattle	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		47471X074	Spring, Arm Lock	1	1
9		47471X068	Pin, Locking	1	1
10		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
10		7930315-NLA	NLA - Package, Screw, Phillips Pan Head (#6-20 x 3/8")	25	1
11		1027097	Plug Button, Black (3/4")	1	1
12		1015450	Handle, Quad Release	1	2
13		1018225	Spring, Torsion with Button Lock	1	2
14		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
15		1028645	Rivet	1	6
16		1068511	Guard, Clothing	1	1
17	B	1020027-NLA	NLA - Arm Base Assembly, Chrome - Right	1	1
17	B	1020028-NLA	NLA - Arm Base Assembly, Chrome - Right	1	1
17	B	1037634	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Right	1	1
17	B	1037635	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Left	1	1
17	A,B	8881037636-NLA	NLA - Arm Base Assembly, Color - Right	1	1
17	A,B	8881037637-NLA	NLA - Arm Base Assembly, Color - Left	1	1
	C,D	1036503-NLA	NLA - Hardware, Attaching, Armrest	1	1

Space Saver Adjustable Height Arm

Desk/Full Length - "79"/"89"

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors"
B - Includes items 6-16
C - Not Shown
D - See page titled "Space Saver Fixed Height Arm - Desk Length - 88" item number 19 "Hardware, Attaching Armrest"