


Forks and Hardware

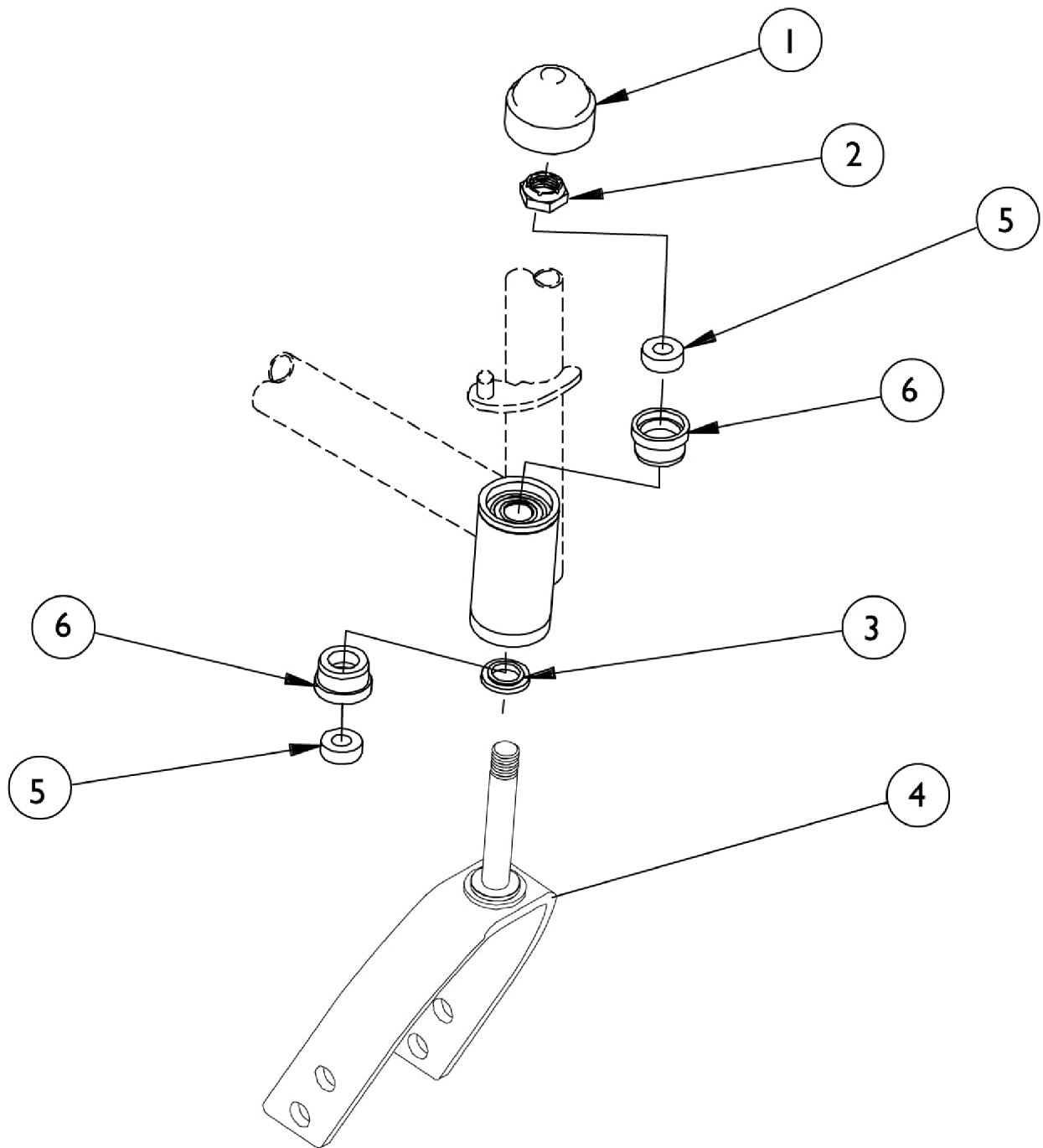

Forks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001843	Cap, Head Tube, Black	1	1
1		1131189	Package, Head Tube Cap	10	1
2		20225X000	Locknut (7/16-20)	1	1
3		1016300-NLA	NLA - Washer, Fork (3/4 x 7/16 x 1/8")	1	1
4	A,B	1060992-NLA	NLA - Fork and Stem (8" Caster)	2	1
4	C	1023237-NLA	NLA - Fork and Stem (8" Caster)	1	1
4		1019972	Fork, 6" Caster (7/16" Stem)	1	1
5		1006049	Bearing (7/16")	1	2
6		57768X030-NLA	NLA - Sleeve, Aluminum	1	2
NOTE: A - Kit contains 2 forks plus instruction sheet. As of 12/95 forks changed to 2 axle mounting holes. Those chairs built before 12/95 will need to replace both forks. B - Before 12/95 C - After 11/95					