


Rear Wheel Hardware

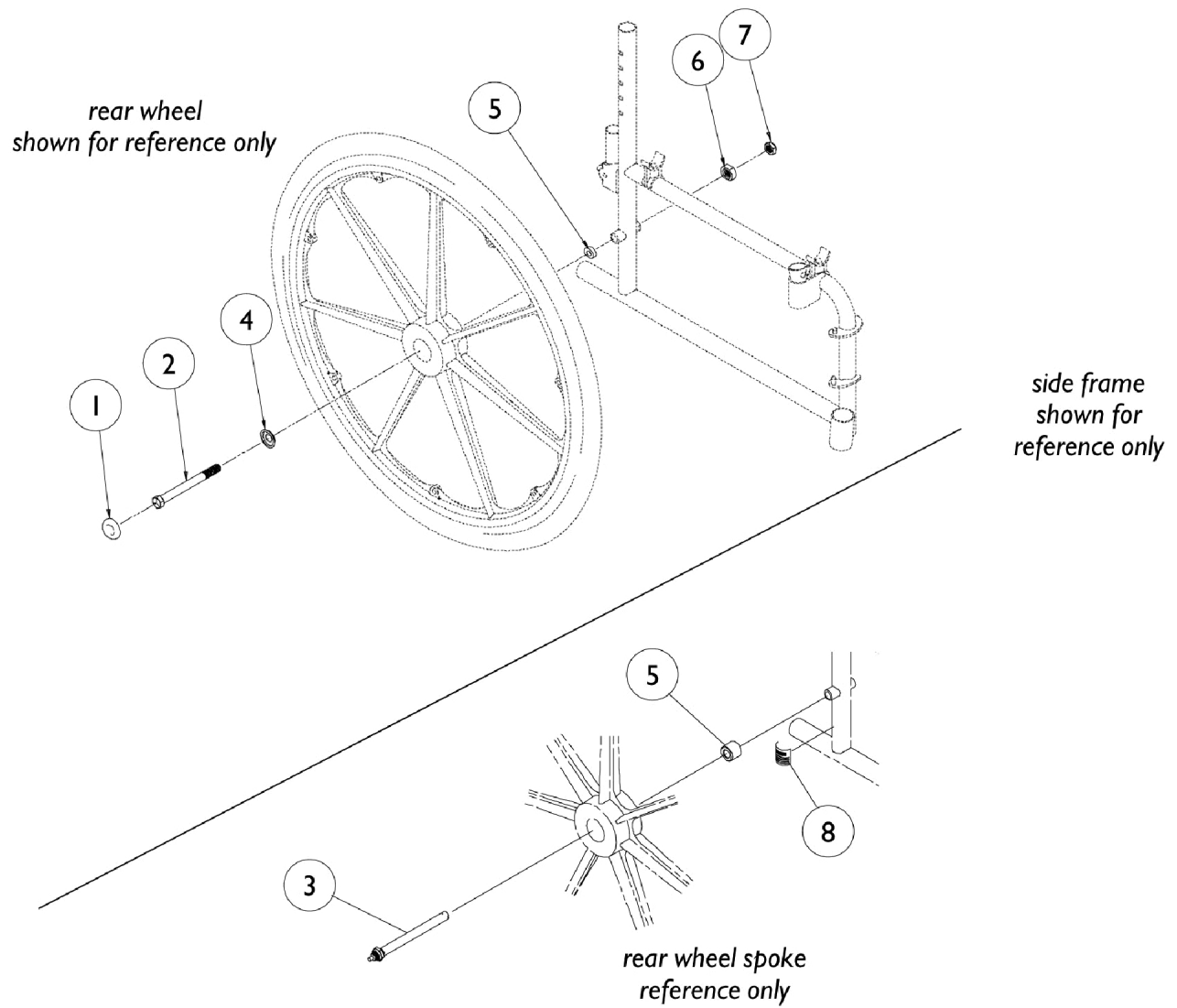

Rear Wheel Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1016291	Cap, Hub, Black	1	1
1	C	1027600-NLA	NLA - Cap, Black	1	1
2		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
3	E	56000X651	Axle, Quick Release (7/16" x 5-7/16")	1	1
4		1016289	Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
5		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
6	A	1014652-NLA	NLA - Bushing, Threaded	1	1
	D	1048319-NLA	NLA - Locknut Retrofit Kit	1	1
7		1045608	Locknut (7/16-20)	1	1
8		1001948-NLA	NLA - Label, Warning	1	1
NOTE: A - Bushing is NLA as of 7/94. Order (1 ea.) Locknut Retro-fit Kit part number 1048319. B - Pneumatic and Urethane Wheels C - DTM Wheels D - Not Shown, replaced item 6. E - For chairs with Conventional Arms					