


Wheel Lock

without Attaching Hardware

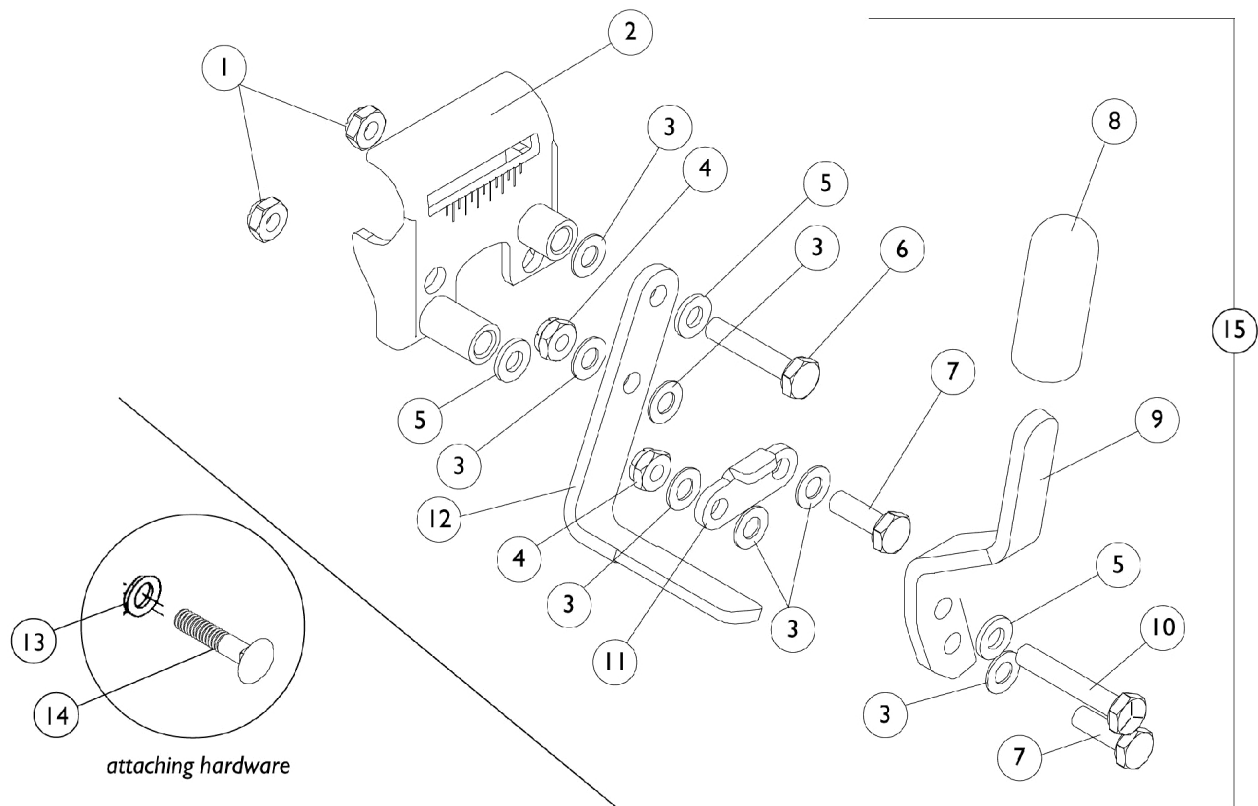

Wheel Lock without Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
2		1016037-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
2		1016038-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
3		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	7
4		7930367-NLA	NLA - Locknut (1/4-28)	25	2
5		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
6		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
7		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
8		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
9		1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
9		1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
10		7930350-NLA	NLA - Package, Hex Head Screw (1/4-20 x 1-1/2")	25	1
11	B	1016592-NLA	NLA - Link, Chrome	1	1
12		1015443-NLA	NLA - Shoe	1	1
13		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
14		60264X010-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	1
15	A	1020001-NLA	NLA - Wheel Lock Assembly, Chrome - Right	1	1
15	A	1020002-NLA	NLA - Wheel Lock Assembly, Chrome - Left	1	1
NOTE: A - Includes items 1-12 B - All Chairs Except Super Hemi					