


Hill Holder Wheel Lock

without Attaching Hardware

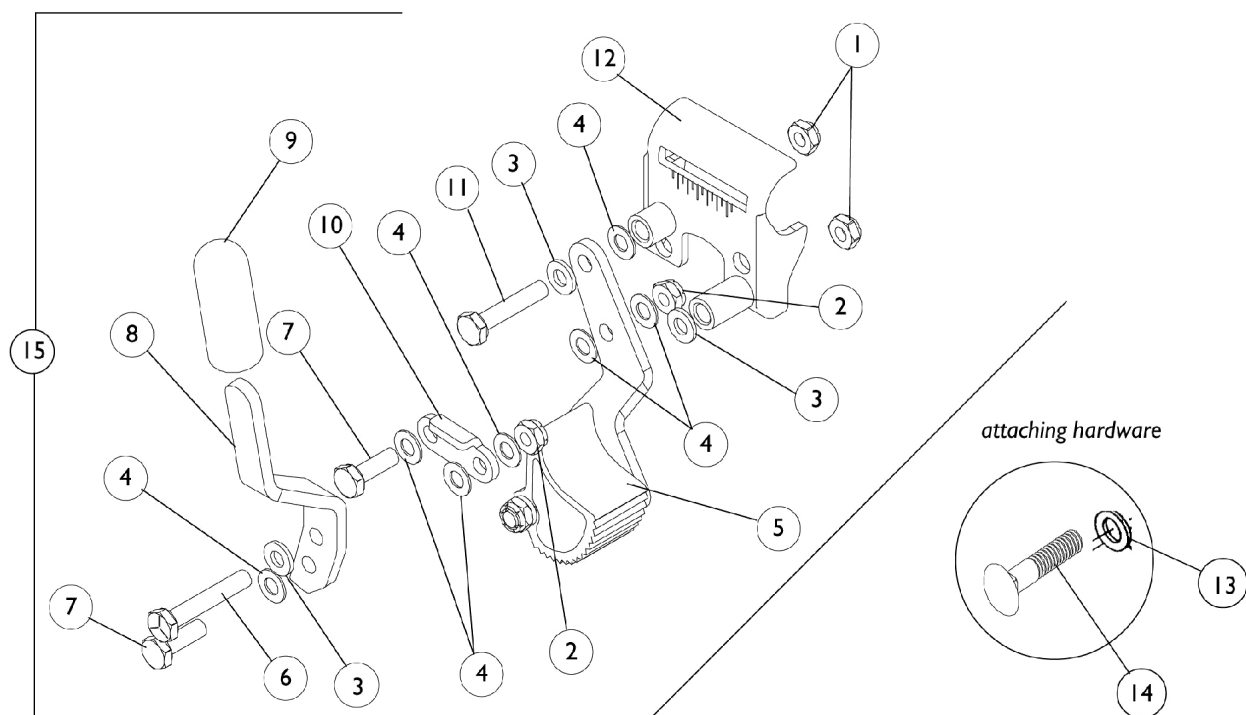

Hill Holder Wheel Lock without Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
2		7930367-NLA	NLA - Locknut (1/4-28)	25	2
3		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	7
4		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	3
5	C	1016892-NLA	NLA - Hill Holder Assembly - Right	1	1
5	C	1016893-NLA	NLA - Hill Holder Assembly - Left	1	1
6		7930350-NLA	NLA - Package, Hex Head Screw (1/4-20 x 1-1/2")	25	1
7		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
8		1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
8		1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
9		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
10		1017861-NLA	NLA - Link, Chrome	1	1
11		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
12		1016037-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
12		1016038-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
13		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
14		60264X010-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	1
15	A,B	1034210	Wheel Lock Assembly, Hill Holder, Push to Lock - Right	1	1
15	A,B	1034211	Wheel Lock Assembly, Hill Holder, Push to Lock - Left	1	1
NOTE: A - Hill Holder Wheel Locks not for use with Urethane Tires B - Includes Items 1-12 C - Part #1016892 and 1016893 are no longer available as a service part individually. Order item #15 as a replacement.					