


Footrest Support Assembly H93E

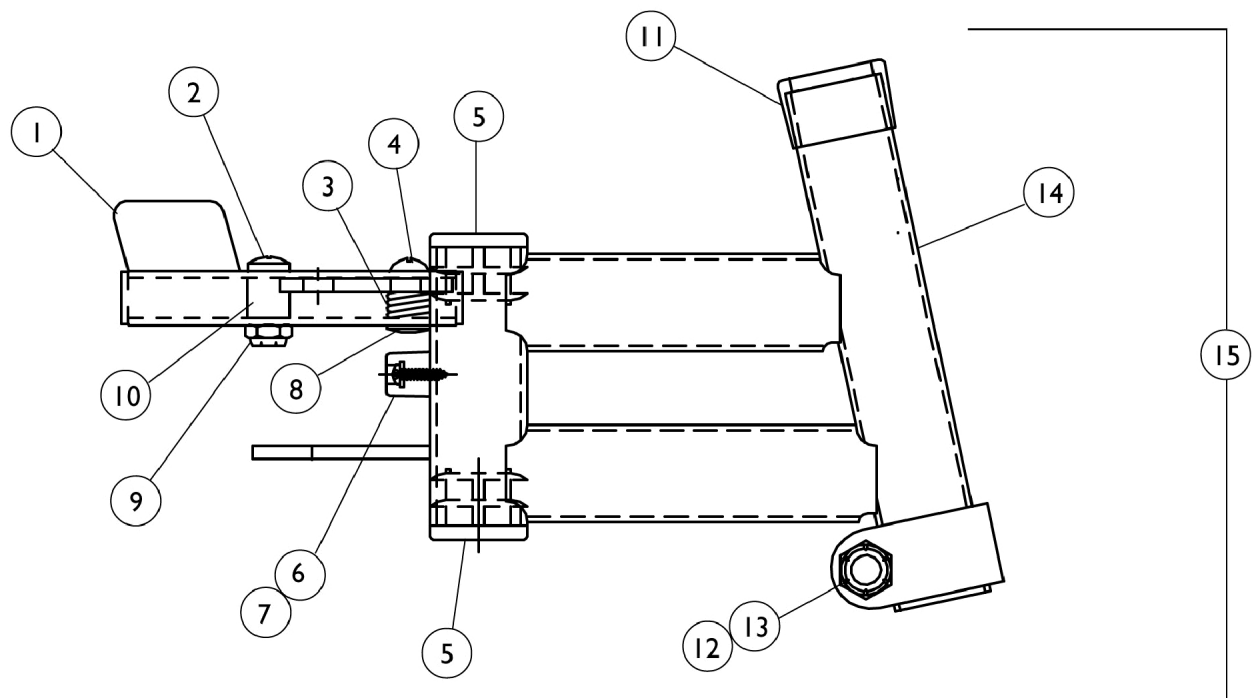

Footrest Support Assembly H93E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035539	Handle, Release, Black - Right	1	1
1		1035540	Handle, Release, Black - Left	1	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
3		1008935	Spring, Release - Right	1	1
3		1008936	Spring, Release - Left	1	1
4		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		1027095	Plug Button, Black (7/8")	1	2
6		1022874	Bumper	1	1
7		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
8		67005X000	Rivnut (#10-32)	1	1
9		7930365-NLA	NLA - Locknut (#10-32)	25	1
10		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
11		15680X012	Collar, Anti-Rattle (7/8")	1	1
12		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
13		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
14	B	1007123-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
14		1007124-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
	A	See Page	See Footnote	1	1
NOTE: A - See page titled "Front Riggings Index" for Support Assembly B - Part #1007123 is no longer available individually. See Catagory titled "Front Riggings Index and Calf Pads" and order part #1040715 Support Assembly					
Due to design change, for alll chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.					