


Footrest Support Assembly

H93M

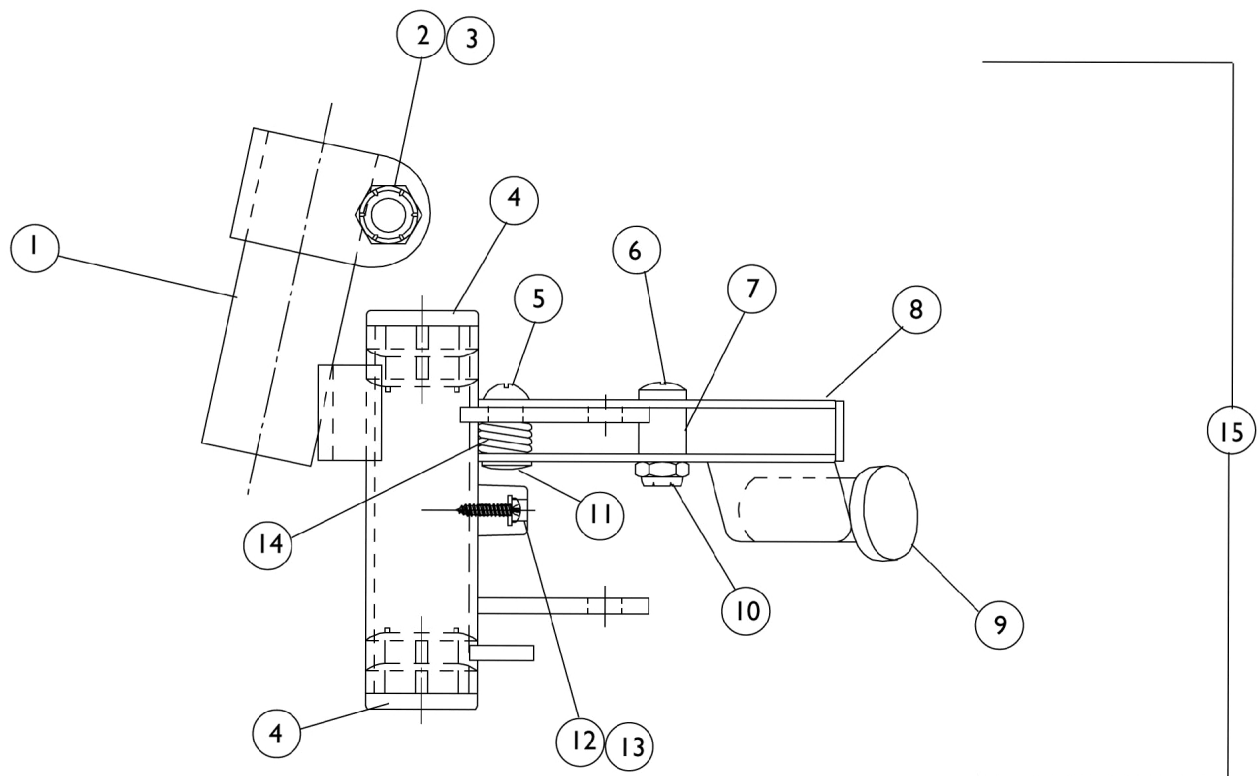

Footrest Support Assembly H93M

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1035141-NLA	NLA - Footrest Support, Chrome - Right	1	1
1		1035140-NLA	NLA - Footrest Support, Chrome - Left	1	1
2		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
3		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
6		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
7		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
8		1035539	Handle, Release, Black - Right	1	1
8		1035540	Handle, Release, Black - Left	1	1
9		1006490-NLA	NLA - Tip, Plastic, Black	1	1
10		7930365-NLA	NLA - Locknut (#10-32)	25	1
11		67005X000	Rivnut (#10-32)	1	1
12		1022874	Bumper	1	1
13		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
14		1008935	Spring, Release - Right	1	1
14		1008936	Spring, Release - Left	1	1
	B	See Page	See Footnote	1	1
NOTE: A - Before 10/1/00 B - See page titled "Front Riggings Index" for Support Assembly					
Due to design changes, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.					