


Footrest Support Assembly

H903MC/H903MK

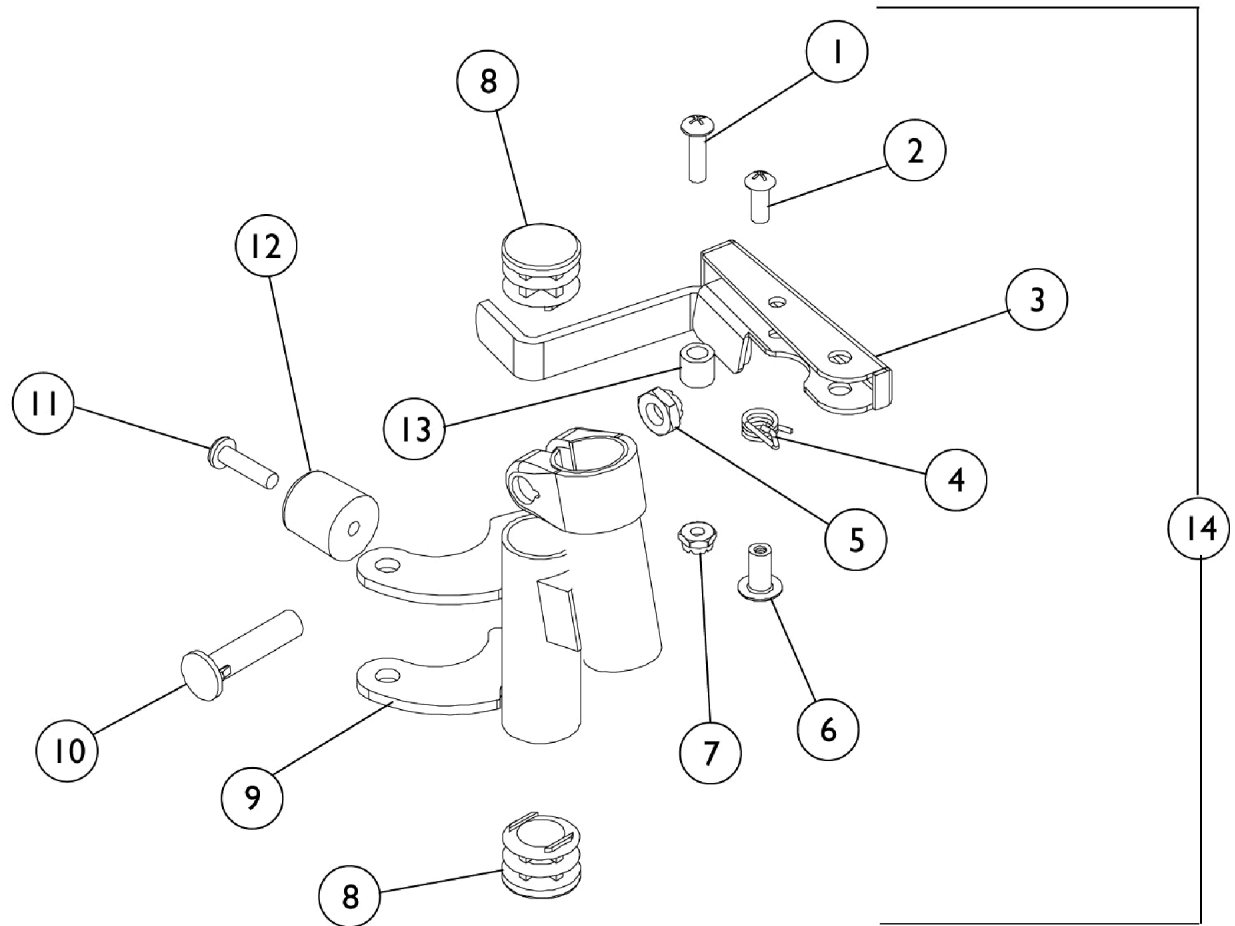

Footrest Support Assembly H903MC/H903MK

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
2		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
3		1035539	Handle, Release, Black - Right	1	1
3		1035540	Handle, Release, Black - Left	1	1
4		1008935	Spring, Release - Right	1	1
4		1008936	Spring, Release - Left	1	1
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		67005X000	Rivnut (#10-32)	1	1
7		7930365-NLA	NLA - Locknut (#10-32)	25	1
8		1027095	Plug Button, Black (7/8")	1	2
9	A	1007113-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
9	A	1007114-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
9	B,D	1021929-NLA	NLA - Tube, Footrest Support, Hemi, Black - Right	1	1
9	B,D	1021930-NLA	NLA - Tube, Footrest Support, Hemi, Black - Left	1	1
10		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
11		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
12		1022874	Bumper	1	1
13		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
14	C	See Page	See Footnote	1	1
NOTE: A - Chrome B - Black C - See page titled "Front Riggings Index" for Support Assembly D - Part #1021929 and 1021930 are no longer available as service parts individually. See category titled "Front Rigging Index" and order part #8881031198 or 8881031199. Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.					