


# 90 Degree Footrest with Aluminum Footplate

H990C/H990K

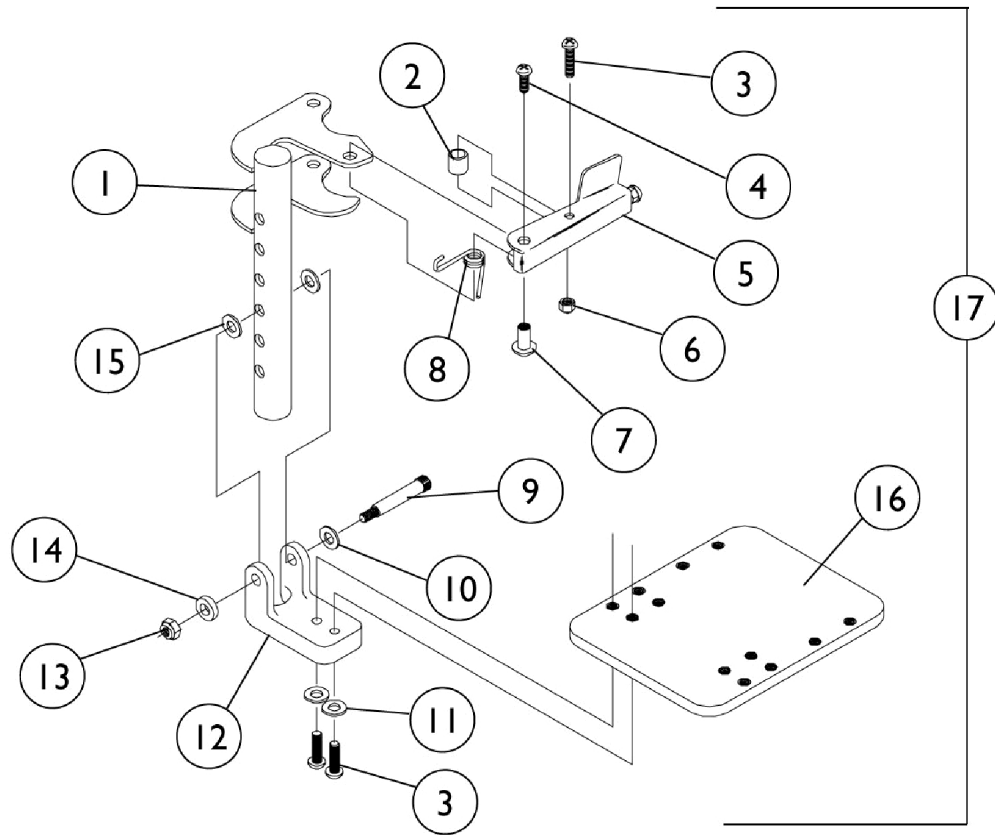

## 90 Degree Footrest with Aluminum Footplate H990C/H990K

| Item Number | Note | Part Number   | Description                                               | Quantity Per |          |
|-------------|------|---------------|-----------------------------------------------------------|--------------|----------|
|             |      |               |                                                           | Package      | Assembly |
| 1           |      | 1019869-NLA   | NLA - Tube, Footrest Support, Hemi, Chrome - Right        | 1            | 1        |
| 1           |      | 1019870-NLA   | NLA - Tube, Footrest Support, Hemi, Chrome - Left         | 1            | 1        |
| 1           |      | 1019867-NLA   | NLA - Tube, Footrest Support, Hemi, Black - Right         | 1            | 1        |
| 1           |      | 1019868-NLA   | NLA - Tube, Footrest Support, Hemi, Black - Left          | 1            | 1        |
| 2           |      | 47565X057     | Bushing (7/32 x 3/8 x 11/32")                             | 1            | 1        |
| 3           |      | 7930360-NLA   | NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16") | 25           | 3        |
| 4           |      | 7930359-NLA   | NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")   | 25           | 1        |
| 5           |      | 1031469-NLA   | NLA - Handle, Release, Black - Right                      | 1            | 1        |
| 5           |      | 1031470-NLA   | NLA - Handle, Release, Black - Left                       | 1            | 1        |
| 6           |      | 7930365-NLA   | NLA - Locknut (#10-32)                                    | 25           | 1        |
| 7           |      | 67005X000     | Rivnut (#10-32)                                           | 1            | 1        |
| 8           |      | 1008935       | Spring, Release - Right                                   | 1            | 1        |
| 8           |      | 1008936       | Spring, Release - Left                                    | 1            | 1        |
| 9           |      | 1000212       | Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)         | 1            | 1        |
| 10          |      | 1010398       | Washer, Nylon (1/4 x 1/2 x 1/16")                         | 1            | 1        |
| 11          |      | 1028516       | Washer, Steel (3/16 x 7/16 x 1/16")                       | 1            | 2        |
| 12          |      | 1022817-NLA   | NLA - Block, Hinge                                        | 1            | 1        |
| 13          |      | 10092X000-NLA | NLA - Locknut (1/4-2)                                     | 1            | 1        |
| 14          |      | 16347X000-NLA | NLA - Washer, Nylon (3/16 x 7/16 x 1/8")                  | 1            | 1        |
| 15          |      | 7930357-NLA   | NLA - Washer, Nylon (1/4 x 1/2 x 1/32")                   | 25           | 2        |
| 16          | B    | 1044078-NLA   | NLA - Footplate, Black (3-1/4" W x 5-1/4" L) (10" W)      | 1            | 1        |
| 16          | B    | 1044082-NLA   | NLA - Footplate, Black (4-1/4" W x 5-1/4" L) (12" W)      | 1            | 1        |
| 17          | D    | See Page      | See Footnote                                              | 1            | 1        |
|             | A,C  | 1018525-NLA   | NLA - Pad, Footplate (10" W)                              | 1            | 1        |
|             | A,C  | 1018526       | Pad, Footplate (12 - 14" W)                               | 1            | 1        |

NOTE: A - Not Shown  
 B - As of 4/94 this footplate now has a "wrinkle" finish versus the previous "flat black", which are NLA. When replacing 1 footplate on chairs built before 4/94, order 2 of the current footplates.  
 C - Before 4/94 Only  
 D - See page titled "Front Riggings Index" for Support Assembly with Footplate

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.