


Legrest Support Assembly H904M

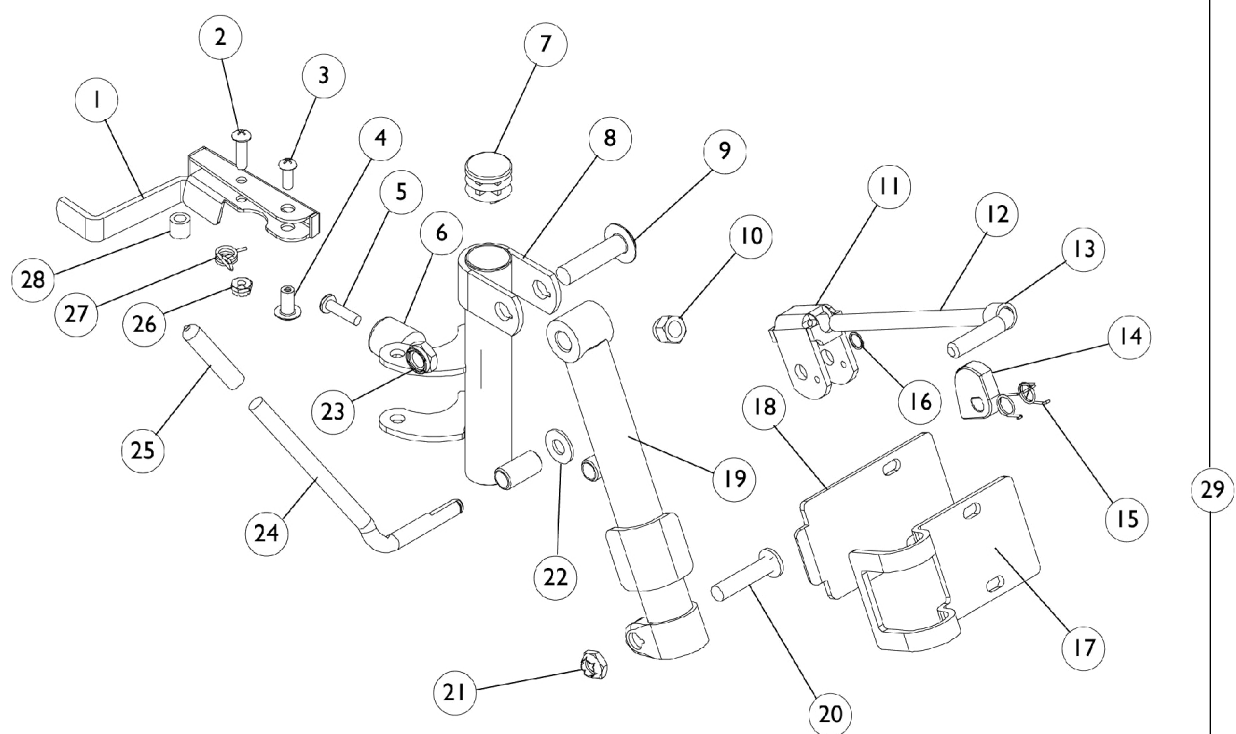

Legrest Support Assembly

H904M

H904M

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1023428-NLA	NLA - Handle, Release, Black- Right	1	1
1		1023431-NLA	NLA - Handle, Release, Black- Left	1	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
3		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
4		67005X000	Rivnut (#10-32)	1	1
5		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
6		1022874	Bumper	1	1
7		1027095	Plug Button, Black (7/8")	1	1
8		04707X069-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
8		04707X070-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
8		04707X067	Legrest Support Only, Black- Right	1	1
8		04707X068	Legrest Support Only, Black- Left	1	1
9		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
10		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
11	B	04528X002	Bracket, Release, Black	1	1
12		42326X001	Bar, Ratchet	1	1
13		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
14	B	1037125	Cam, Legrest	1	1
15	B	1037485	Spring, Release	1	1
16	C,D	1040831	Ring, Snap	1	1
16	C,E	40187X000-NLA	NLA - Pin, Roll (1/8" x 9/16")	1	1
17		47352X004	Clamp, Black	1	1
18		47352X003	Lock, Pad Plate, Legrest, Black	1	1
19		04526X006-NLA	NLA - Tube, Elevating Support, Chrome - Right	1	1
19		04526X007-NLA	NLA - Tube, Elevating Support, Chrome - Left	1	1
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
22		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
23		40363X000	Hex Nut (3/8"-16)	1	1
24	B	1037126	Handle, Release	1	1
25	D	1040060	Tip, Plastic, Black	1	1
25	E	1006490-NLA	NLA - Tip, Plastic, Black	1	1
26		7930365-NLA	NLA - Locknut (#10-32)	25	1
27		1008935	Spring, Release - Right	1	1
27		1008936	Spring, Release - Left	1	1
28		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
29	G	See Page	See Footnote	1	1
	A,B,F	1046328-NLA	NLA - Kit, Ratchet	2	1

Legrest Support Assembly

H904M

NOTE: A - Not shown
B - Due to product improvement - items 7, 9, 12, and 13 are NLA for chairs built before 6/94. Ratchet bar kit (item 5) must be ordered for those chairs.
C - After 5/94, the snap ring replaced the roll pin to attach the ratchet handle to the ratchet.
D - After 5/94
E - Before 6/94
F - Includes Items 10-14, 16, 22, 24, 25 & instruction sheet.
G - See page titled "Front Riggings Index" for Support Assembly

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.