


Pivot Slide Tube with Footplate

H93E / H93M / H903MC / H903MK Footrests

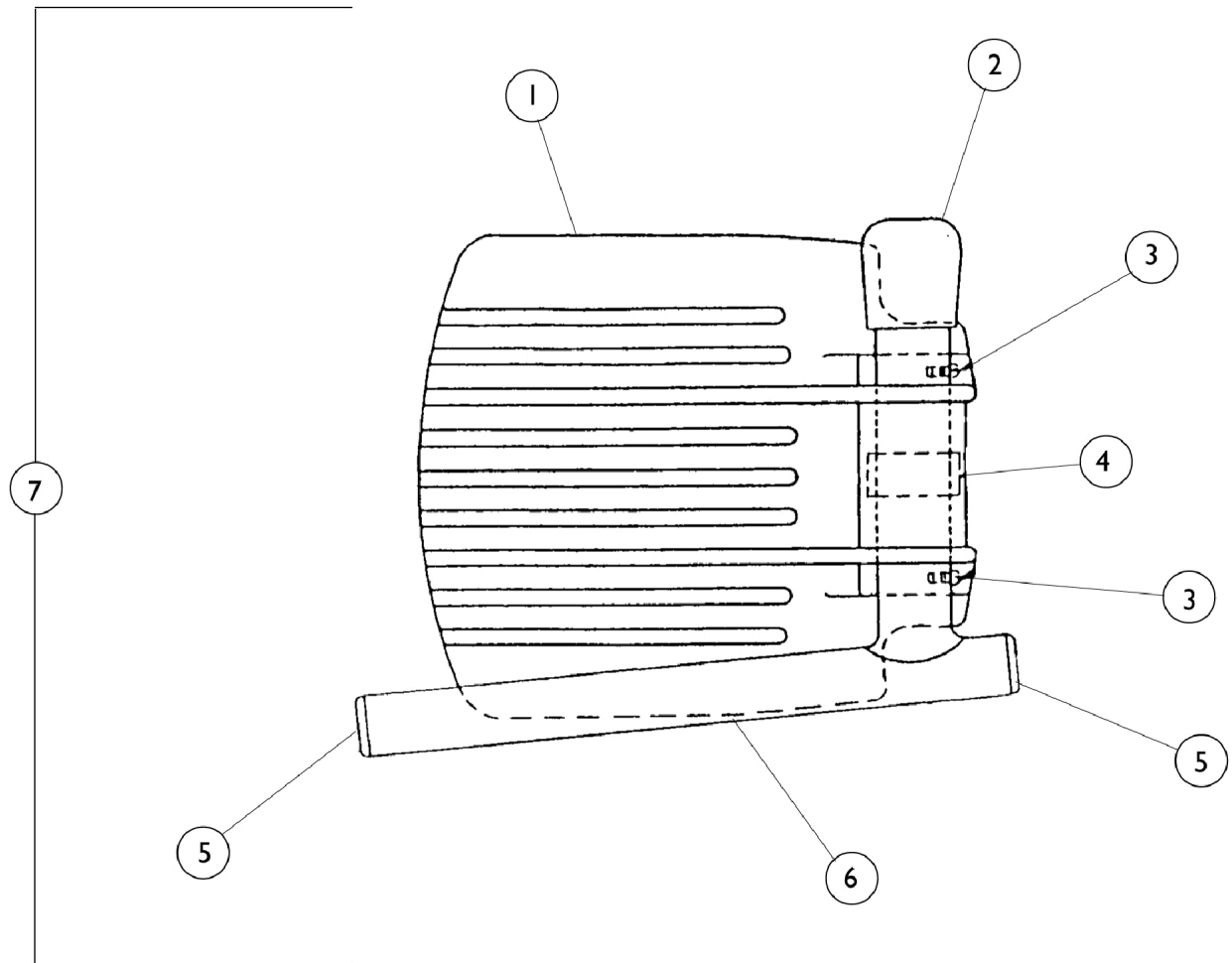

Pivot Slide Tube with Footplate

H93E / H93M / H903MC / H903MK Footrests

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
1	A,C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
2		1016346	Cap, End, Tube, Black (7/8")	1	1
3		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
4		1006981	Spring, Tension	1	1
5		1027097	Plug Button, Black (3/4")	1	2
6		1041528-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Chrome	1	1
7		See Page	See Footnote	1	1
NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Ball Burnish" finish. Footplates after 1/96 now have a black finish. When replacing 1 ball burish footplate it is suggested that 2 new (black) footplates be ordered. B - 10-12" Wide Chairs C - 14" Wide Chairs					