


1" Forward Option

Available on Chairs with 20" and 22" Rear Wheels Only

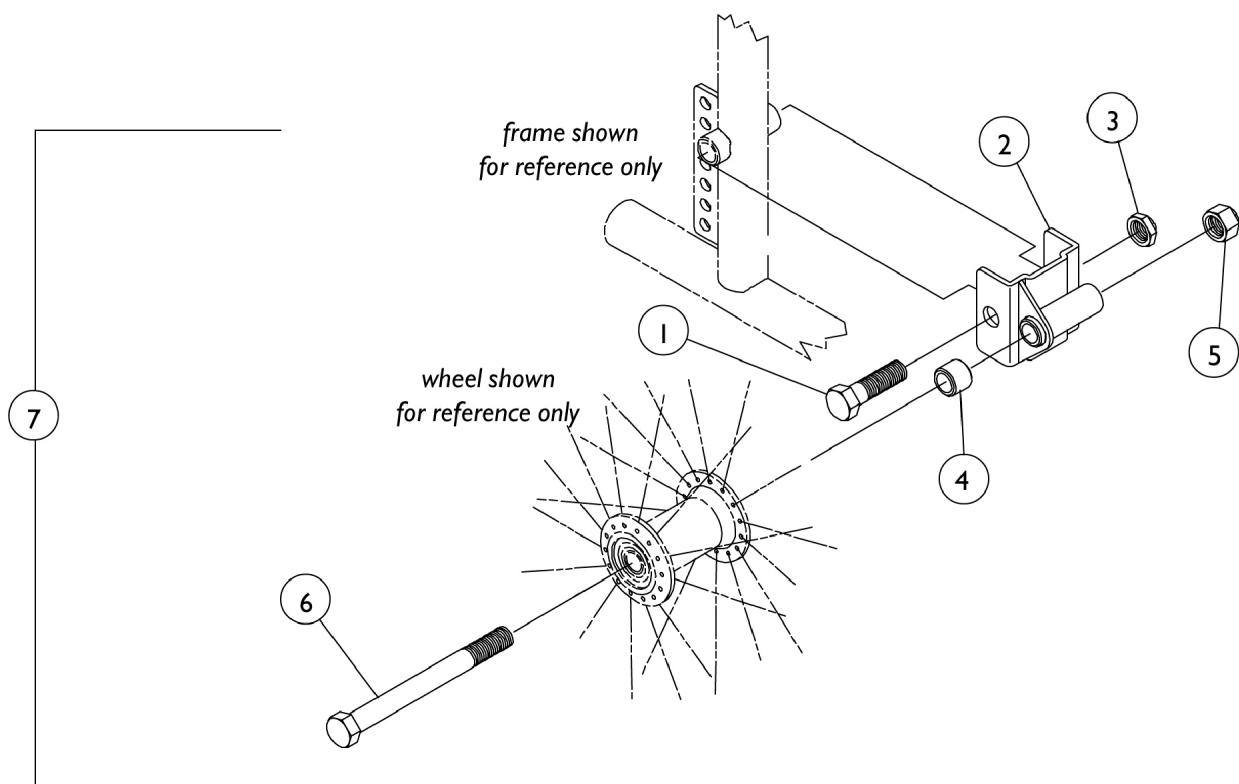

1" Forward Option

Available on Chairs with 20" and 22" Rear Wheels Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1021755-NLA	NLA - Screw, Hex Head (7/16-20 x 2-1/2")	1	1
2		1021691-NLA	NLA - Bracket, Forward, Chrome (1")	1	1
3		7930352-NLA	NLA Locknut (7/16-20)	25	1
4		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
5		1045608	Locknut (7/16-20)	1	1
6		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
7		1036619-NLA	NLA - Option, Forward (1") (Pair)	2	1
NOTE: A - Part no. 1045608 Locknut, replaced Threaded Bushing as of 7/94.					