


Headrest Extension

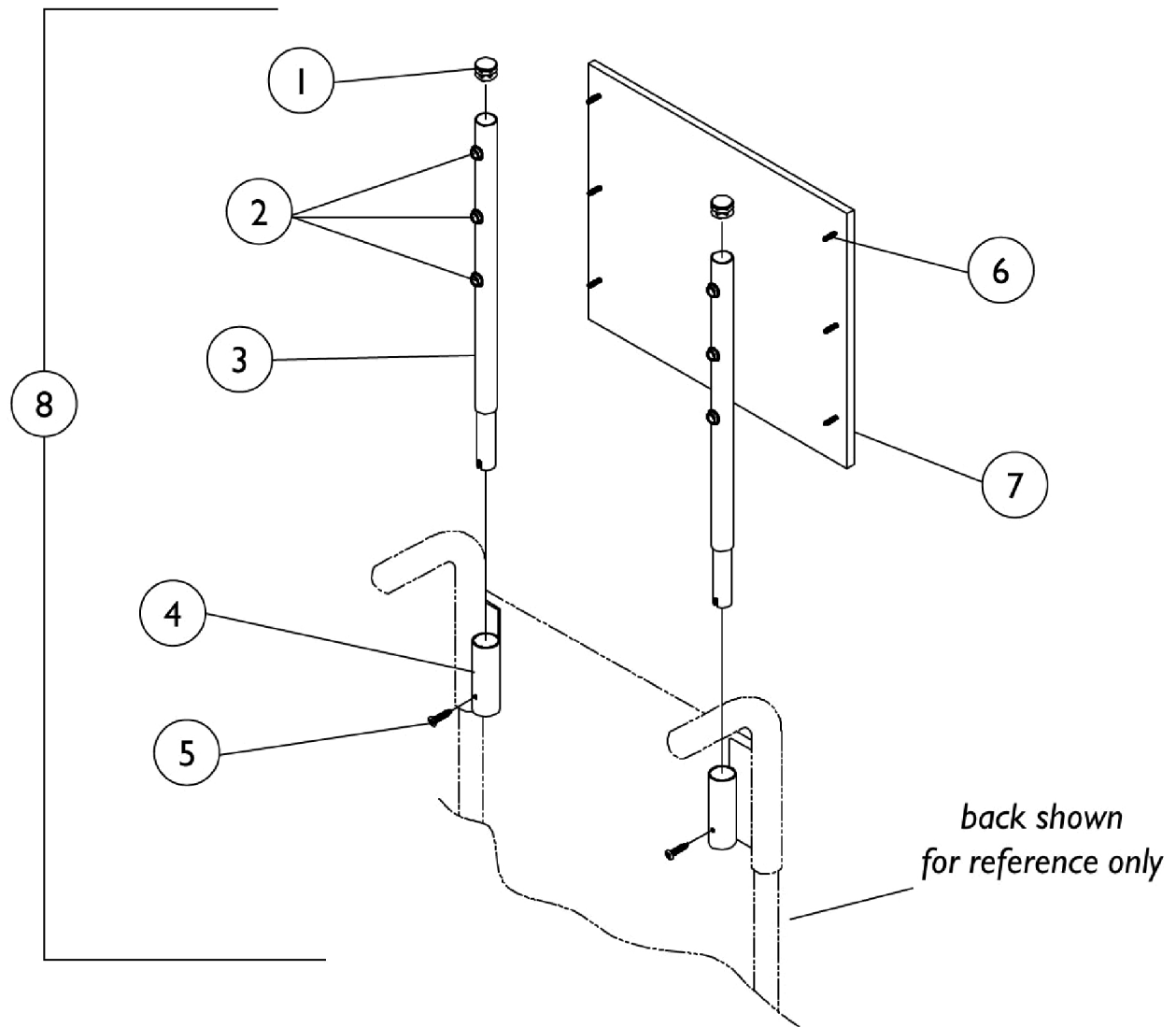

Headrest Extension

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	2
2		47447X009-NLA	NLA - Screw, Phillips (#8 x 3/8")	1	6
3		1055152-NLA	NLA - Headrest Extension Tube with Fastners	1	2
4		1053110-NLA	NLA - Bracket, Headrest, Black - Right	1	1
4		1053111-NLA	NLA - Bracket, Headrest, Black - Left	1	1
5		40163X013-NLA	NLA - Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	2
6		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
	A	See Chart	Headrest Extension Assembly	1	1
	A	See Chart	Headrest Upholstery with Screws	1	1

NOTE: A - Not Shown

#7 - Headrest Upholstery with Screws

Width	10"	12"	14"
Vinyl	8881053097-NLA See Note(s) A	8881053099-NLA See Note(s) A	8881053101-NLA See Note(s) A
Nylon	1053098-NLA	1053100-NLA	1053102-NLA

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".

#8 - Headrest Extension Assembly

Width	10"	12"	14"
Vinyl	8881053112-NLA See Note(s) A	8881053114-NLA See Note(s) A	8881053116-NLA See Note(s) A
Nylon	1053113-NLA	1053115-NLA	1053117-NLA

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".