


# Van Seat Frame and Mounting Hardware Before 9/1/02

Note: Seat post can be found on the base frame assembly page.

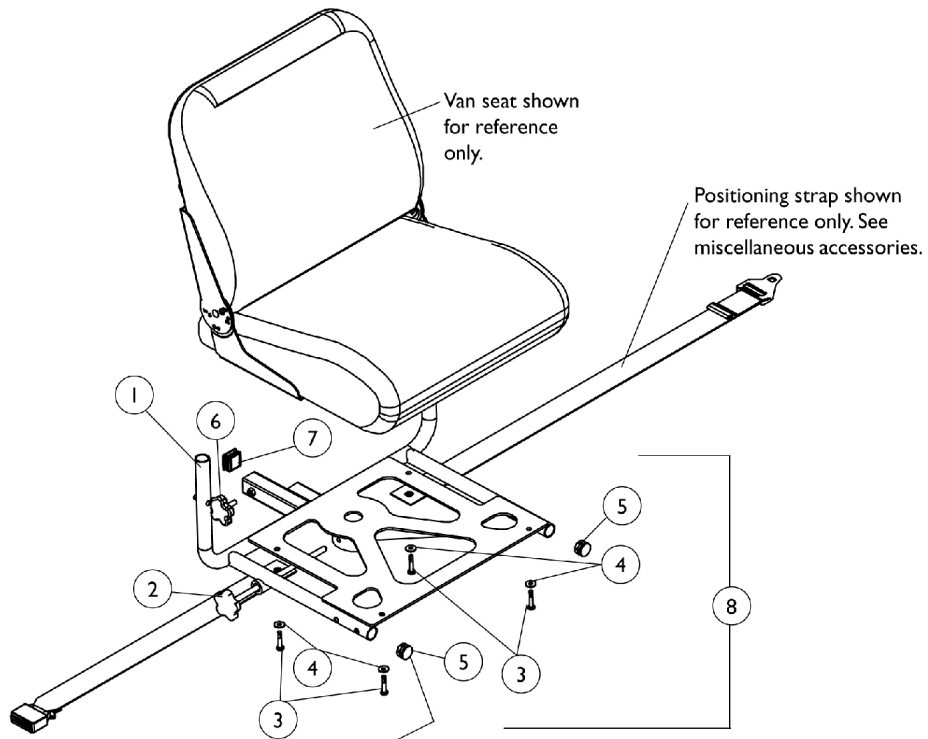

**NOTE:** Plug buttons are used when the footboard assembly is present.

# Van Seat Frame and Mounting Hardware

## Before 9/1/02

**Note: Seat post can be found on the base frame assembly page.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1110597-NLA	NLA - Frame, Van Seat, Black	1	1
1	C	1111696-NLA	NLA - Frame, Van Seat, Black	1	1
2	B	1114998-NLA	NLA - Knob Assembly, Black (3/8-16 x 9-1/4")	1	1
2	C	1114999-NLA	NLA - Knob Assembly, Black (3/8-16 x 10-1/4")	1	1
3		1088897-NLA	NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-1/4")	1	4
4		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
5		1029501	Plug Button, Black (1")	1	2
6		1114911-NLA	NLA - Knob, Adjustment, Black (5/16-18 x 1-1/2")	1	1
7		1093289-NLA	NLA - Plug Button, Square, Black (1-1/4")	1	1
8	A,B	1109485-NLA	NLA - Hardware, Van Seat Mounting w/ Plug Buttons	1	1
8	B,D	1109491-NLA	NLA - Hardware, Van Seat Mounting w/ Plug Buttons	1	1
8	A,C	1111702-NLA	NLA - Hardware, Van Seat Mounting w/ Plug Buttons	1	1
8	C,D	1110623-NLA	NLA - Hardware, Van Seat Mounting w/ Plug Buttons	1	1

NOTE: A - Includes Items 2-7  
 B - 18" Wide Seat  
 C - 20" Wide Seat  
 D - Includes Items 2-4, 6, 7