


Wheel Lock Non-Recliner with Removable Arm

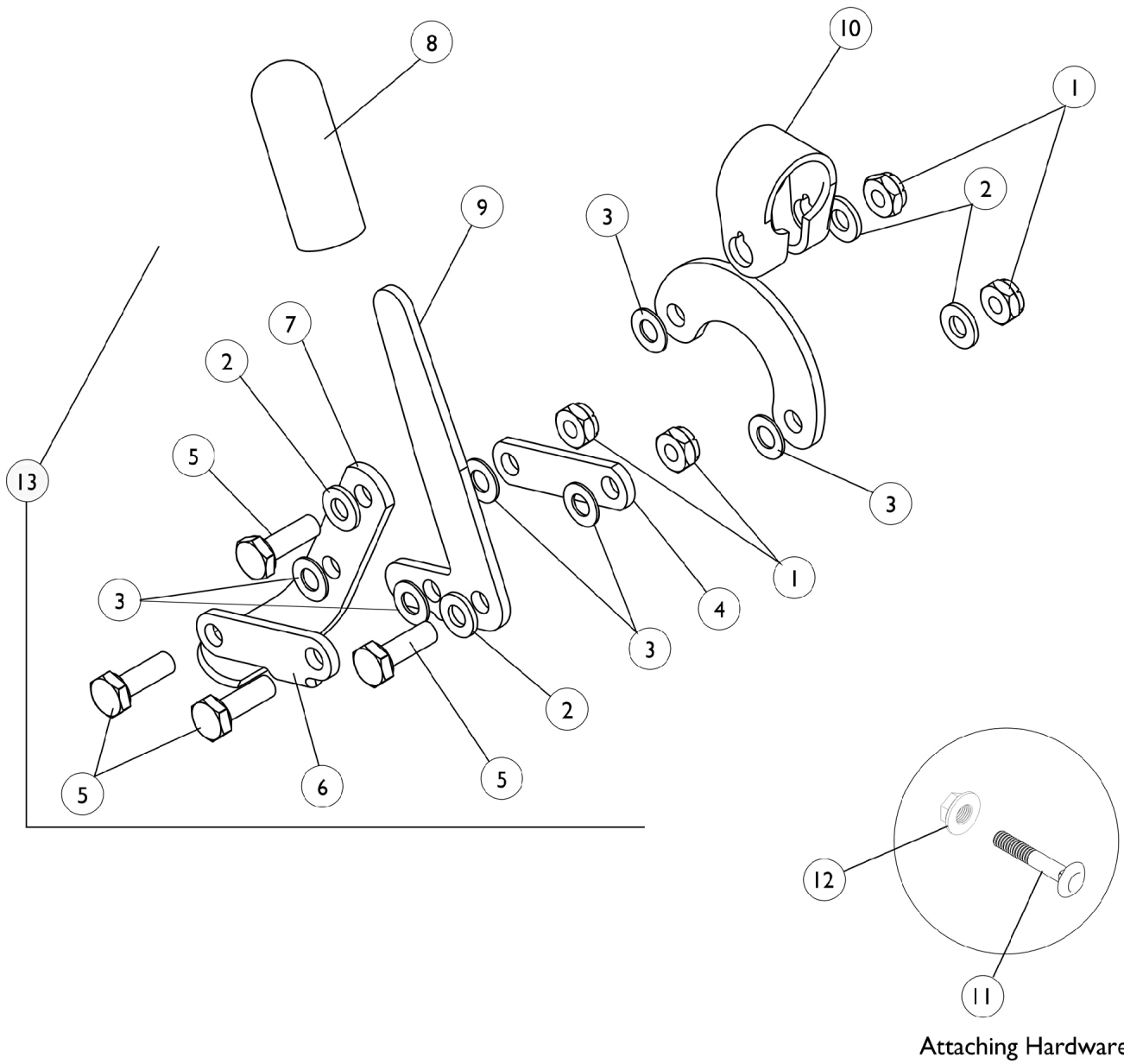

Wheel Lock Non-Recliner with Removable Arm

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930367-NLA	NLA - Locknut (1/4-28)	25	2
3		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	6
4		47559X024-NLA	NLA - Link, Flat, Chrome	1	1
5		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
6		47559X025-NLA	NLA - Link, Chrome - Right	1	1
6		47559X026-NLA	NLA - Link, Chrome - Left	1	1
7		47559X032-NLA	NLA - Shoe	1	1
8		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
9		47559X028-NLA	NLA - Handle, Wheel Lock, Chrome - Right	1	1
9		47559X029-NLA	NLA - Handle, Wheel Lock, Chrome - Left	1	1
10		04512X113-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
10		04512X114-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
11		1107321	Nut, Low Cap (5/16"-24)	1	1
12		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
13	A	1040829-NLA	NLA - Wheel Lock, Pull to Lock - Right	1	1
13	A	1040830-NLA	NLA - Wheel Lock, Pull to Lock - Left	1	1
NOTE: A - Pull to Lock; Includes Items 1-10					