


Crossbraces & Hardware

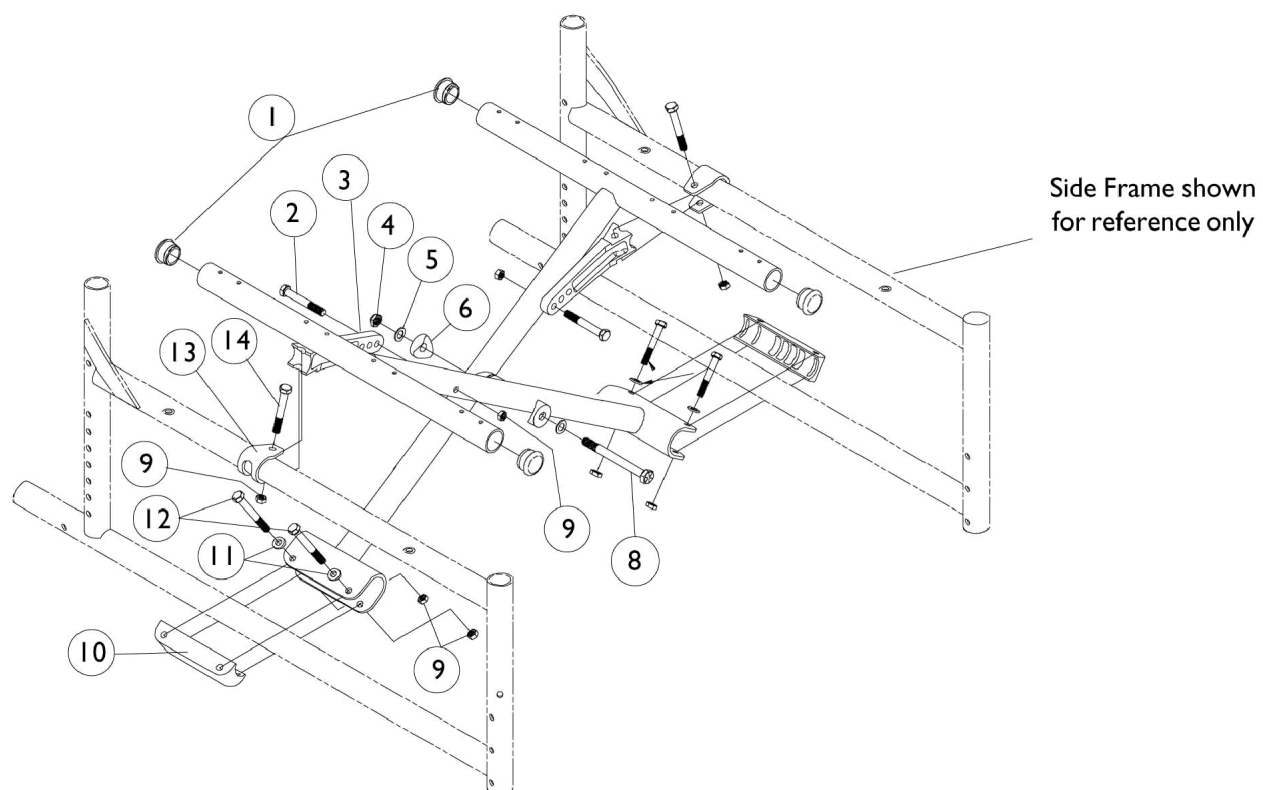

Crossbraces & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	4
2		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	2
3	A	1046501	Link, Pivot, Black (14 - 15" W)	1	2
3	B	1012173	Link, Pivot, Black (16 - 18" W)	1	2
3	C	1018238	Link, Pivot, Black (20 - 22" W)	1	2
4		1025730	Locknut (5/16-24)	1	1
5		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
7		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
8		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	1
9		1065362	Locknut (1/4-28)	1	8
10		1012172-NLA	NLA - Clamp, Saddle	1	2
11		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
12		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	4
13		1012180	Bracket, Pivot, Black	1	2
14		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
	D	See Chart	Crossbraces without Hardware	1	1
NOTE: A - 14" Wide B - 16-18" Wide C - 20" Wide D - Not Shown.					

Crossbraces without Hardware

Depth	14" Wide	16" Wide	17" Wide	18" Wide	20" Wide
16"	1046107-NLA	1016250	1037547-NLA	1016251	1038381-NLA
18"	N/A	1053344-NLA	1053345-NLA	1053346	1053347