


# Half Arm & Hardware

(Half Arm NLA on chairs built after 6/23/99)

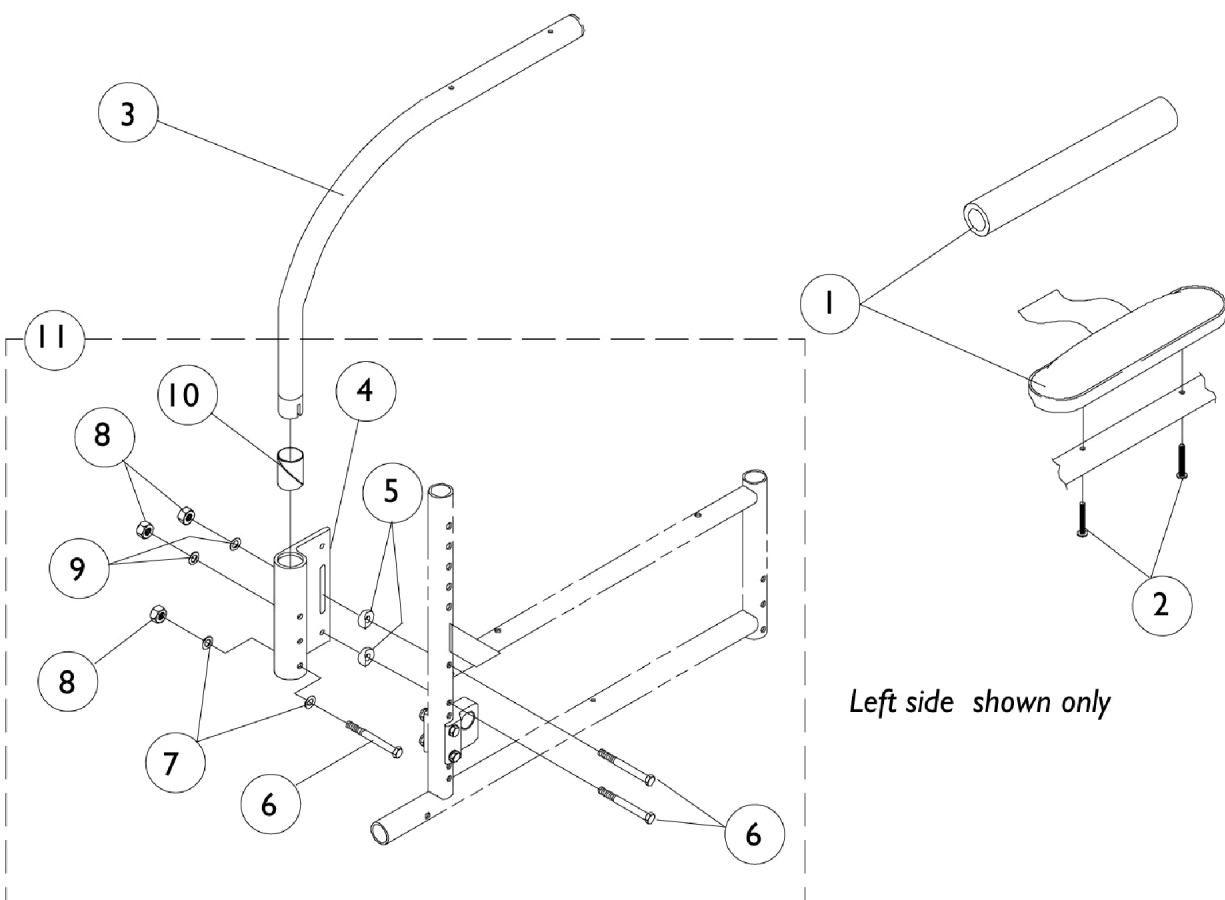

*Left side shown only*

## Half Arm & Hardware

### (Half Arm NLA on chairs built after 6/23/99)

| Item Number | Note | Part Number | Description                                | Quantity Per |          |
|-------------|------|-------------|--------------------------------------------|--------------|----------|
|             |      |             |                                            | Package      | Assembly |
| 1           | A    | 8881053570  | Kit, Arm Pad w/ Screws, Desk Length (Pair) | 2            | 1        |
| 1           |      | 1032244     | Grip, Foam Hand (9" L)                     | 1            | 1        |
| 2           |      | 1026107     | Screw, Phillips Pan Head (#10-32 x 1-3/8") | 1            | 2        |
| 3           |      | 1069815-NLA | NLA - Half Arm Assembly, Black - Right     | 1            | 1        |
| 3           |      | 1069816-NLA | NLA - Half Arm Assembly, Black - Left      | 1            | 1        |
| 4           |      | 1032206-NLA | NLA - Socket, Arm, Rear - Right            | 1            | 1        |
| 4           |      | 1032207-NLA | NLA - Socket, Arm, Rear - Left             | 1            | 1        |
| 5           |      | 1009861     | Spacer, Coved (1/4 x 11/16 x 1/2")         | 1            | 2        |
| 6           |      | 1055348     | Screw, Hex Head (1/4-28 x 1-3/4")          | 1            | 3        |
| 7           |      | 1026907     | Washer (1/4 x 1/2 x 1/16")                 | 1            | 2        |
| 8           |      | 7930334-NLA | NLA - Locknut (1/4-28)                     | 25           | 3        |
| 9           |      | 1026900     | Washer (1/4 x 23/32 x 1/16")               | 1            | 2        |
| 10          |      | 1018687     | Sleeve, Bearing                            | 1            | 1        |
| 11          |      | 1027062     | Kit, Half Arm Mounting Hardware            | 2            | 1        |

NOTE: A - Specify upholstery color, see page titled, "Upholstery Color Options" when ordering.

Half-Arms NLA on chairs built after 6/23/99.