


Conventional Adjustable Height Arms - Removable/Non-Removable

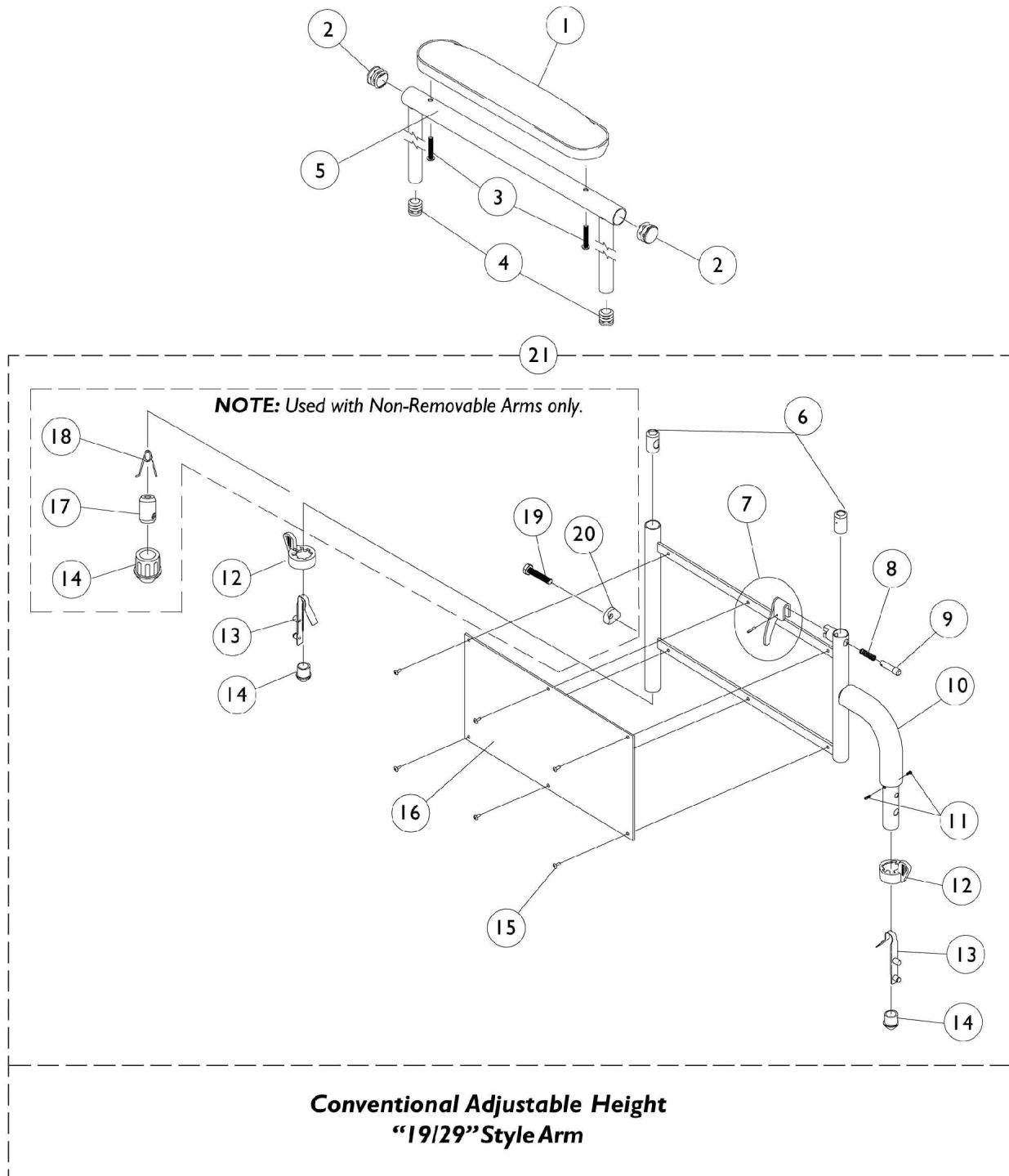

Conventional Adjustable Height Arms - Removable/Non-Removable

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
1	D	1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
1	D	1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
1	D	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1	D	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
4		1035614	Plug Button, Black (5/8")	1	2
5		1038470	Arm Tube Assembly, Conventional w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
5		1068576	Arm Tube Assembly, Conventional w/ Plug Buttons, Full Length, Black - Right/Left	1	1
6		1036150	Anti-Rattle	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		47471X074	Spring, Arm Lock	1	1
9		47471X068	Pin, Locking	1	1
10		1037718	Grip, Hand, Foam (5 1/2")	1	1
11		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
12		1015450	Handle, Quad Release	1	2
13		1018225	Spring, Torsion with Button Lock	1	2
14		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
15		1028645	Rivet	1	6
16		1064129	Guard, Clothing	1	1
17		1005834	Insert, Threaded	1	1
18		40188X002	Spring, Release	1	1
19		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	1
20		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
21	B	1036783	Arm Base Assembly, Conventional Adjustable Height - Right	1	1
21	B	1036784	Arm Base Assembly, Conventional Adjustable Height - Left	1	1
21	C	1040330-NLA	NLA - Arm Base Assembly, Space Saver, Black - Right/ Left	2	1

NOTE: A - Specify upholstery color, see page titled, "Upholstery Color Options" when ordering.
B - Removable
C - Non-Removable, Includes items 6-20
D - Urethane Waterfall Armpads were changed from Upholstered Waterfall Armpads on or around 01/03/2002.

19 & 29 Style Arms have a height range of 10-14".