
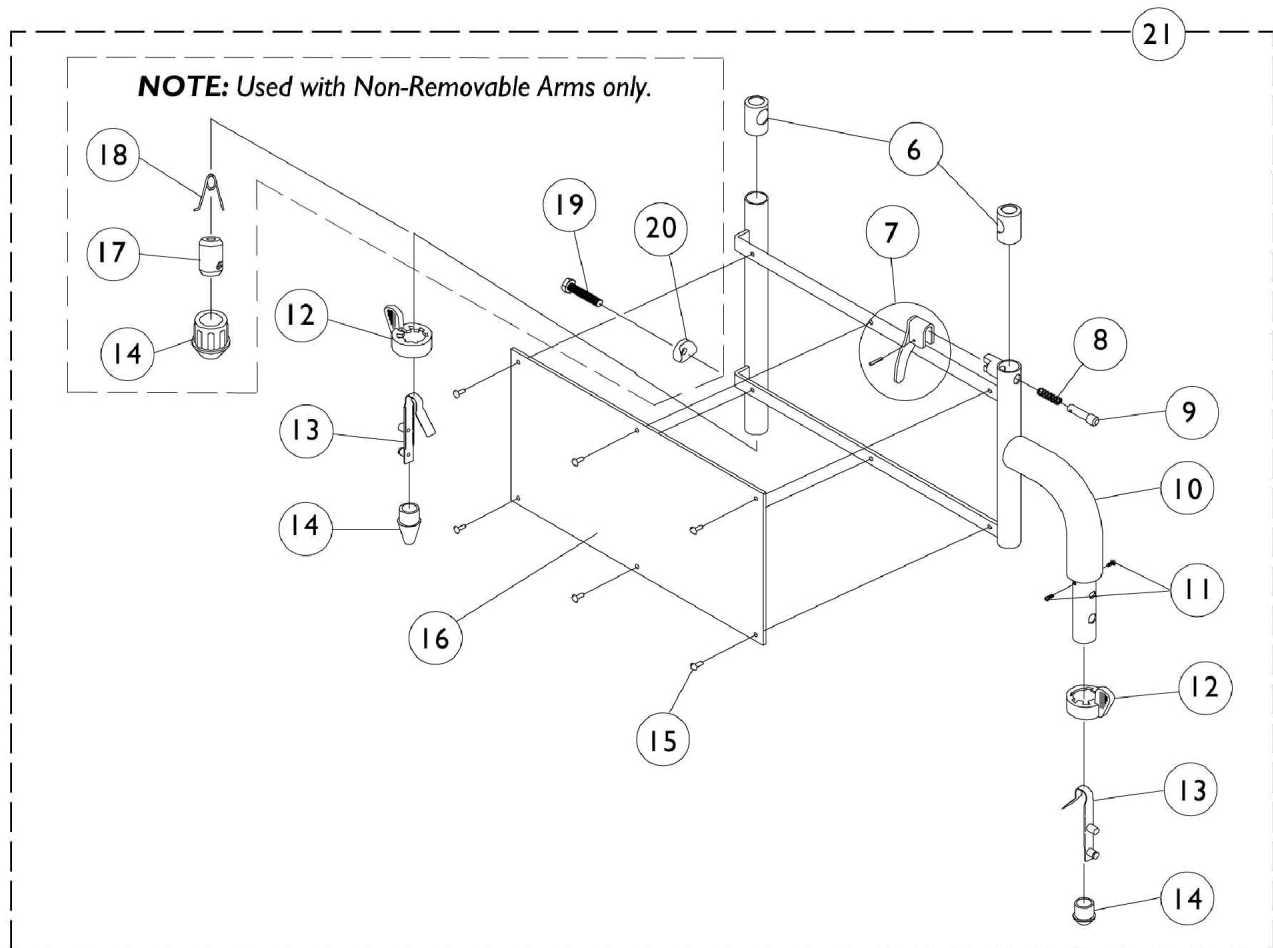


Space Saver Adjustable Height Arms - Removable/Non-Removable 77, 79, 87, and 89 Style Arms

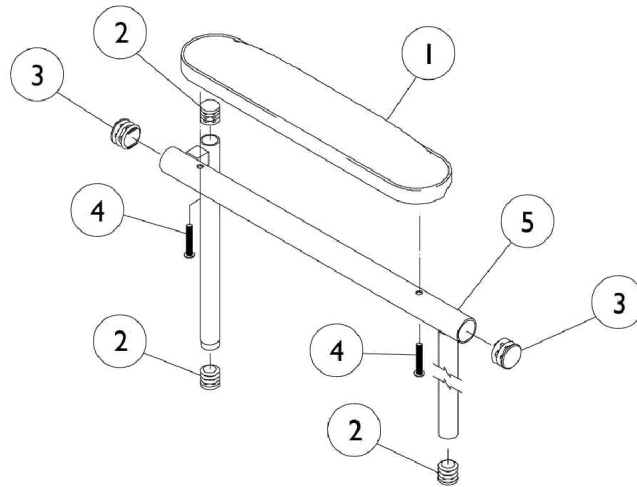

Space Saver Adjustable Height Arms - Removable/Non-Removable 77, 79, 87, and 89 Style Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
1	F,H	1102585-NLA	NLA - Pad, Arm, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
1	F,G	1102586-NLA	NLA - Pad, Arm, Desk Length, Urethane Waterfall, Black	1	1
1	F	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1	F	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
1	F	1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
1	F	1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
2		1035614	Plug Button, Black (5/8")	1	3
3		1027095	Plug Button, Black (7/8")	1	2
4		1034083-NLA	NLA - Package, Screw Phillips Round Head (#10-32 x 1-1/8")	2	2
5		See Chart	#5 Space Saver Arm Tube with Plug Buttons	1	1
6		1036150	Anti-Rattle	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		47471X074	Spring, Arm Lock	1	1
9		47471X068	Pin, Locking	1	1
10		1037718	Grip, Hand, Foam (5 1/2")	1	1
11		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
12		1015450	Handle, Quad Release	1	2
13		1018225	Spring, Torsion with Button Lock	1	2
14		1036484	Plug, Armrest, Black (3/4") (18GA)	1	1
14		1036484	Plug, Armrest, Black (3/4") (18GA)	1	1
15		1028645	Rivet	1	6
16	E	1064129	Guard, Clothing	1	1
16	D	1068511	Guard, Clothing	1	1
17		1005834	Insert, Threaded	1	1
18		40188X002	Spring, Release	1	1
19		1025708-NLA	NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
21	B	1048323-NLA	NLA - Arm Base Assembly, Space Saver - Right	1	1
21	B	1048324-NLA	NLA - Arm Base Assembly, Space Saver - Left	1	1
21	C	1040330-NLA	NLA - Arm Base Assembly, Space Saver, Black - Right/ Left	1	1
21	D	1037634	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Right	1	1
21	D	1037635	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Left	1	1

Space Saver Adjustable Height Arms - Removable/Non-Removable 77, 79, 87, and 89 Style Arms

NOTE: A - Specify upholstery color, see page titled "Upholstery Color Options" when ordering
 B - Removable
 C - Non-Removable, Includes items 6-20
 D - Low Space Saver Removable Arm
 E - Removable and Non-Removable Arms
 F - Urethane Waterfall Armpads were changed from Upholstered Waterfall Armpads on or around 01/03/2002.
 G - No longer available, use 1110817 as its replacement.
 H - No longer available, use 1110818 as its replacement.

79 & 89 Style Arms have a height range of 10 to 14".

77 & 87 Style Arms have a height range of 8-1/2 to 11-1/2".

#5 Space Saver Arm Tube with Plug Buttons

Black

Arm Type	Right	Left
Space Saver Adj. Height Desk Length	1038462	1038463
Space Saver Adj. Height Full Length	1038466	1038467
Low Space Saver Adj. Height Desk Length	1043315-NLA	1043316-NLA
Low Space Saver Adj. Height Full Length	1043319-NLA	1043320-NLA