
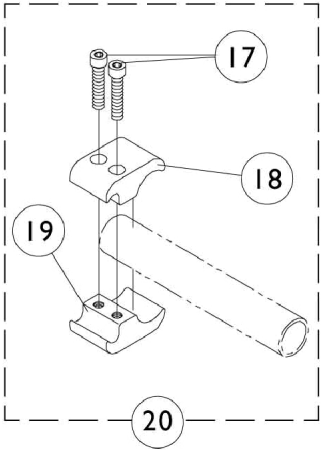


High Mount Wheel Lock w/ Hill Holder

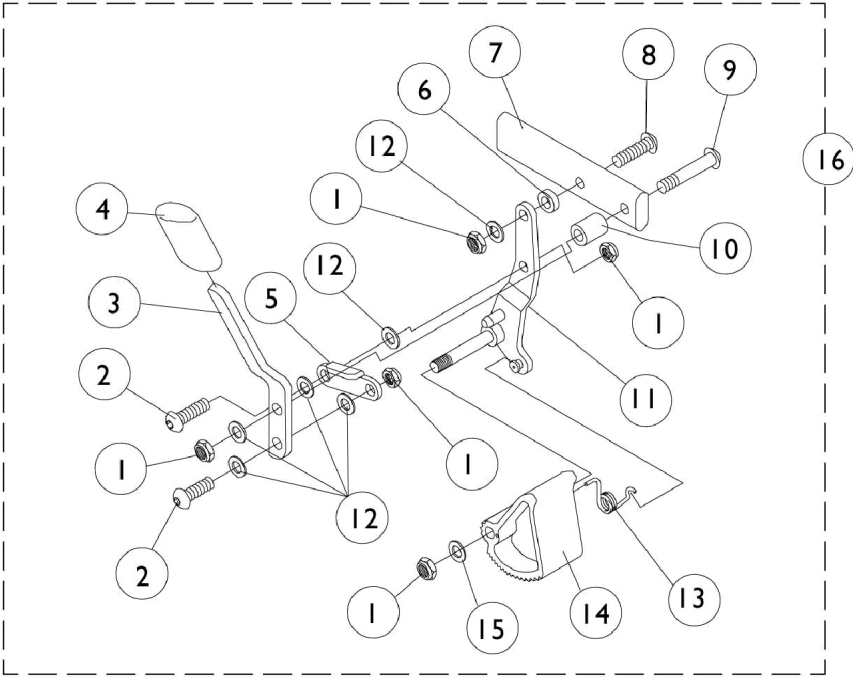

High Mount Wheel Lock w/ Hill Holder

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930334-NLA	NLA - Locknut (1/4-28)	25	5
2		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	2
3		1038344-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
3		1038345-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
4		1006490	Tip, Plastic, Black	1	1
5		1018727-NLA	NLA - Link, Stop, Black	1	1
6		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
7		1028399-NLA	NLA - Strap, Wheel Lock	1	1
8		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
9		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11		1026829-NLA	NLA - Bracket Assembly, Black - Right	1	1
11		1026830-NLA	NLA - Bracket Assembly, Black - Left	1	1
12		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
13		1016650-NLA	NLA - Spring, Torsion - Right	1	1
13		1016651-NLA	NLA - Spring, Torsion - Left	1	1
14		1015857	Shoe, Wheel Lock	1	1
15		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	1
16	B	1039323	NLA - Hill Holder Assembly w/ Hardware - Right	1	1
1			Locknut (1/4-28)	1	5
2			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
3			NLA - Handle, Wheel Lock, Black - Right	1	1
4			Tip, Plastic, Black	1	1
5			NLA - Link, Stop, Black	1	1
6			NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
7			NLA - Strap, Wheel Lock	1	1
8			NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
9			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11			NLA - Bracket Assembly, Black - Right	1	1
12			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
13			NLA - Spring, Torsion - Right	1	1
14			Shoe, Wheel Lock	1	1
15			Washer, Nylon (1/4 x 7/16 x 5/64")	1	1
16	B	1039324	NLA - Hill Holder Assembly w/ Hardware - Left	1	1
1			Locknut (1/4-28)	1	5
2			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
3			NLA - Handle, Wheel Lock, Black - Left	1	1
4			Tip, Plastic, Black	1	1
5			NLA - Link, Stop, Black	1	1
6			NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
7			NLA - Strap, Wheel Lock	1	1
8			NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
9			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11			NLA - Bracket Assembly, Black - Left	1	1
12			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7

High Mount Wheel Lock w/ Hill Holder

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
13	A	1027268 1038114-NLA 1038113 1039343	NLA - Spring, Torsion - Left	1	1
14			Shoe, Wheel Lock	1	1
15			Washer, Nylon (1/4 x 7/16 x 5/64")	1	1
17			Screw, Allen Head (1/4-20 x 1")	1	2
18			NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	1
19			Clamp, Wheel Lock, Threaded (7/8")	1	1
20			Kit, Clamp Wheel Lock Hardware (Pair)	2	1
			NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	2
			Clamp, Wheel Lock, Threaded (7/8")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
NOTE: A - In order to assemble, the non-threaded clamp is above the seat rail or frame and the threaded is below the seat rail or frame. B - Includes items 1 - 15.					