


High Mount, Clamp on Wheel Lock

Push to Lock

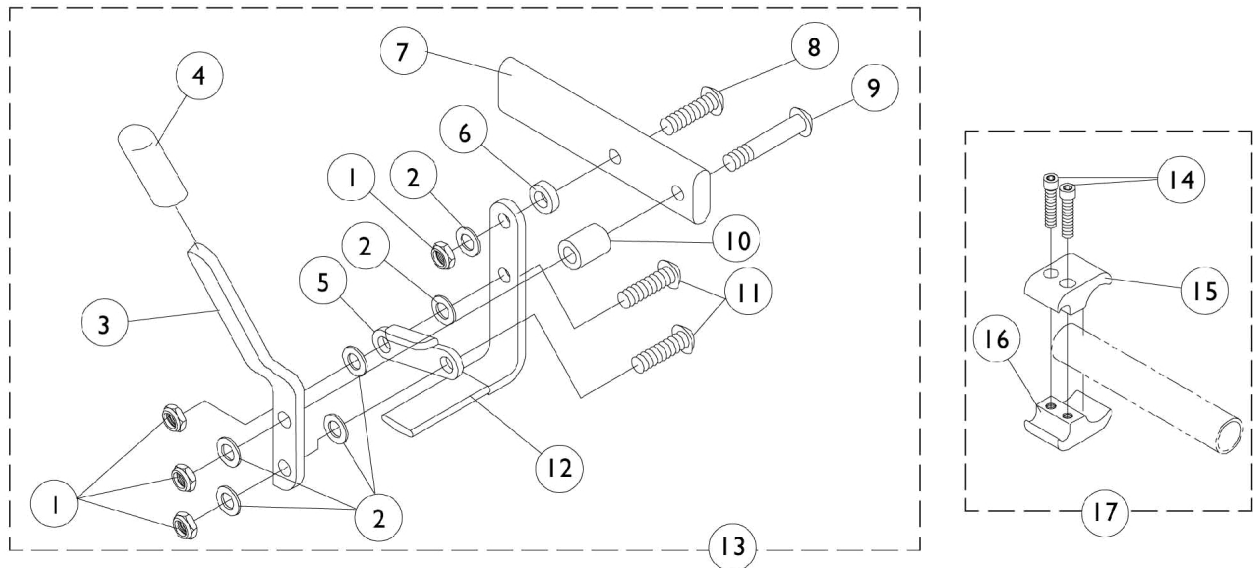

High Mount, Clamp on Wheel Lock Push to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930334-NLA	NLA - Locknut (1/4-28)	25	4
2		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
3		1038344-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
3		1038345-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5		1018733-NLA	NLA - Link, Stop, Black	1	1
6		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
7		1028399-NLA	NLA - Strap, Wheel Lock	1	1
8		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
9		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	2
12		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
13	A	1029423	Wheel Lock, High Mount, Push to Lock - Right	1	1
13	A	1029424	Wheel Lock, High Mount, Push to Lock - Left	1	1
13	B	1047703	Wheel Lock, High Mount, Push to Lock - Right	1	1
13	B	1047704	Wheel Lock, High Mount, Push to Lock - Left	1	1
14		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
15		1038114-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	1
16		1038113	Clamp, Wheel Lock, Threaded (7/8")	1	1
17		1039343	Kit, Clamp Wheel Lock Hardware (Pair)	2	1
			NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	2
			Clamp, Wheel Lock, Threaded (7/8")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
NOTE: A - Used on chairs manufactured Before 6/94. Includes items 1 - 12. B - Used on chairs manufactured After 5/94. Includes items 1 - 12.					